

Article

Attachment for amputation supp right/left
 Amputations Mech R or L
 Amputation Support 25x21
 Amputation rest mechanism
XX: Color codes can be found in the "Seat systems" folder.

Art.nr.

800786
 AS13
 AS10XX
 800827

Necessary tools

Allen key 6 mm
 Ring spanner 13 mm

Instructions

Mount the attachment for amputations support(1) on the front of the seat frame using the supplied screws. Observe, if the angle of the seat is adjustable, the attachment is to be mounted in the upper half of the seat frame. Fasten using lock nut. Spacer should be used on both sides. The amputation support(2) is mounted in the opening(3) and is kept in place with screws(4). Adjust the support in height(5) and angle(6) so that the support is not above the seat plate and the angle doesn't cause a straight knee.

