

Legsupport bracket Medic

REAL 9000 PLUS, REAL 6100 PLUS

Art.nr: 802896
 Changed: 1608
 Valid from: 0706

Article	Art.nr.
Legsupp.bracket Medic f B37 f seatdepth \geq 48	LS109001
Legsupp.bracket Medic f B37 f seatdepth \leq 44	LS109002
Legsupp.bracket Medic f B25 f seatdepth \leq 48	LS109003
Necessary tools	
Allen key 5 mm alt. Allen key 6 mm	800897

Instructions

Mount the bracket(1),(2) or (3) with two M8 screws through the holes(4) on the bracket and through the corresponding holes on the seat frames front (Note that if the seat is angle adjustable, the bracket should be mounted on the upper part of the seat frame).

Mount the bracket for the larger seat sizes(1) by loosely fitting it to the seat frame and marking the position of the holes on the seat plate. Remove the bracket, drill the previously marked holes in the seat plate with a 10 mm drill. Insert the threaded inserts in the holes. Mount the bracket and secure with the supplied screws.

