

Article	Art.nr.
Foot brake for Plus base 20 mm shaft	802865
Necessary tools	
Allen key 5 mm	800897

Instructions

Place the brake on the brake shaft. Loosely mount the two brackets(2) with screws(1). Adjust the brake to a desired position where comfortable use is achieved. Make sure no other parts of the chair interfere with use of the brake. Tighten the screws firmly.

Make sure that the brake lever is removed or attached securely.

NOTE! The foot brake can only be mounted on “Base 9000 PLUS w reversed brake”

