

Back-/Stomachrest

REAL 9000 PLUS, REAL 6100 PLUS, REAL 2000

Art.nr: 804694
 Changed: 1701
 Valid from: 9606

Article

For a complete list of back-/stomachrests, see the "Ergonomic Seat Systems" folder.

Necessary tools

Allen key 5 mm

Art.nr.

800897

Instructions

To mount a new back-/stomachrest, unscrew the screws (1) on the backside of the back-/stomachrest (three screws on the high back-/stomachrest or two screws on the low back-/stomachrest) using an allen key.

Mount the new back-/stomachrest by fastening the screws (1).

