

Article	Art.nr.
Side Support Right mechanism PLUS	SS11
Side Support Left mechanism PLUS	SS12
Necessary tools	
Allen key 5 mm	800897

Instructions

Loosen the armrest and remove it from the armrest mechanism(1). Slide the side support on to the armrest(2). Secure by tightening the M6 screw at the rear of the side support mechanism. Put the armrest back into the armrest mechanism(3) and secure it by tightening the knob on the back of the armrest mechanism.

NOTE! To loosen the PLUS armrest, the sprint at the bottom of the armrest must be pushed in.

To loosen the foldable armrest, the fastening knob must be unscrewed as far as it goes.

