

Article

Table Tray 45x30 cm

Art.nr.

TA1020

Necessary tools

Allen key 5 mm

800897

Instructions

Mount the adapter(1) on the left or right U-bar(2). **NOTE! The oval holes should be facing away from the chair.** Mount the U-bar(2) in the seat frame. Adjust the width and fasten the U-bar. Secure the adapter in place by tightening the screws(3) with an Allen key. The depth placement of the tray is adjusted with the knob(4). The tray mechanism can be adjusted in height or be rotated using the knob(5). Underneath the tray is a knob(6) that locks the rotation of the tray. The tray mechanism can be mounted on either side of the chair.

Make sure all controls are tightened before putting load on the tray.

