

## Article

Trunk support straight Right  
 Trunk support straight Left\*  
 Trunk Support curved Right  
 Trunk Support curved left  
 Trunk Support Cushion 15x13

\* Shown in picture below

## Art.nr.

TS11  
 TS12  
 TS21  
 TS22  
 TS10XX

**XX = Color codes can be found in the  
 "Seat Systems" folder.**

## Necessary tools

Allen key 5 mm  
 Phillips head screwdriver

800897

## Instructions

Assemble the trunk support as shown in the picture below.

Fasten the screws(4) in the existing threads in the cushion. The cushion can be mounted to provide different depths.

Adjust the trunk supports height and width to a neutral setting before mounting it on the backrest. Find a position that suits the user and secure the trunk support to the backrest using the supplied screws.(3)

The trunk support can be adjusted for height using the screw(2) and width using the screw(1).

