

Push handle & push bar

REAL 9000 PLUS

Art.nr: 803415
 Changed: 1701
 Valid from: 1001

Article

Push handle PLUS Right+left
 Push bar for 48 PLUS-base
 Push bar for 43 PLUS-base

Art.nr.

PB1010
 PB1011
 PB2011

Necessary tools

Allen key 5 mm

800897

Instructions

The lower brackets(1) is mounted on the rear legs of the base as shown below. **NOTE! Castors does not need to be removed!** If the chair is equipped with rear base extenders, the brackets for push bar/handle is mounted on the extenders. Adjust the bar or handle for desired position, then tighten the screws(2) firmly. Adjust handle/bar for desired angle, then tighten the screws(3). If the chair has seat tilt, the handles or bar should be adjusted so that the backrest mechanism moves freely when the seat is tilted backwards. The handles or bar is height adjusted using the knobs(4).

