

Article	Art.nr.
Adapter legsupp., Cross R+L	AD4010
Adapter legsupp., Cross R	AD4011
Adapter legsupp., Cross L	AD4012
Necessary tools	
Allen key 5 mm	800897

Instructions

Mount the adapter(1) on the left or right U-bar(2) in the front or rear holes, depending on the size of the seat.

Note! The oval holes should be placed facing away from the chair.

Mount the U-bar(2) in the seat frame(3). Adjust for correct width and lock the U-bar in place with knob or screw(4). Lock the adapter(1) in place by tightening the screws(5) with an Allen key. Adjust for correct depth and fasten using the knob(6).

