

## Article

For a complete overview of the different sizes and types of seat options, see the "Seat Systems" folder.

## Necessary tools

Allen key 5 mm  
 Allen key 6 mm  
 Ring spanner 10 mm

## Art.nr.

800897

## Instructions

Start by detaching the chairs backrest.

Loosen the screws in central tube of the base and remove the upper part of the chair. Turn the upper part of the chair upside down and let it sit on the armrests. Remove the seat plate(1) by unscrewing the screws in the seat frame(2). Unscrew the screws keeping the seat frame and electric lift together(3) and detach the lift. The manual lift is mounted with the help of lock nuts, make sure to replace them when mounting a new lift.

Reverse the process to mount the new seat.

