

Article

Gas spring adjustment armrest

Art.nr.

SAR31

Necessary tools

Torx T25

Instructions

Mount the gas spring and the supplied parts as shown(1). **NOTE! Don't forget the locking pins!**

Place the control under the armrest as shown(2). The control attachment is placed next to the square tube, on the side facing away from the chair. Make sure the control is not sticking out in front of the armrest. Secure the attachment to the armrest using the supplied wood screws. The wire should run on the inside of the U-bar of the armrest mechanism.

Adjust the control so that the lever can be lifted about 2 mm before engaging the seat tilt by adjusting the nut below the control. Control the function of the seat tilt.

