

Article

Leg Divider Mech L 115 mm
 Leg Divider Mech L 175 mm
 Leg Divider Mech L 57 mm
 Knee Divider Mechanism
 Leg Divider Small 11x11 cm
 Leg Divider Large 15x15 cm

XX - Color codes can be found in the "Seat Systems" folder.

Art.nr.

LD16
 LD17
 LD15
 LD25
 LD10XX
 LD20XX

Necessary tools

Allen key 5 mm

800897

Instructions

Mount the attachment(1) to the front of the seat frame and secure with screws(2). Mount leg/knee divider as shown(A). Mount the mechanism in the attachment and secure it using the screws(3). Adjust for desired height and secure using screw(4). Adjust for desired depth and secure using screw(3).

