

Brackets H-harness

HB2540



ART. NR.	Article	Number of required pieces
804184	Bracket Upper for H-harness	2
800878	Attachment for hip belt, pair	1
800436	SPAX KXTB 5x20 black-chrome screw.H-belt	6

INSTRUCTIONS


1. Disassemble the upper brackets so all the components are separated.



2. Place the mount (G) for the upper bracket in the desired position.



3. To prepare for the threaded inserts (D), drill a 6 mm hole. To be sure the hole is in the right place, drill through the upper hole in the mount (G).

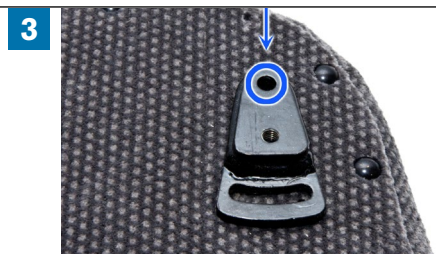

4. Remove the mount (G) and enlarge the hole using a 9,5 mm drillbit. Secure a threaded insert (D) in the hole using a 6 mm allen key. Screw until it is about 1 mm below the wood of the backrest.



5. Place the mount (G) over the threaded insert (D) and secure the mount with the screw (A) using a 5 mm allen key. Adjust the mount (G) as desired.



INSTRUCTIONS

6. Use a 6 mm drillbit to drill another hole in the track on the bottom of the mount (G).



7. Remove the mount for the upper bracket (G). Repeat step 4. The result should look like the picture shown to the right.



8. Place the mount (G) over the threaded inserts (D) and secure it with the screw (B) through the track on the bottom of the mount and into the lower threaded insert (D) using a 5 mm allen key.



9. Place the H-harness bracket (F) on top of the mount (G).



10. Place the steering disc (E) on top of the bracket (F).



11. Put the screw (A) through the upper hole of the disc (E), the bracket (F), the mount (G) and into the upper threaded insert (D). Do the same with screw (C) in the lower hole of the disc (E). Tighten the screws using a 5 mm allen key.



INSTRUCTIONS

12. Repeat steps 1-11 for the upper bracket on the opposite side of the backrest. The result should look like the picture to the right.

12



13. Assembly of the hip belt attachment (800878).

13



INSTRUCTIONS

14. Place the attachments as shown in the picture to the right.



15. Secure the attachments using the supplied SPAX(800436).



RECYCLING & DESTRUCTION

H-harness bracket

- sort as *steel*



Mercado Medic AB
Tryffelslingan 14
181 57 Lidingö
Sweden

Phone & Fax
P + 46 8 555 143 00
F + 46 8 555 143 99

Email & Web
E info@mercado.se
W www.mercado.se