

Article

Push Handle Clinic (Mounted under seat)

Art.nr.

PB1012

Necessary tools

Allen key 5 mm

800897

Instructions

Start off by turning the chair upside down, resting it on its armrests on top of a table.

Remove the backrest. Remove the backrest mechanism(1) by unscrewing the four screws(2). Place the attachment(3) for the push handle on the bottom of the seat plate. Re-mount the backrest mechanism(1) on top of the push handle attachment(3). Screw the MC65 6x25(2) tight. NOTE! Use the supplied screws, not the previously mounted screws.

Turn the chair right side up. Mount the backrest. Adjust the push handle to desired height using the knobs(5). Make sure to tighten the knobs firmly when the push handle is adjusted. Make sure that the push handle doesn't collide with other parts of the chair.

