

# Replacing Castor

REAL 9000 PLUS

Art.nr: 803189  
 Changed: 1701  
 Valid from: 0706

## Article

Castor 100 XL M12  
 Castor 100 SL M12  
 Castor 100 BXL M12  
 Castor 125 mm  
 Castor 150 mm

## Art.nr.

801213  
 801144  
 803162  
 800904  
 800905

## Necessary tools

Ring spanner 19 mm

## Instructions

Remove the old Castor using a 19 mm ring spanner too loosen the bolt(1).

An optional method is to remove just the wheel using two 5 mm allen keys and then remove the bracket with a socket wrench.

Mount the new Castor . The bolts should be secured using Loctite 243 (or a similar product) and a torque of 80-90 Nm.

