

Seat height control under seat

REAL 9000 PLUS

Art.nr: 803184
 Changed: 1701
 Valid from: 0706

Article

Seat height control under seat for seat angleadjustment,Adult
 Seat height control under seat for fixed seat angle,Adult

Art.nr.

802908
 802974

Necessary tools

Allen key 5 mm
 Ring spanner 13 mm

800897

Instructions

Adjust the height control with allen key(1) and a 13 mm ring spanner. Just loosen the bolts(2), adjust the control for desired position and tighten the bolts(2).

