

Legsupport Center-mounted

REAL 9000 PLUS, REAL 6100 PLUS

Artikelnr: 804570
Ändrad: 1706
Giltig från: 0601

Article	Art.nr.
Legsupp compl. Middle Length shallow L 30-38 Footr.B32xD24	LS232110
Legsupp compl. Middle Length shallow XL 37-53 Footr.B32xD2	LS232210
Legsupp compl. Middle Length shallow L 30-38 Footr. B25xD18	LS242110
Legsupp compl. Middle Length shallow XL 37-53 Footr B25xD18	LS242210
Legsupp compl. Middle Length shallow L 30-38 Footr.B32xD10	LS252110
Legsupp compl. Middle Length shallow XL 37-53 Footr.B32xD10	LS252210
Legsupp compl. Middle Length shallow L 30-38 Footr.B29xD24	LS272110
Legsupp compl. Middle Length shallow XL 37-53 Footr.B29xD24	LS272210

Necessary tools

Allen key 5 mm alt. Allen key 6 mm

Ring spanner 13 mm

Instructions

Mount the legsupport to the seatframe(1) using the screws(2). NOTE! Depending on the kind of seatframe, other kinds of screws and possibly nuts could be needed. The picture below depicts assembly on a seatframe w/o seat angle adjustment.

Adjust the length of the legsupport using the knob(3) so the users thigh rests on the seat. Adjust the angle of the footplate by adjusting the screw(4), using an Allen key. The feet should have full contact with the footplate. Change the angle of the legsupport by unscrewing the securing screw(5). Mount the screw(5) in one of the holes to decide the angle of the legsupport. Tighten the screws and knobs.

