

Article

LS109001 Legsupp. bracket Medic \geq 48 seatd.	LS161310 Legsupp. Mini Medic Div. L R+L 28-41 cm
LS109002 Legsupp. bracket Medic \leq 40 seatd.	LS111410 Legsupp. Mini Medic compl. XL 38-51 cm
LS151310 Legsupp. Medic Div. L 28-41 cm	LS151210 Legsupp. Medic Div. M 23-32 cm
LS151110 Legsupp. Medic Div. S 18-24 cm	LS151410 Legsupp. Medic Div. XL 38-51 cm
CS1038 Calf Support 15x13 Skai Black	802214 Calf supp. compl. Medic Legsupp. W37

Necessary tools

800897	Allen key 5 mm
	Allen key 6 mm

Instructions

For mounting and adjustment instructions of legsupport, see the assembly instructions for "Legsupport Medic".

The ring (1) has to be removed (NOTE! Two screws) to make the mounting of the calf support bracket (2) possible. Mount the bracket (2) and the ring (1). Mount the shaft (3) in the bracket (2). Loosen the screws (4) in the support attachment and mount it on the shaft (3). Tighten the screws (4).

Adjust the calf support for desired depth and height by turning the shaft (3). Adjust for maximum support.

