


# Battery replacement for 24V control system

## for REAL® 9000 PLUS

Doc. no: 20-06597  
Revision 02  
Amended 26/05/2021  
Valid from 01/01/2015

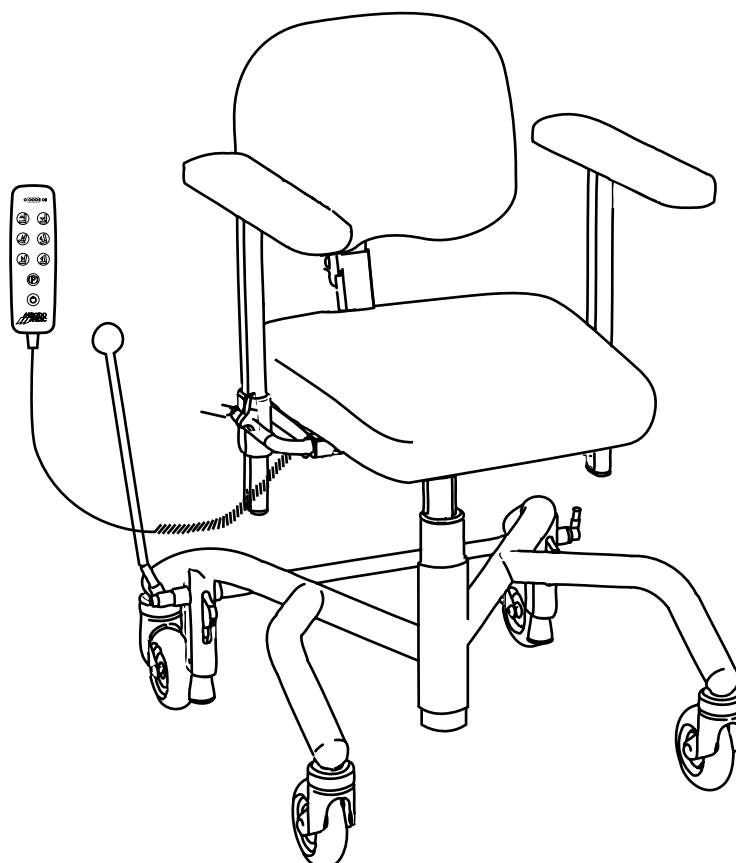

### PLEASE NOTE!

BEAR IN MIND THE FOLLOWING WHEN HANDLING BATTERIES:

- Never short-circuit the battery.
- Do not expose the battery to heavy impact or naked flames.
- In the event of bodily contact with battery acid, rinse with water for approx. 30 mins and seek medical advice.
- Used batteries should be taken to recycling centres.



Mercado Medic AB  
Tryffelslangan 14  
SE-181 57 Lidingö

Telephone & Fax  
Tel +46 (0)8-555 143 00  
Fax +46 (0)8-555 143 99

E-mail & Internet  
E-mail [info@mercado.se](mailto:info@mercado.se)  
Internet [www.mercado.se](http://www.mercado.se)

## AFFECTED ITEMS

805288 Electrical system control box 24V RCB10+L

805320 Battery pack 2.6Ah 24V f RCB10+L

## EQUIPMENT/TOOLS

BATTERY REPLACEMENT EQUIPMENT AND TOOLS:


- Torx screwdriver (TX15)
- Goggles
- Flat pliers or equivalent may be required

## PREPARATION

- Installing the backrest according to the user manual
- Turn the chair upside down so that it is resting on the armrests
- Remove the control box according to the installation instructions 'Bracket for control system 24V'


## BATTERY REPLACEMENT

1. Loosen the screws that hold the control box together.

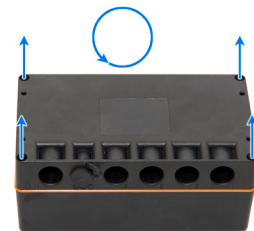

2. Open the control box and disconnect the plus and minus cables. **Please note! Flat pliers may be required for this step.**

Remove the old batteries.

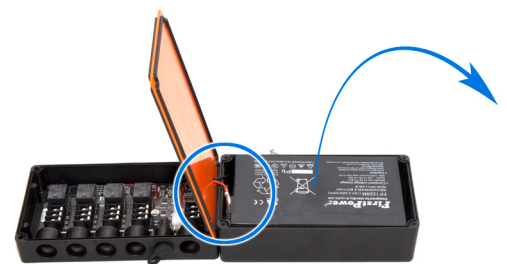

3. Insert the new batteries in the control box and connect the plus and minus cables. **Please note! Sparks may occur when plugged in; wear goggles.**

4. Close the control box, then insert and tighten the screws holding it together.

## FINAL ASSEMBLY

- Install the control box according to the installation instructions 'Bracket for control system 24V'
- Turn the chair to the upright position
- Install the backrest according to the user manual