




EN

# BATTERY REPLACEMENT FOR REAL 9000 PLUS 24V

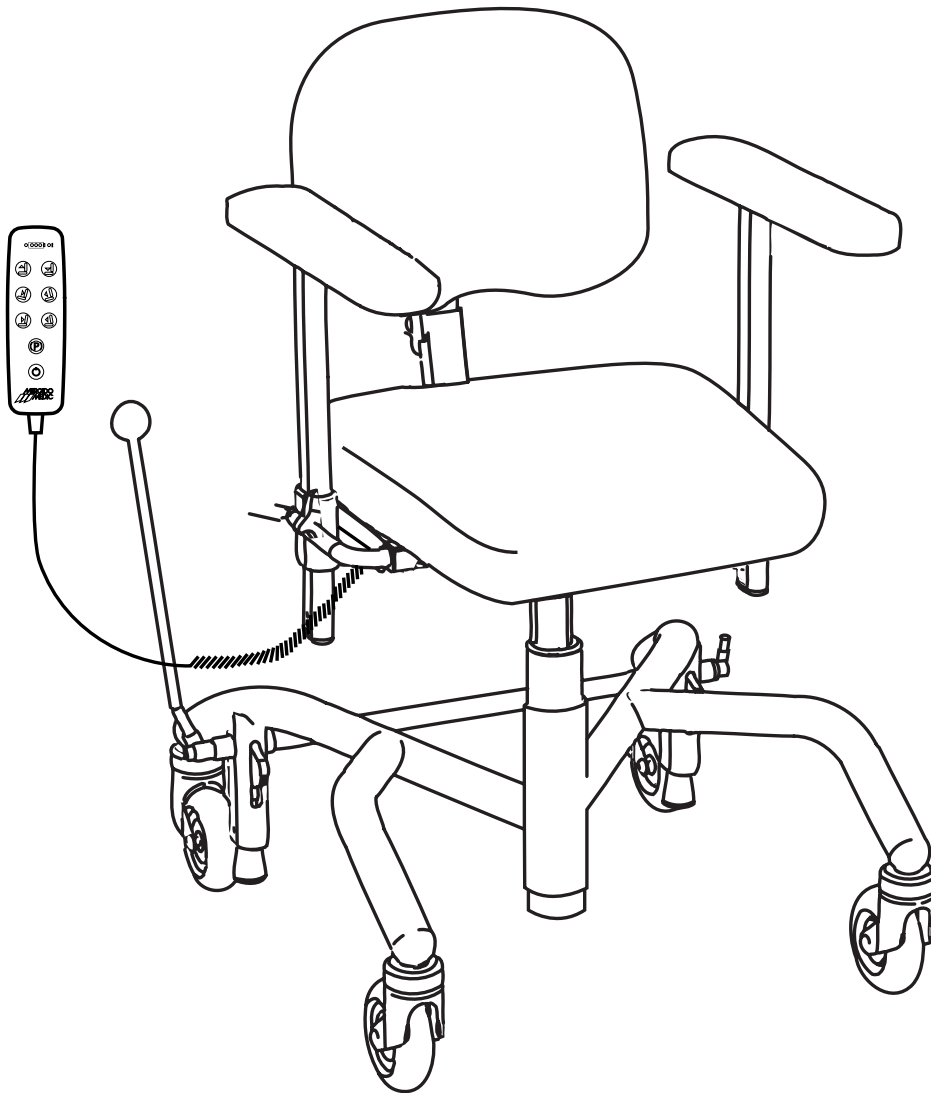

# 1. Important information

Bear in mind the following when handling batteries:

- Never short-circuit the battery.
- Do not subject the battery to strong shocks.
- Always dispose of used batteries at recycling centres.

## First aid

If the battery is defective and handling it results in direct contact with the contents of the battery, electrolytes (sulphuric acid), the following applies:

Incident	Action
Skin contact	Rinse with water. Clean with soap and water. Remove and wash affected clothes.
Inhalation of acid mist	Breathe fresh air. Seek medical attention immediately!
Contact with eyes	Wash eyes with running water for at least 15 minutes. Seek medical attention immediately!
When swallowed	Immediately drink plenty of water, do not induce vomiting. Seek medical attention immediately!

# 2. Item list

805288 Electrical system control box 24V RCB10+L

805320 Battery pack 2.6Ah 24V for RCB10+L

# 3. Necessary tools

- Torx screwdriver (TX15)
- Goggles
- Flat nose pliers or equivalent may be required


## 4. Assembly instructions

### 4.1. Preparations

- Dismount the backrest in accordance with instructions in the user manual
- Turn the chair upside down so that it is resting on the armrests
- Remove the control box according to the 'Bracket for control system 24V' installation instructions

### 4.2. Remove old batteries


- (A) Loosen the screws that hold the control box together.
- (B) Open the control box and disconnect the plus and minus cables; flat nose pliers may be needed for this step. Remove the old batteries.

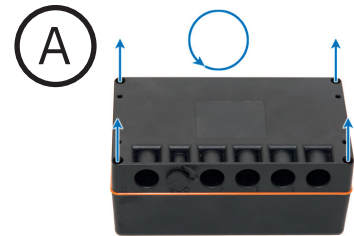

### 4.3. Fit new batteries

! Sparks may occur during connection; wear goggles.

Insert the new batteries in the control box and connect the plus and minus cables. Reseal the control box, then insert and tighten the screws holding it together.

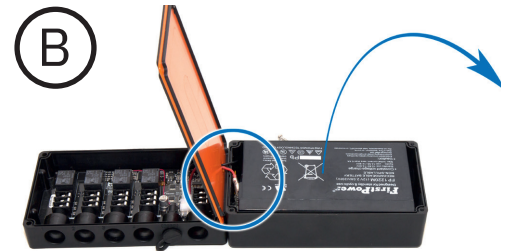

### 4.4. Final assembly

- Mount the control box in accordance with the 'Bracket for control system 24V' installation instructions
- Turn the chair to the upright position
- Mount the backrest in accordance with instructions in the user manual