
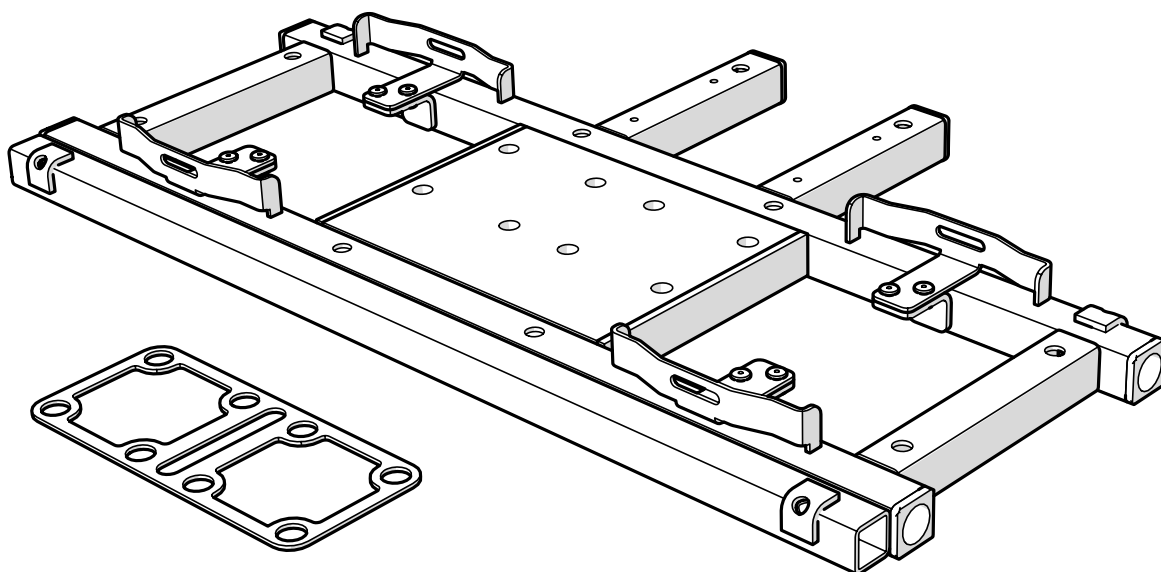


SITTKRYSSBYTE REAL[®] 9200 TWIN 12V

Seat frame change REAL[®] 9200 TWIN 12V

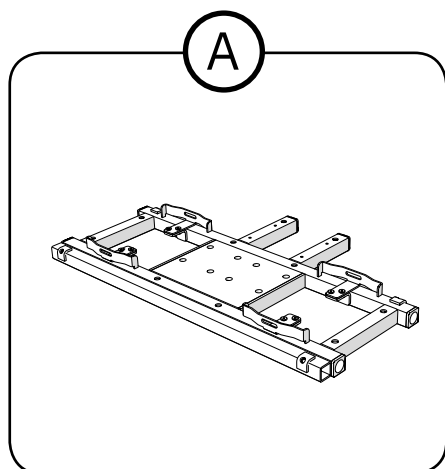


**A** 801873Sittkryss 9200 Twin Br52-61 12V
Seatframe 9200 Twin w52-61 12V

801434

Sittkryss 9200 Twin Br62-71 12V
Seatframe 9200 Twin w62-71 12V

801874

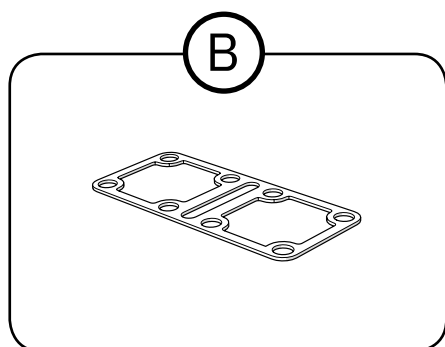Sittkryss 9200 Twin Br72-81 12V
Seatframe 9200 Twin w72-81 12V**B** 805065Shimsplåt 162x73x2 f REAL 9200 TWIN
Shims plate 162x73x2 f REAL 9200 TWIN





Svenska


Klämrisk när lyftomaterna monteras loss från stativet.


English


Pinching hazard when the lyftomats are disassembled from the base.

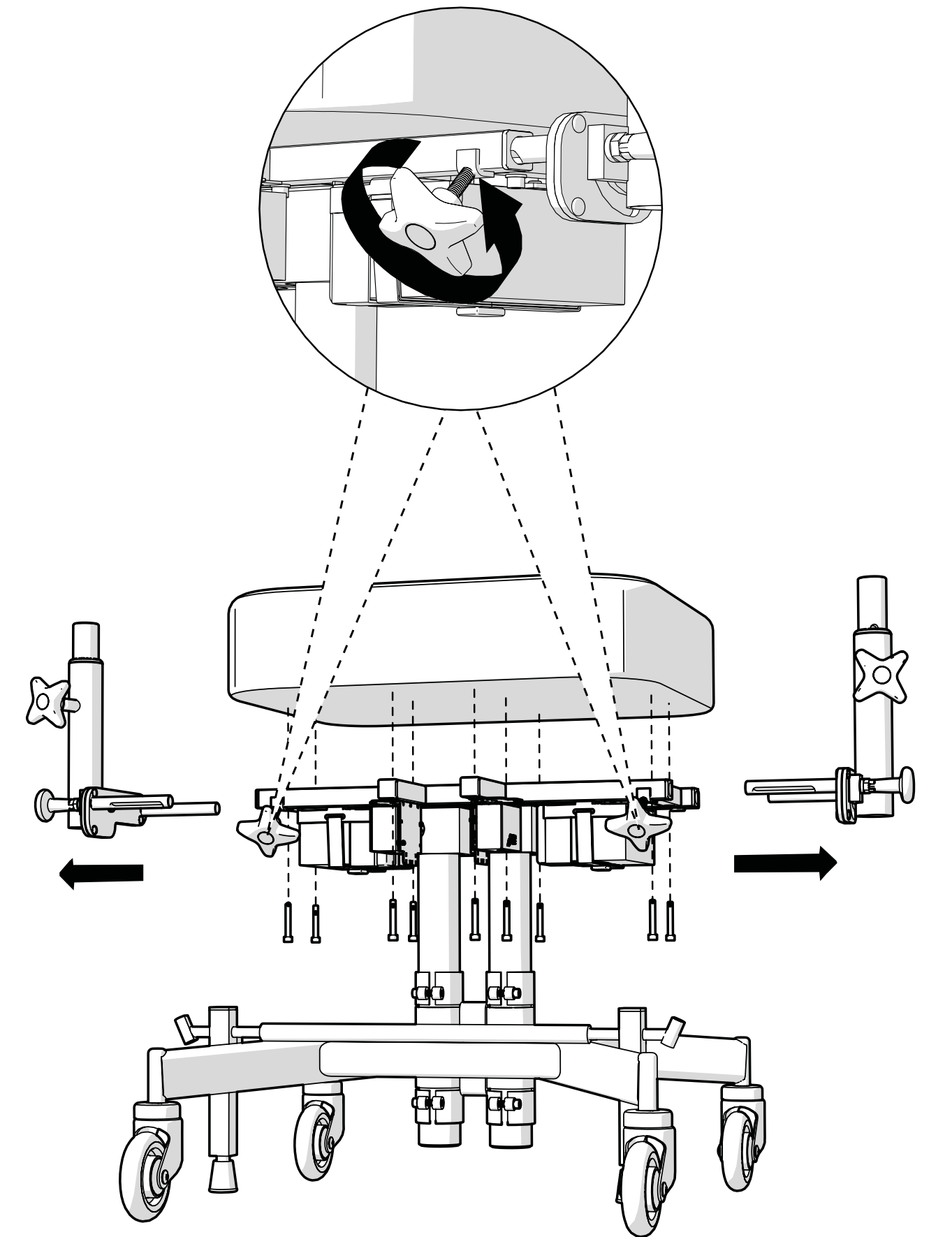


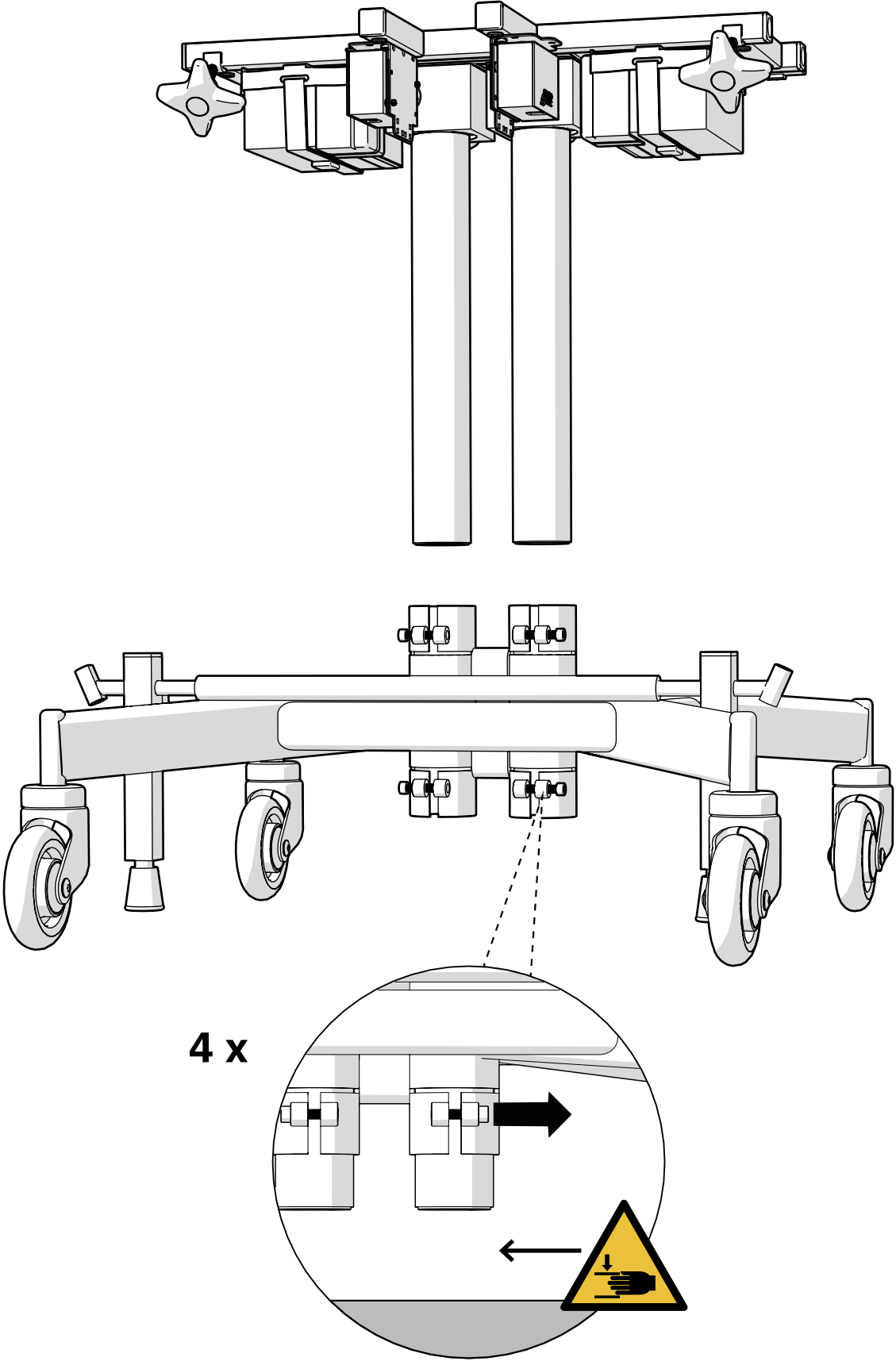


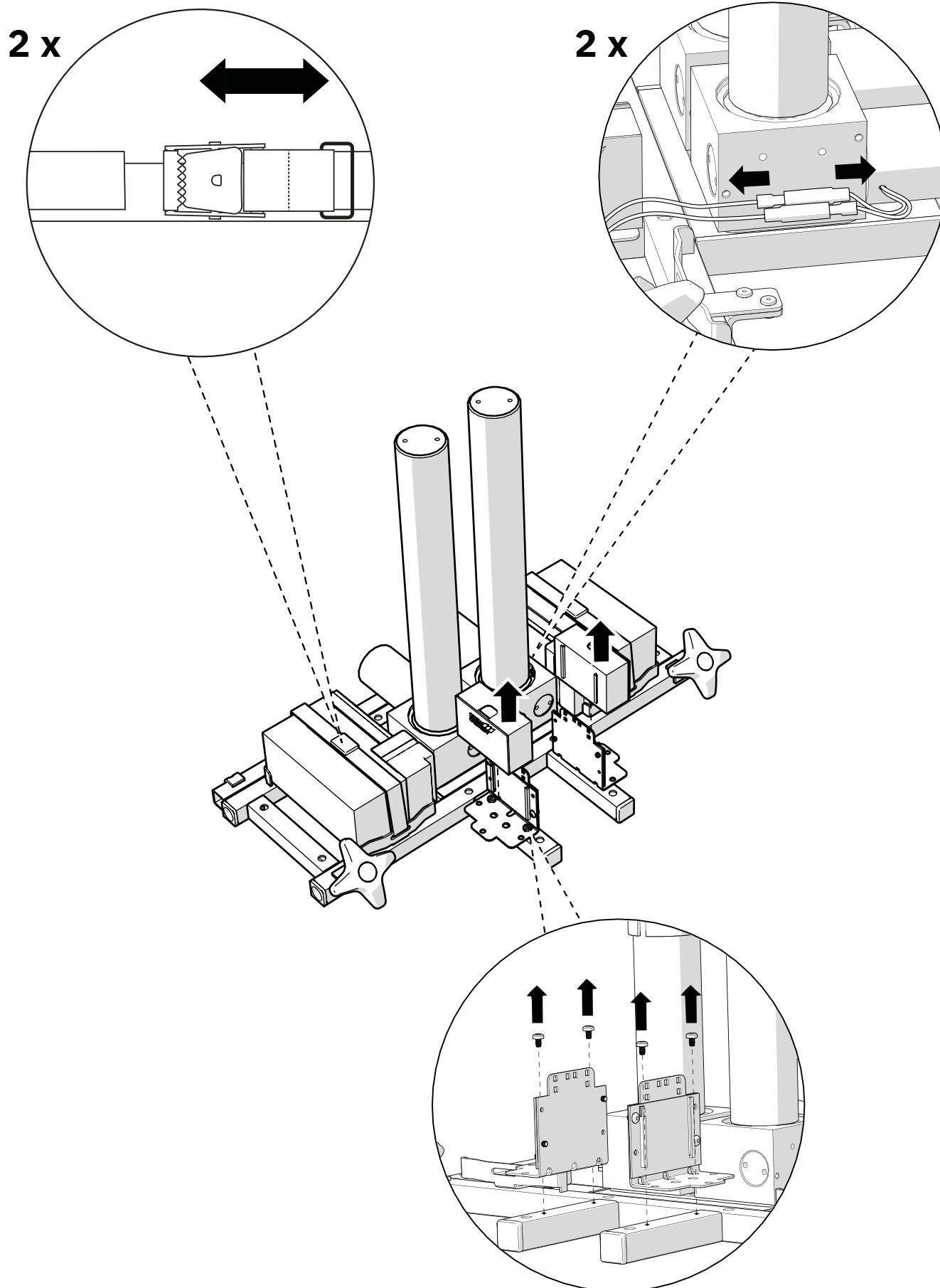


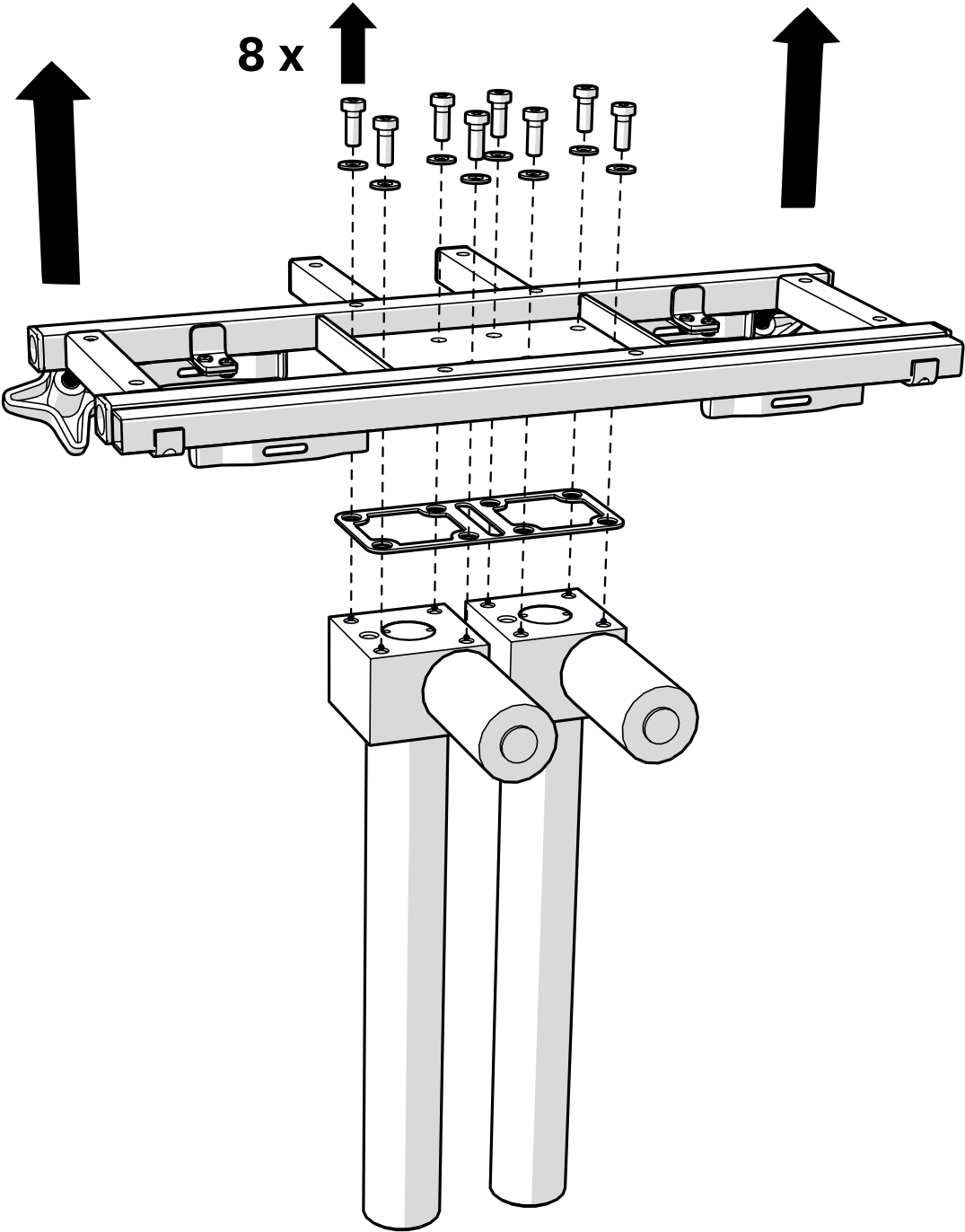


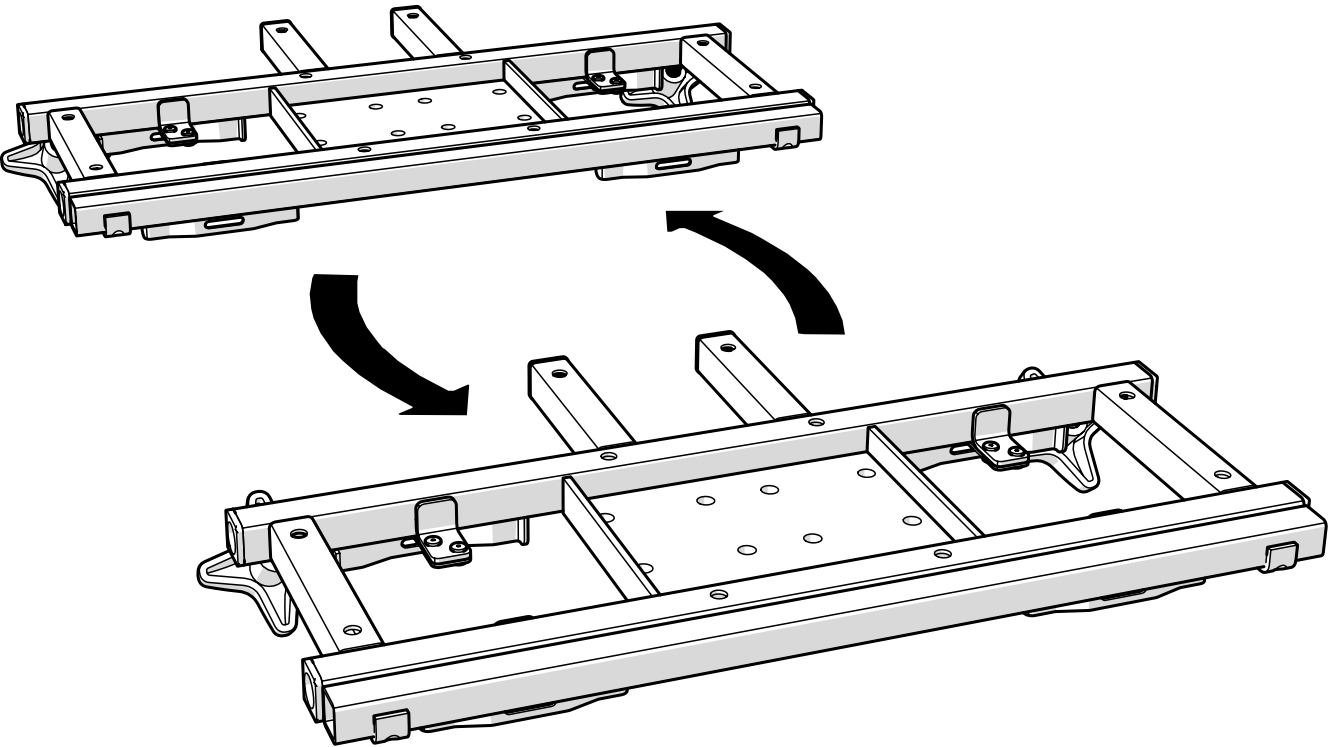


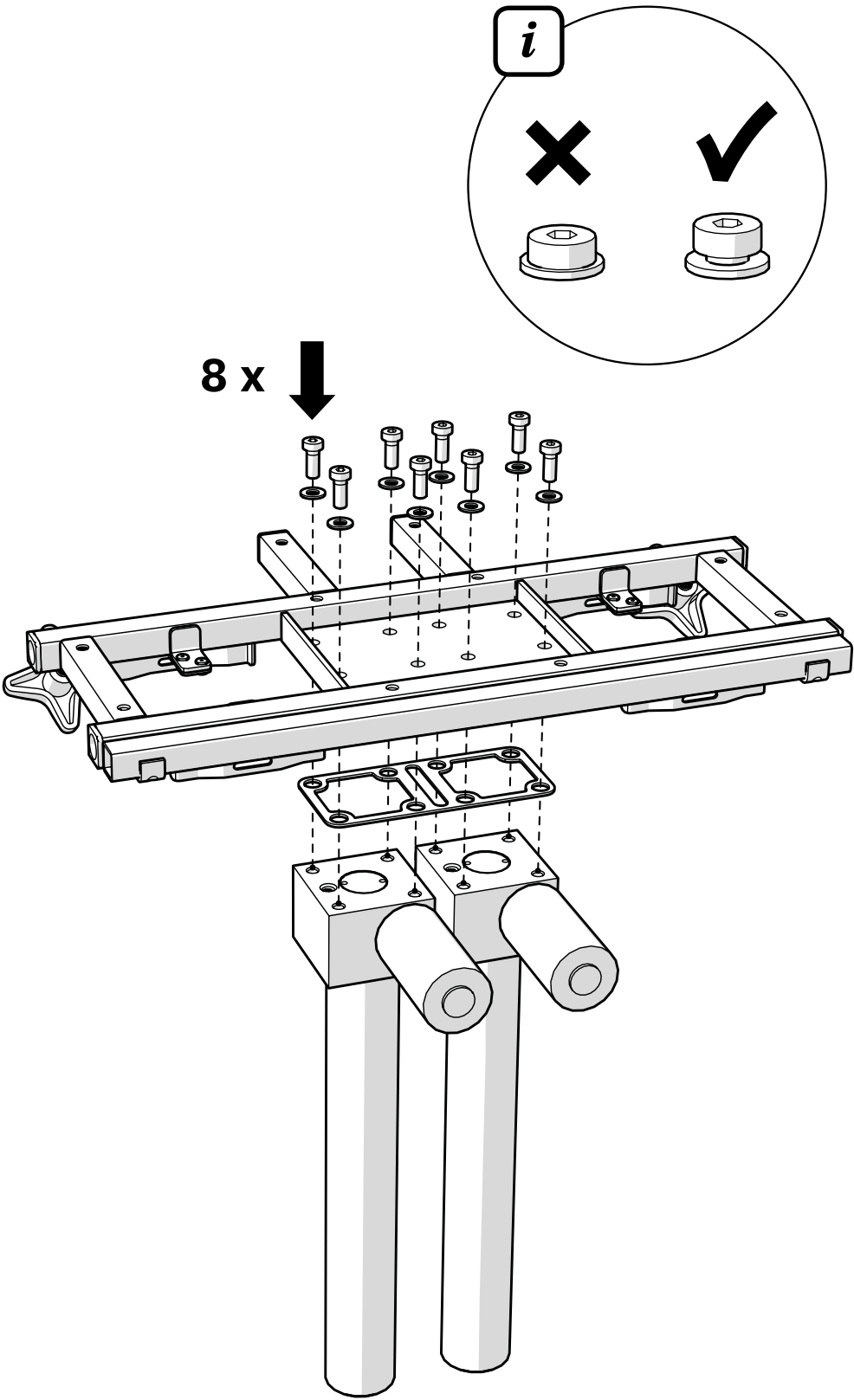


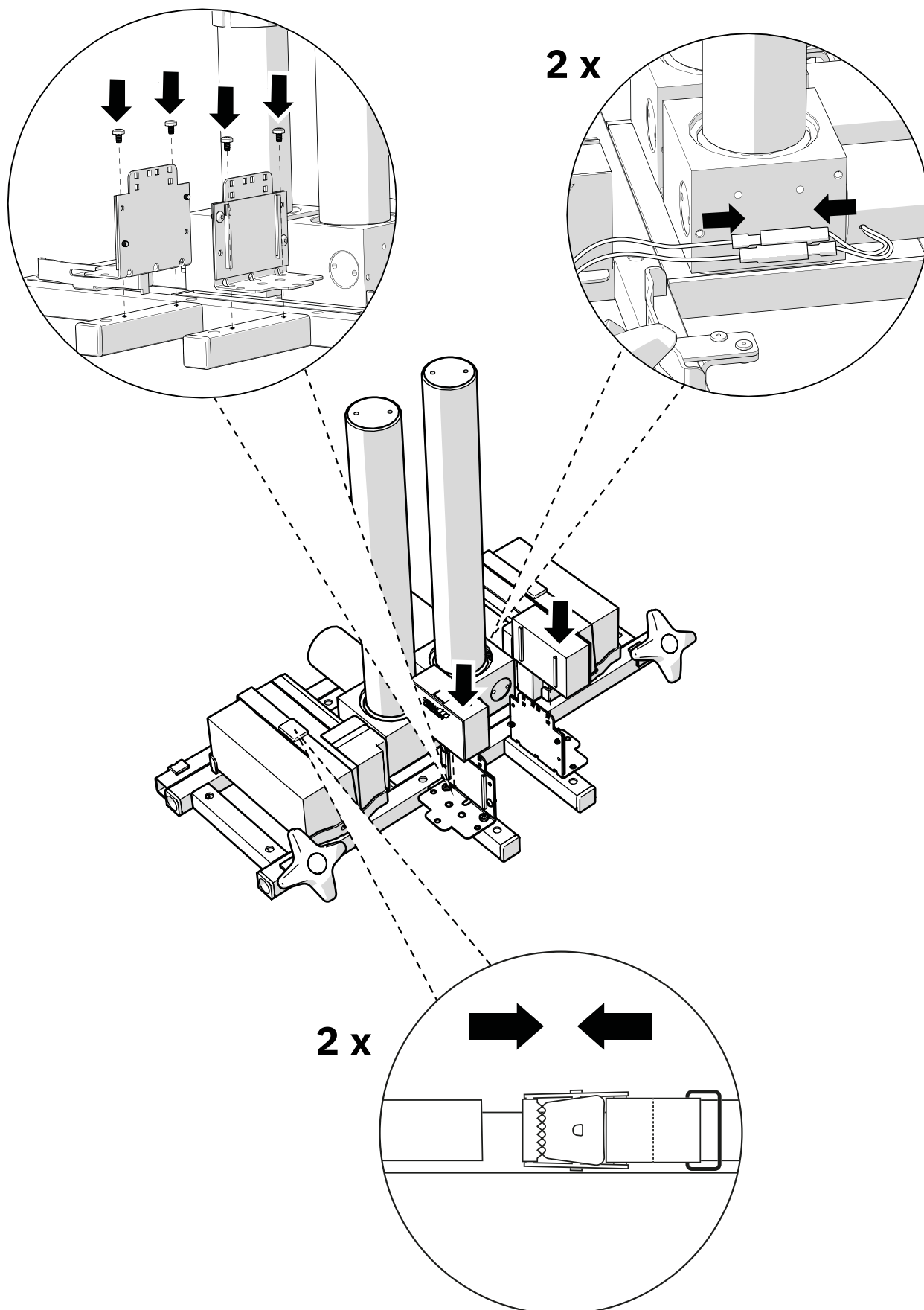

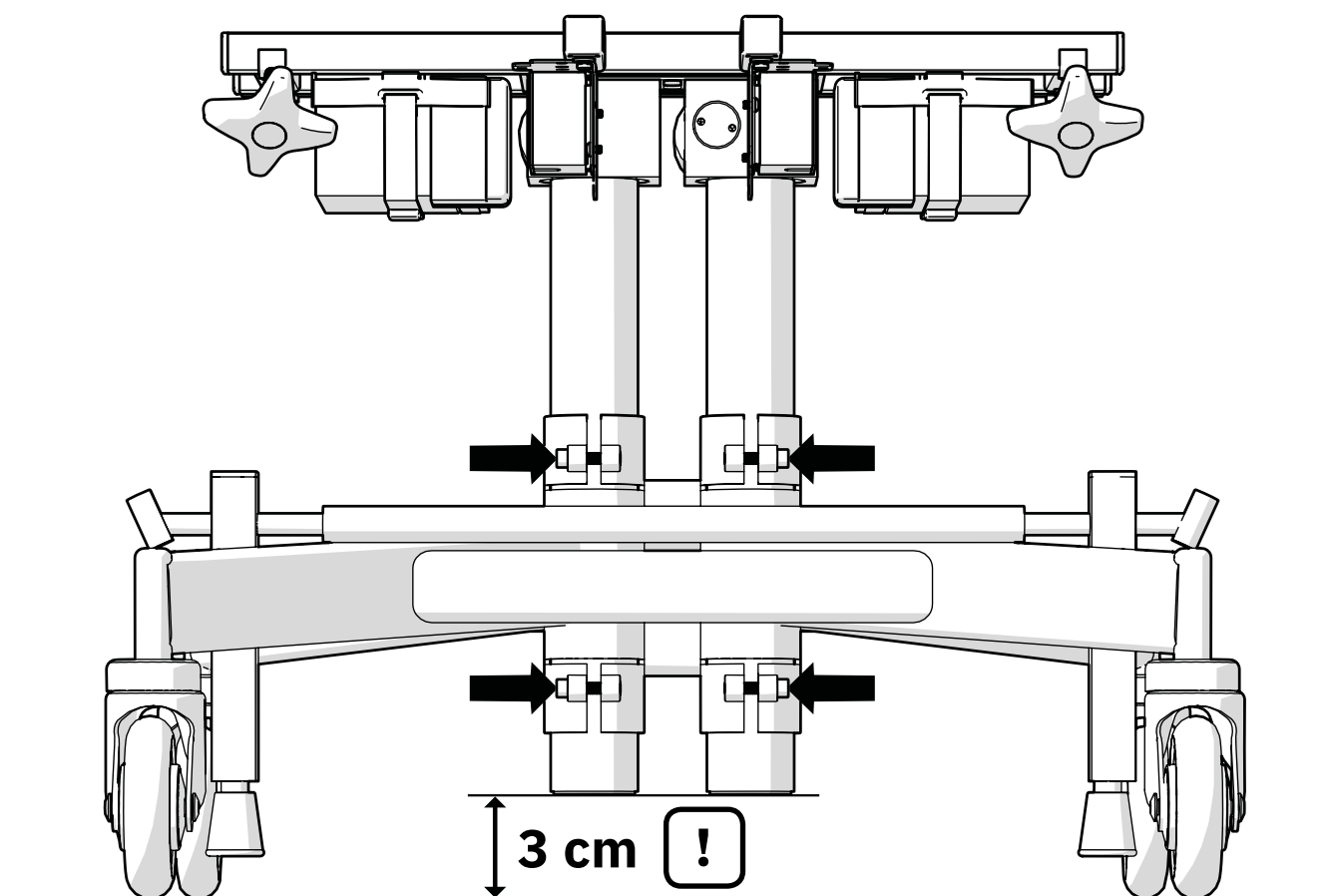

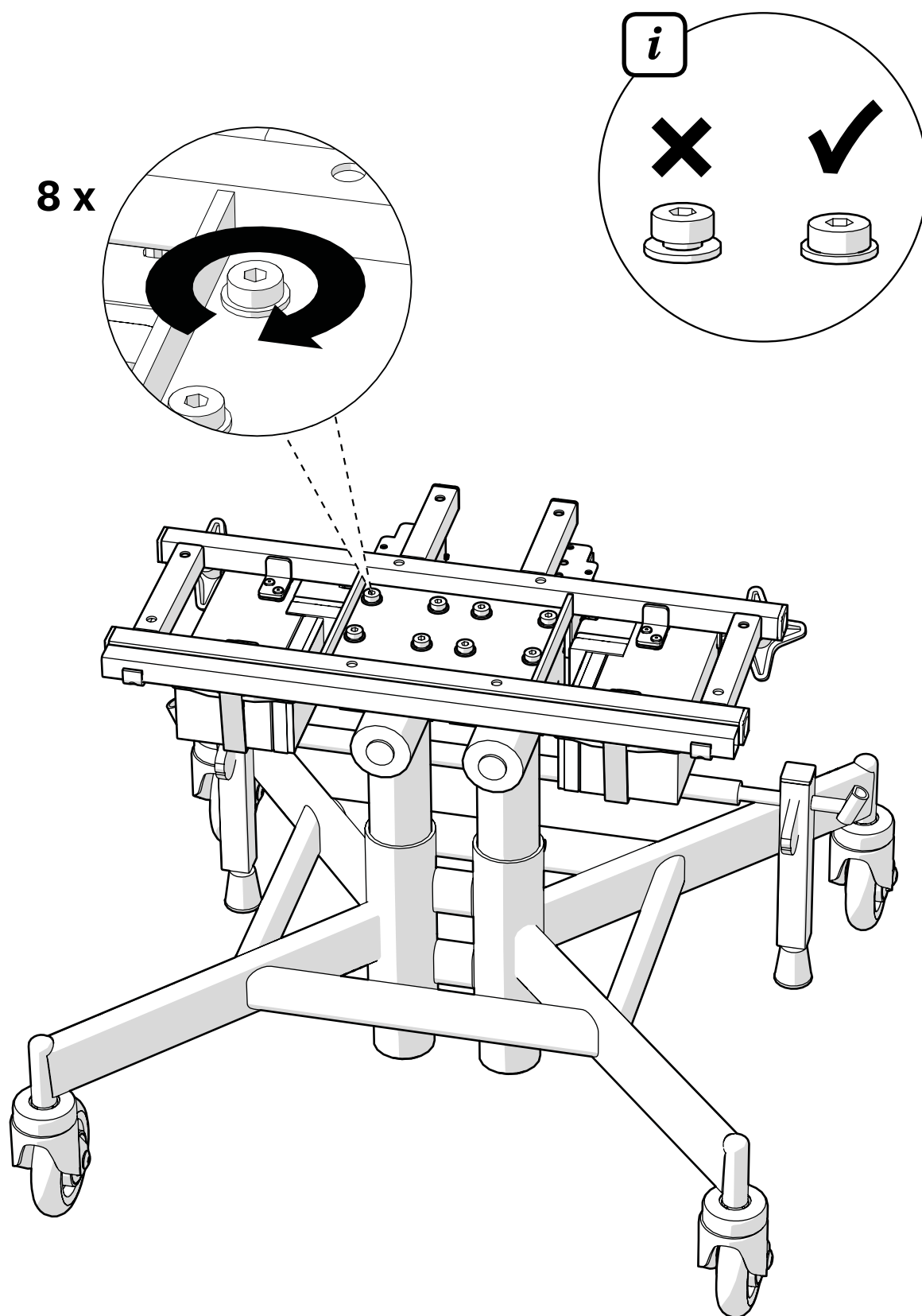

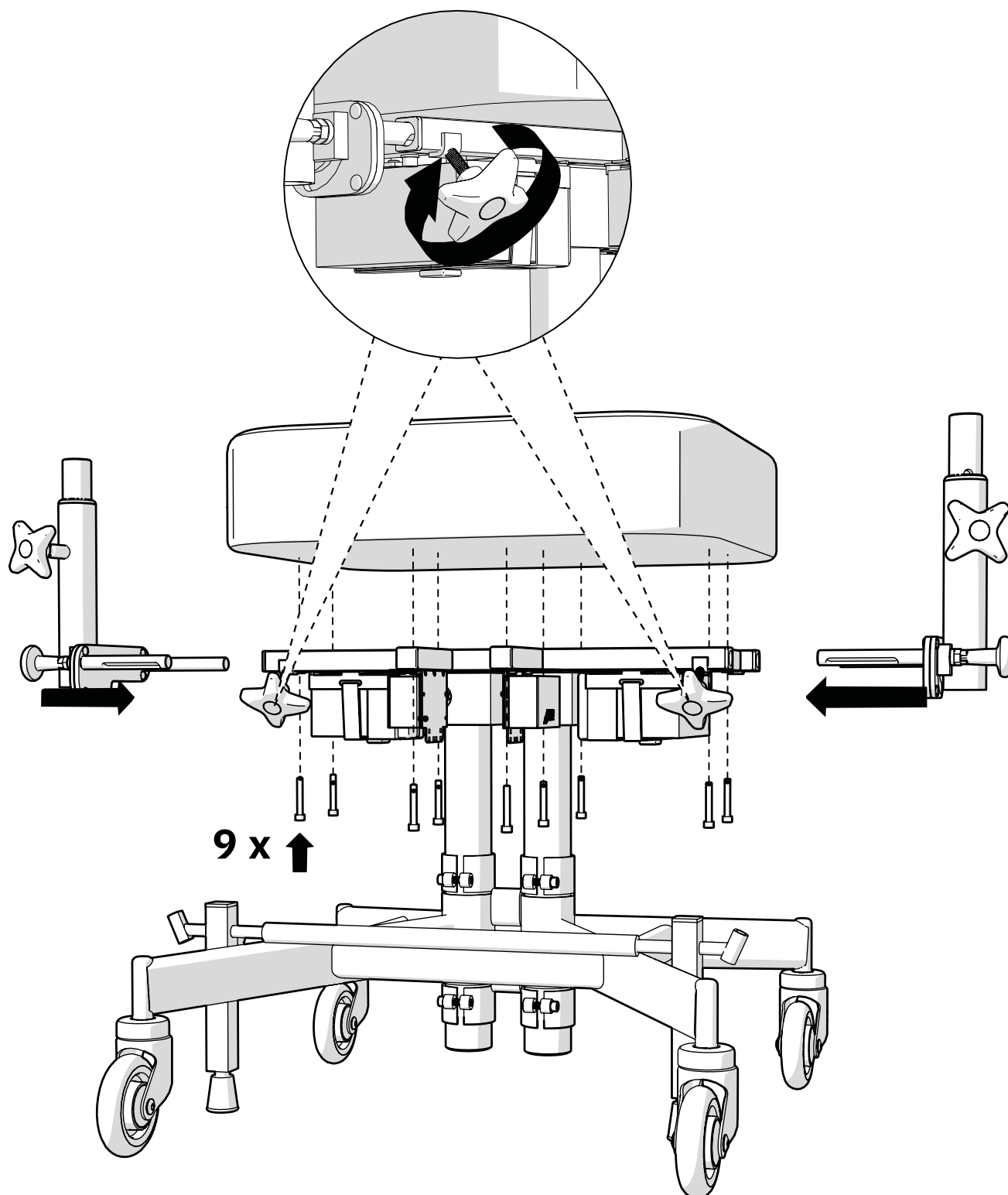

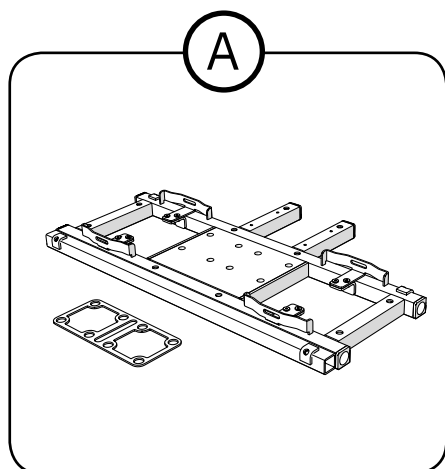**Skötselråd**

Produkten bör torkas av och hållas ren från damm och smuts. Vi godkänner rengöringsmedel från 7 upp till max 12 på pH-skalan (koncentrerat).

Destruktion

Ⓐ Sorteras som metall

Care instructions

The product should be dried and kept clean from dust and dirt. We accept cleaning agents from 7 up to a maximum of 12 on the pH scale (concentrated).

Destruktion

Ⓐ Sorted as metal