




EN

# BATTERY REPLACEMENT REAL® 9200 TWIN

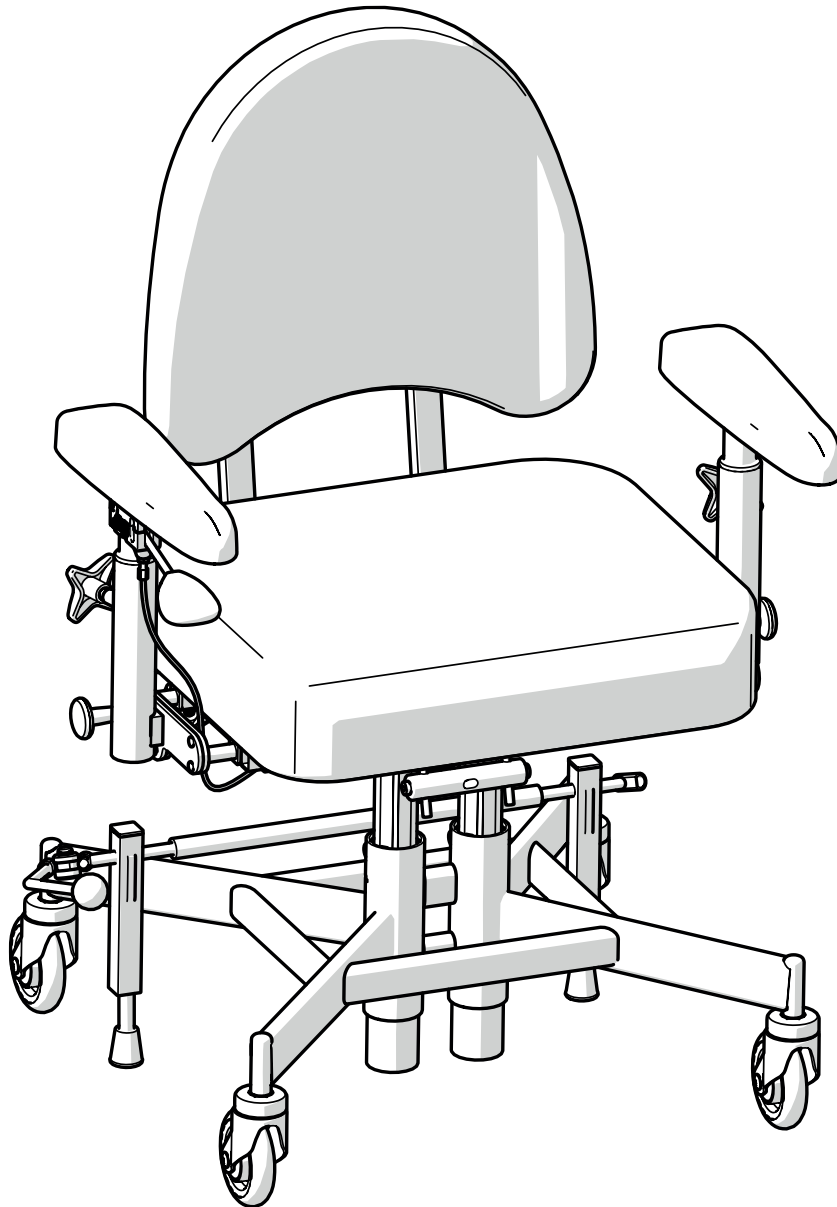

# 1. Important information

Bear in mind the following when handling batteries:

- Never short-circuit the battery.
- Do not subject the battery to strong shocks.
- Always discard of used batteries at recycling stations.

## First aid

If the battery is defective and handling it results in direct contact with the contents of the battery, electrolytes (sulphuric acid), the following applies:


Incident	Action
Skin contact	Rinse with water. Clean with soap and water. Remove and wash affected clothes.
Inhalation of acid mist	Breathe fresh air. Seek medical attention immediately!
Contact with eyes	Wash eyes with running water for at least 15 minutes. Seek medical attention immediately!
When swallowed	Immediately drink plenty of water, do not induce vomiting. Seek medical attention immediately!

# 2. Item list

805161 Battery box including batteries (TWIN 24V)

### 3. Assembly instructions

- A. Unplug the battery box by disconnecting the cord.
- B. Remove the battery box from the bracket by loosening the four screws.
- C. Install the new battery box by following the instructions above in reverse order.
- D. Repeat the same procedure for the second battery box. Battery boxes must always be replaced at the same time.

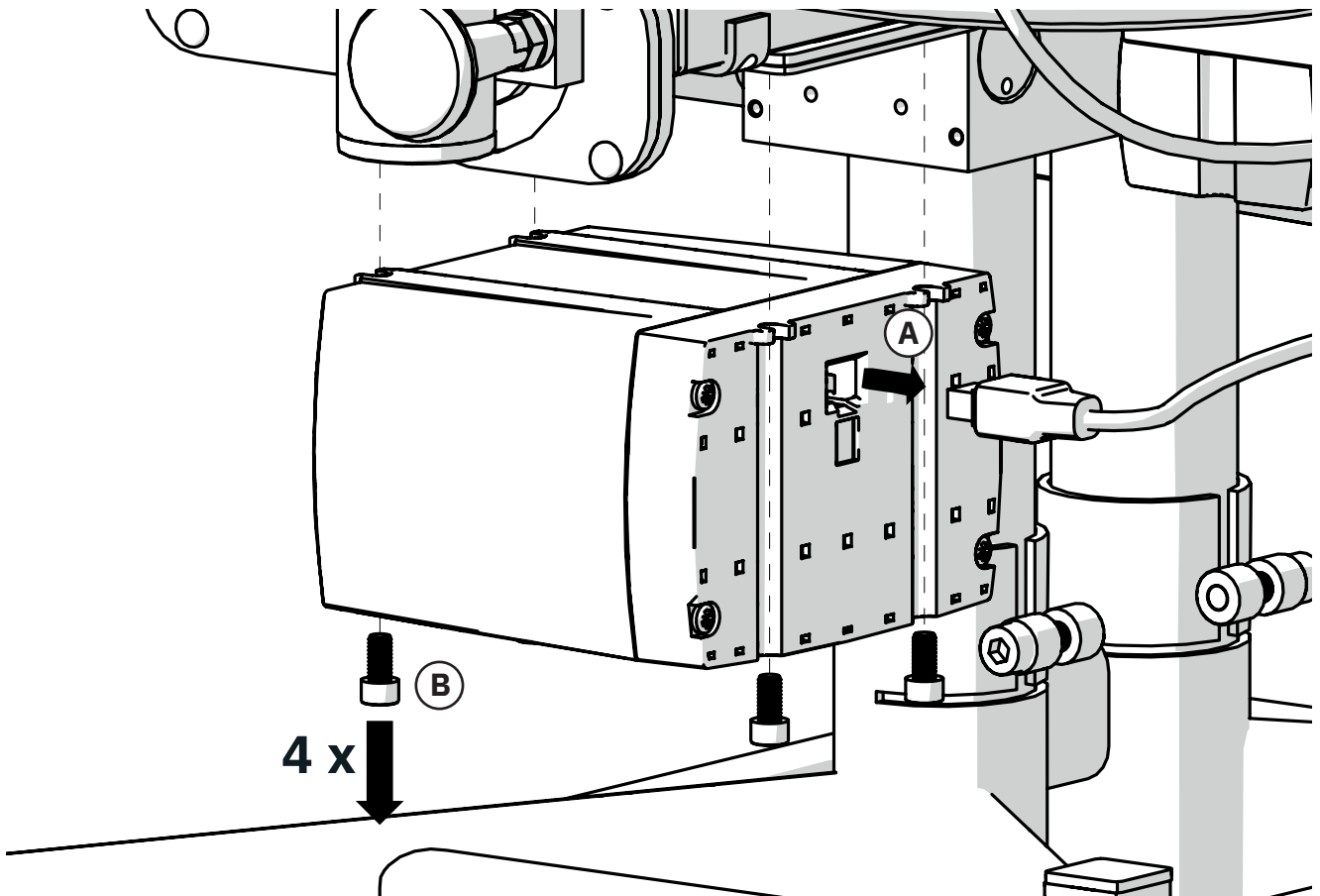

### 4. Disposal

Battery boxes, including the batteries, should be disposed of as batteries.