




SCRIBA

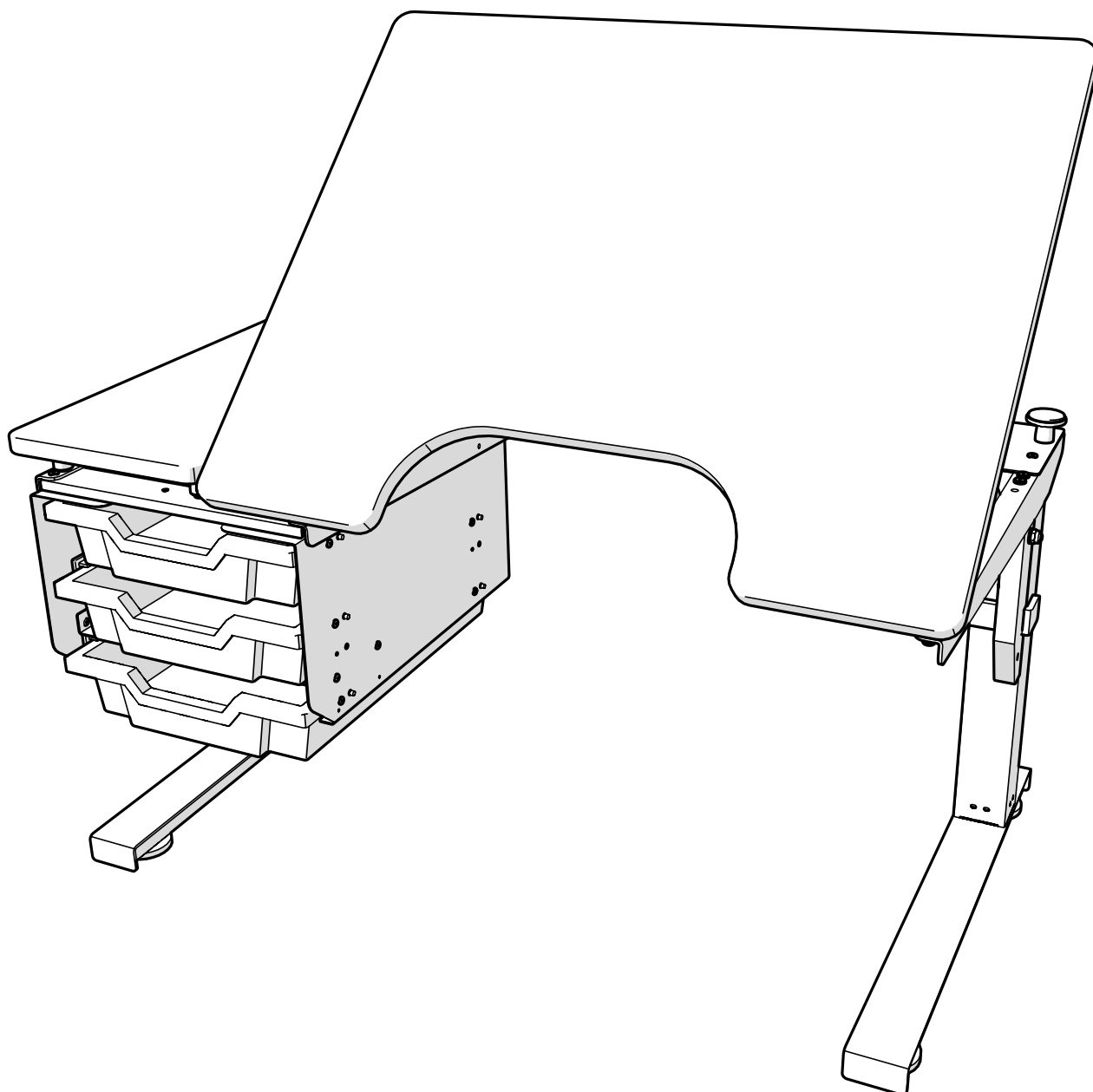



If you have any questions about your product or if something unexpected has happened, please contact your dealer first, otherwise you are welcome to contact Mercado Medic.

PDF versions of our operating and maintenance instructions with magnification options can be found on our website www.mercado.se.

We reserve the right to make changes to this manual and its contents.


Table of content

1. Parts list	4	6.1. Mounting the rastomat bracket.	24
1.1. Table tops.	4	7. Tabletop Attachments	25
1.2. Base frame	4	7.1. Rotation joint	25
1.3. Attachments	5	7.2. Tabletop support, fixed	25
1.4. Drawer box parts.	6	7.3. Tabletop support, furniture pad	26
1.6. Tilt parts	7	7.4. Spacer (for tabletop support)	26
1.5. Fasteners	7	8. Attachments Points.	27
2. Frame (Art. 805118).....	8	8.1. Possible attachment points on the frame .	27
2.1. Subassembly – Legs.....	8	8.2. Tabletop 750x600mm (Flat).....	28
2.2. Subassembly – Crossbar	9	8.3. Tabletop attachments	28
2.3. Mounting the legs	10	8.4. Tabletop 750x600mm (Tilt)	29
2.4. Subassembly – Drive	11	8.5. Tabletop attachments	29
2.5. Mounting the drive	12	8.6. Tabletop 750x600mm (Hinge)	30
2.6. Mounting the drive shaft	13	8.7. Tabletop attachments	30
2.7. Mounting the side bars.....	14	8.8. Tabletop 1000x600mm (Flat).....	31
2.8. Mounting the crank rest.....	15	8.9. Tabletop attachments	31
2.9. Mounting the crank.....	16	8.10. Tabletop 1000x600mm (L-shape, hinge) .	32
3. Arm & bar	17	8.11. Tabletop attachments	32
3.1. Placement of support console	17	8.12. Tabletop 1000x600mm (Divided, tilt)	33
3.2. Mounting the support consoles	18	8.13. Tabletop attachments	33
3.3. Mounting the support arm and securing the second support console	19	8.14. Tabletop 1000x600mm (Centered, hinge)	34
3.4. Mounting drawer box crossbars	20	8.15. Tabletop attachments	34
4. Drawer Box.....	21	8.16. Tabletop 1000x600mm (Centered, flat) ..	35
4.1. Mounting crossbar for drawer.....	21	8.17. Tabletop attachments	35
4.2. Mounting drawer box	22	8.18. Tabletop 1000x600mm (Centered, tilt)...	36
5. Frame Cover.....	23	8.19. Tabletop attachments	36
6. Rastomat Bracket	24	9. Tilt mechanism	37
		9.1. Mounting the tilt mechanism	37

1. Parts list


1.1. Table tops

- 805140 Table top flat 1000x600 cpl Right
- 805139 Table top flat 1000x600 cpl Left
- 805145 Table top side 600x245 cpl
- 805141 Table top flat 750x600 mm cpl Magnetic
- 805190 Table top flat 750x600 mm cpl None Magnetic
- 805142 Table top hinge fold 1000x600 cpl Left
- 805143 Table top hinge fold 1000x600 cpl Right
- 805144 Table top hinge fold 750x600
- 805280 Table top centered 1000X600 cpl None magnetic
- 805281 Table top centered 1000X600 cpl Magnetic
- 805282 Table top hinge fold 1000x600 cpl No belly hole
- 805283 Table top centered 1000X600 cpl None magnetic 30x12 cm Belly hole
- 805284 Table top centered 1000X600 cpl Magnetic 30x12 cm Belly hole

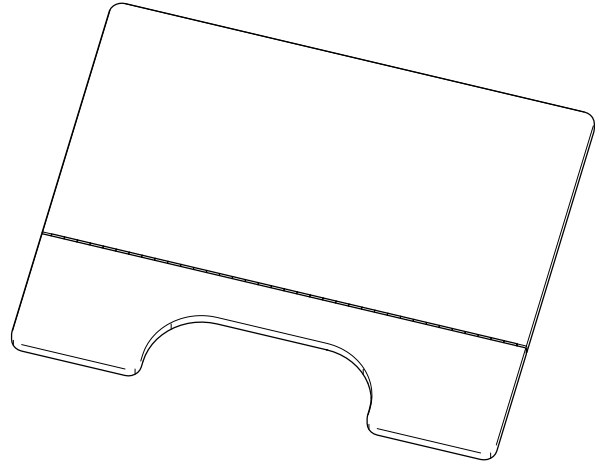


1.2. Base frame

- 805118 Table frame ATYP SIMPLE 1 C 2s V5 H300

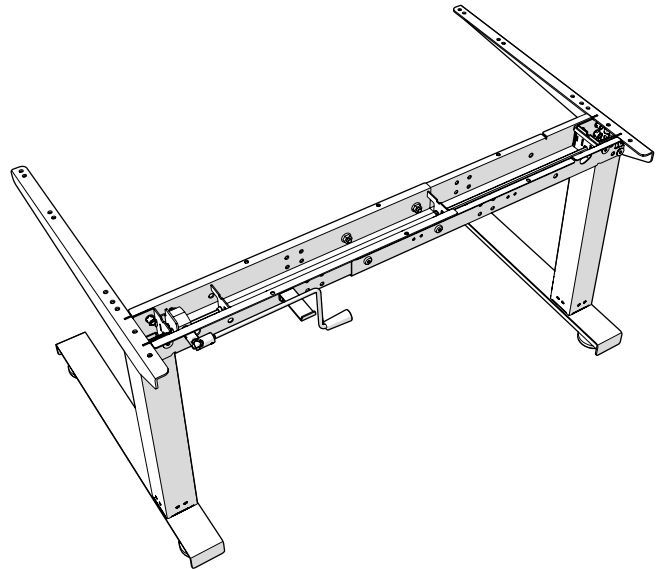


1.3. Attachments


- 805120 Support Console
- 805134 Support arm left side
- 805135 Support arm right side
- 805125 Crossbar welded
- 805124 Crossbar small
- 805128 Frame cover 710 mm
- 805129 Frame cover 960 mm
- 805136 Table support assembly
- 805137 Table support distance

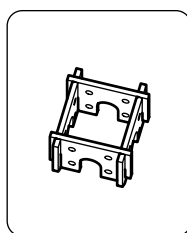


805120

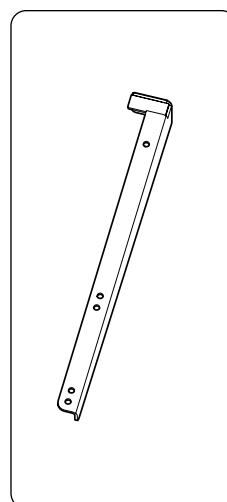


805134

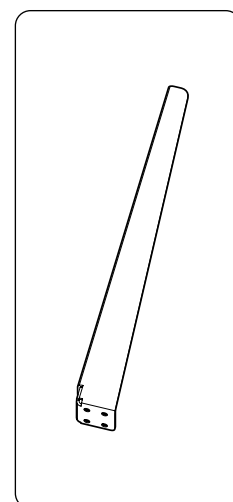


805135

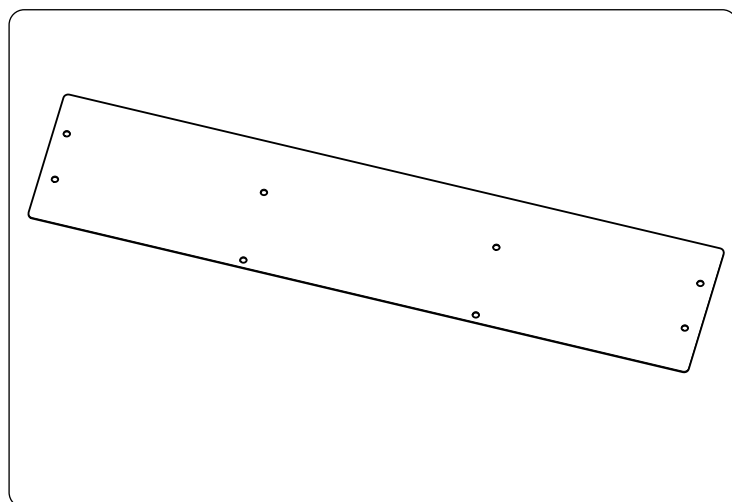


805128

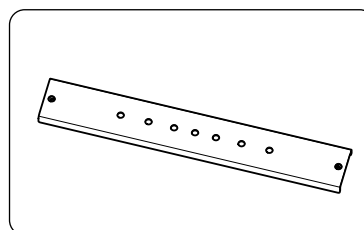


805125

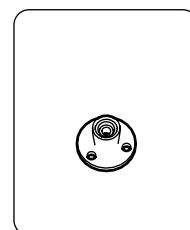


805136

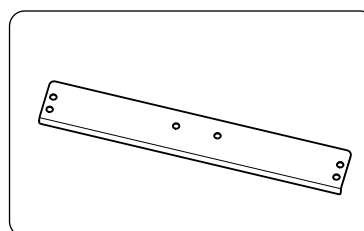

805124

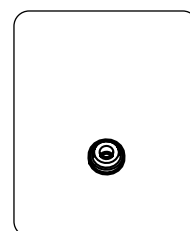



805137

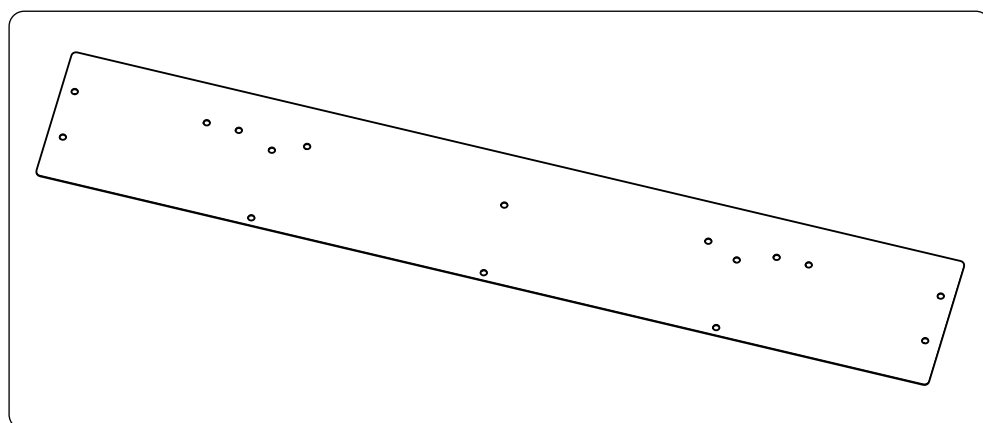


805129

1.4. Drawer box parts


804642	Drawer box welded
805149	Plastic Drawer, Black
805150	Plastic Drawer, Blue
805151	Plastic Drawer, Cyan
805152	Plastic Drawer, Green
805153	Plastic Drawer, Grey
805154	Plastic Drawer, Lime
805155	Plastic Drawer, Orange
805156	Plastic Drawer, Pink
805157	Plastic Drawer, Red
805158	Plastic Drawer, Yellow
805159	Drawer Slides
805127	Distance Asmb for drawer box
802038	K6S M6x20 black chromated
800127	NB 6x22x1,5 FZB

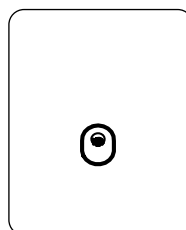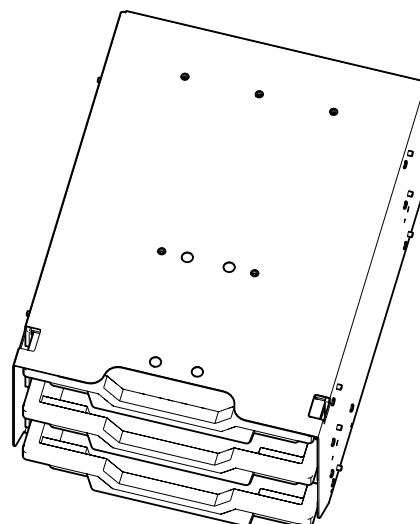

805127

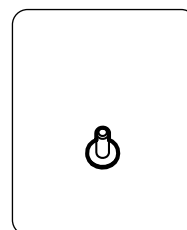


802038

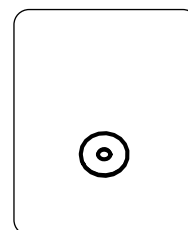

800127


1.6. Tilt parts

805148	Vertical tilt mech hold mount.
805119	Bracket tilt mechanism holder
805146	Tilt mechanism holder screw
805133	Rotation part
800131	CPK 8x36 Tol.h6
803840	Rastomat 214-354 mm
805126	Damper assembly
805138	Table top bracket
804624	MC6S M8x35 FZB
800091	MLC6S M8x16 8.8 FZB
800445	LM6M M8 FZB
800385	MF6S M8x16 FZB 10,9
801289	Alu. distance 15x8.5x9.5 f gas spring seat angle

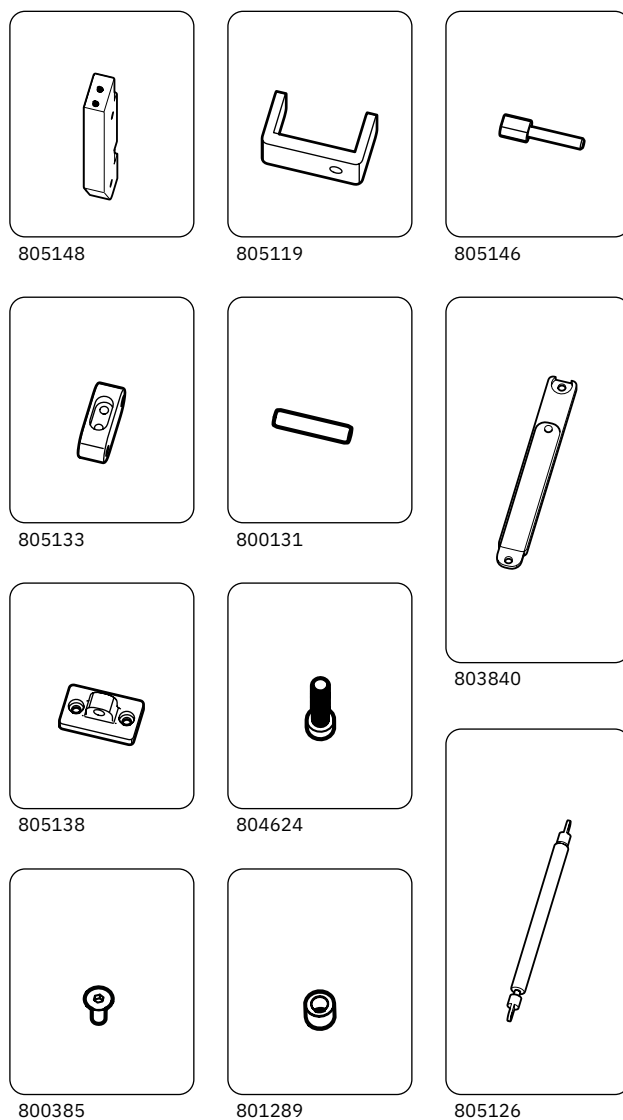

1.5. Fasteners


805131	MC6LS M6x12
800417	MC6S M6x14 FZB 8.8
801566	MF6S M6x16 OBEH 10.9
803771	MC6S M6x30 8.8 FZB
800462	BRB 6,4x12x1,5
800126	BRB 8,4x16x1,6
800117	LM6M M6
805130	Furniture pad Ø38

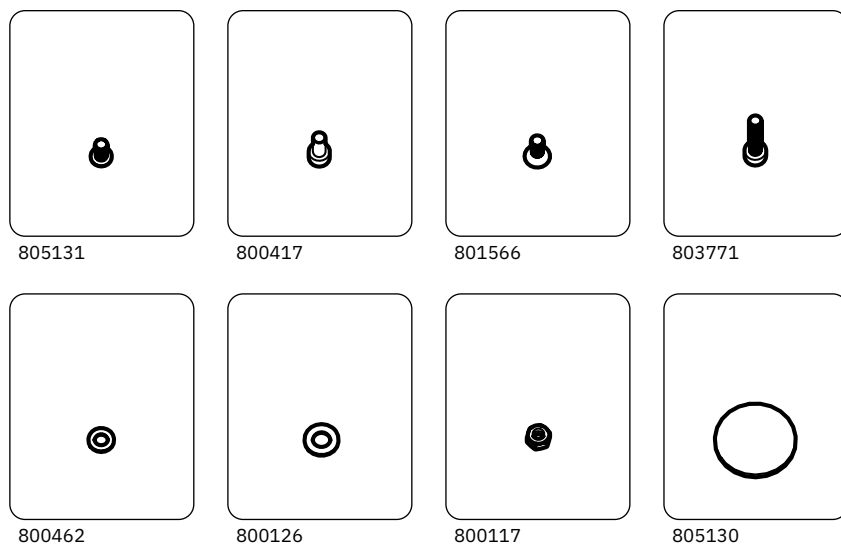


2. Frame (Art. 805118)

2.1. Subassembly – Legs


Looking at the table legs from the front, the blue part of the motor housing should be on the left side, and the U-shaped bracket should be facing the center of the table.

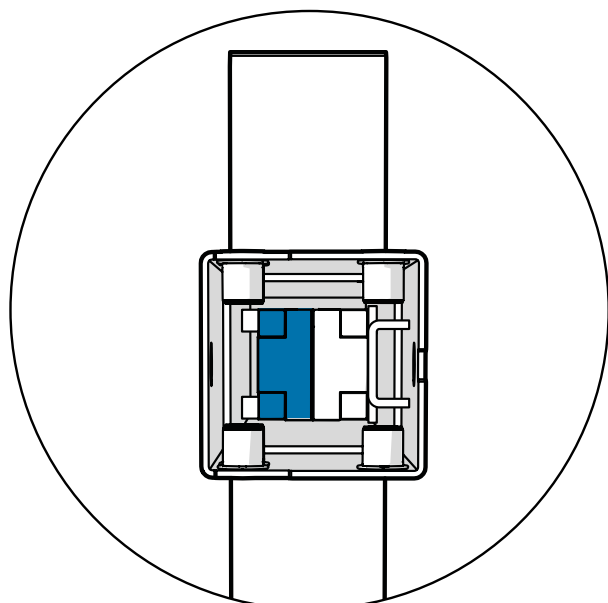


Left

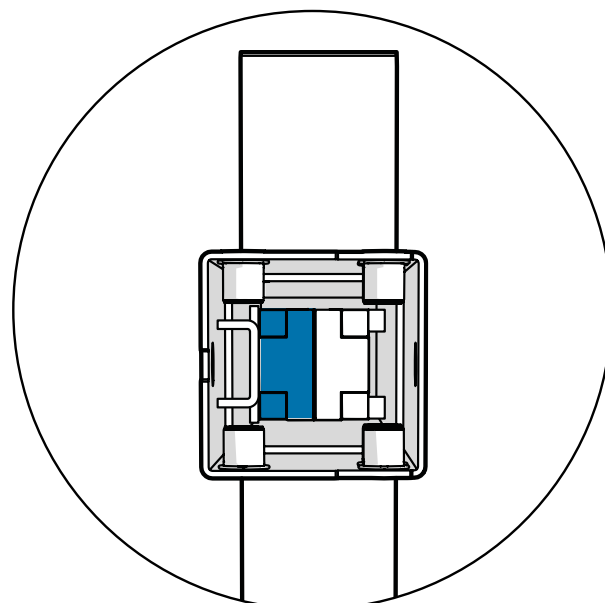


Right

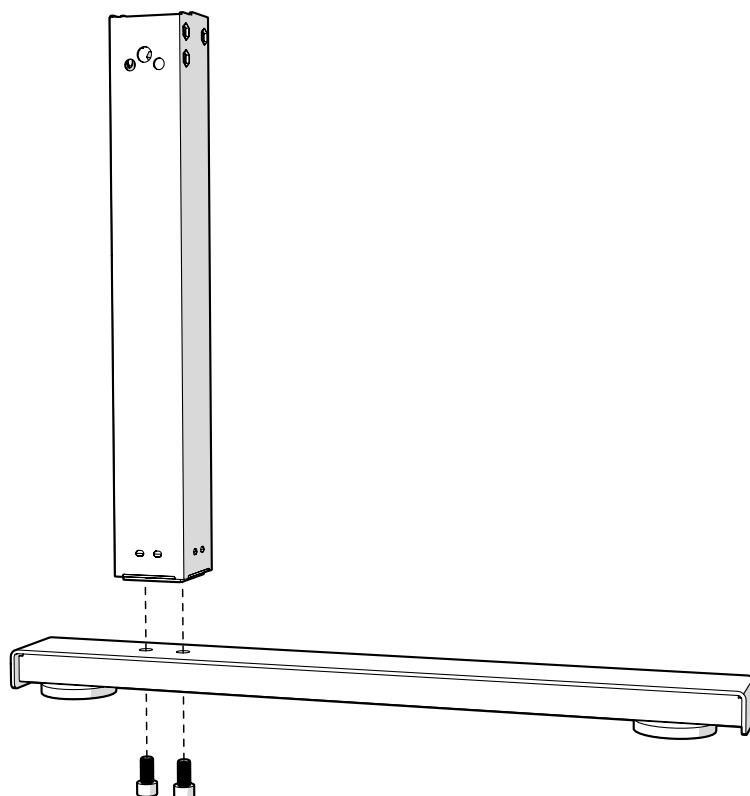



2.2. Subassembly – Crossbar

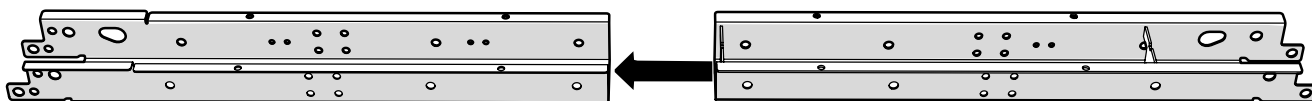


For small tabletop, 750x600mm

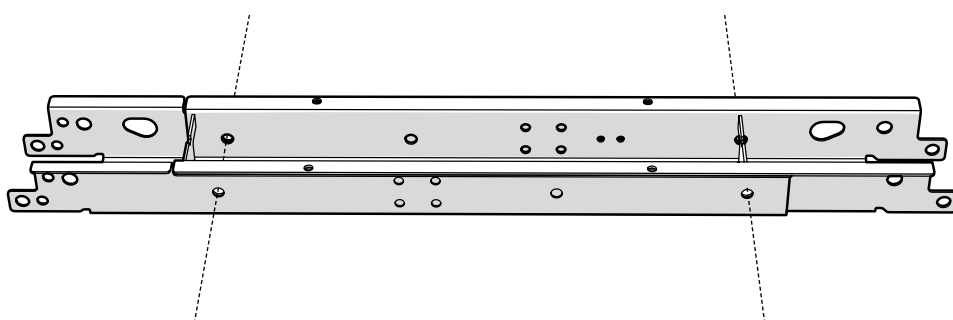

For large tabletop, 1000x600mm


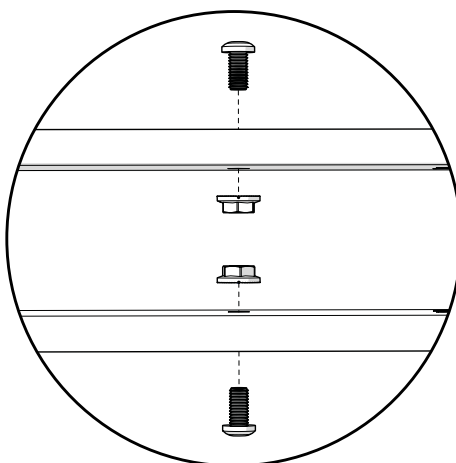
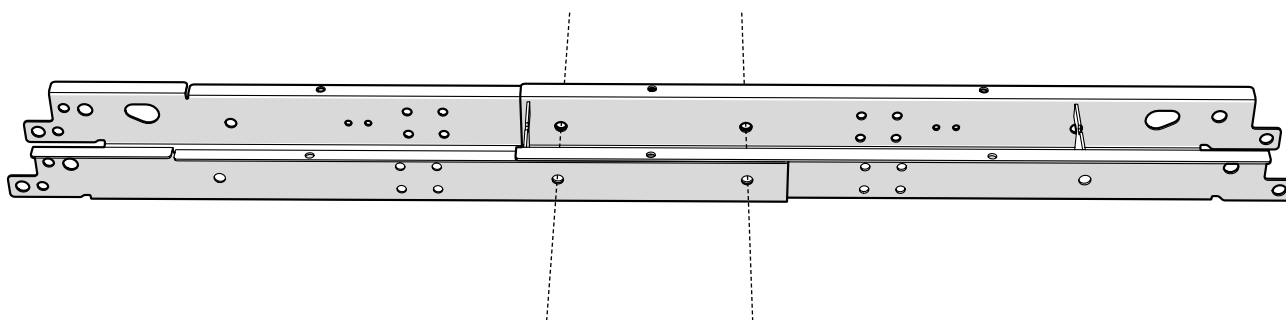


2.3. Mounting the legs

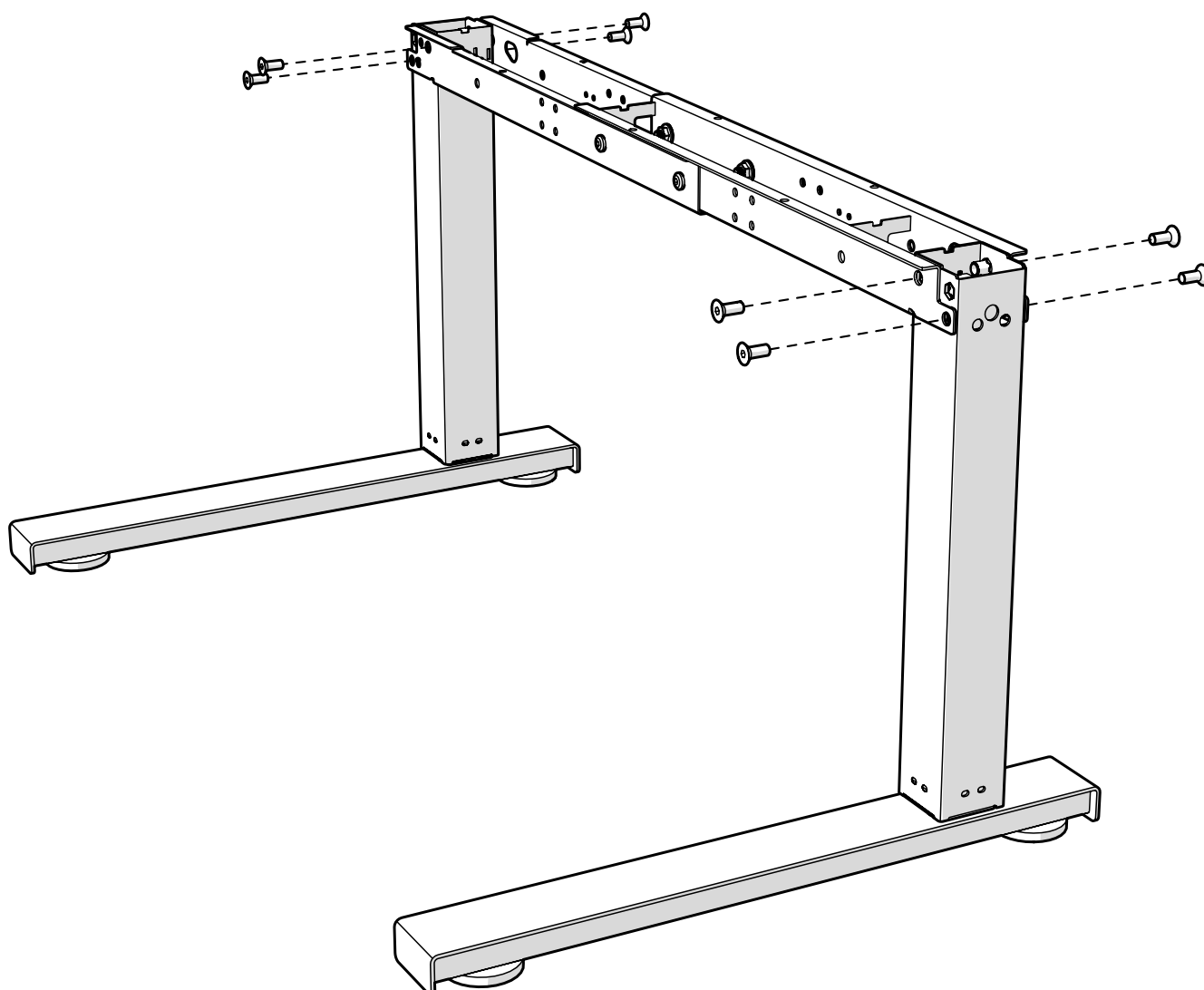




2.4. Subassembly – Drive

Depending on the desired location of the height adjustment crank, the drive needs to be assembled appropriately for mounting on the left(2A) or right(2B) side of the table.

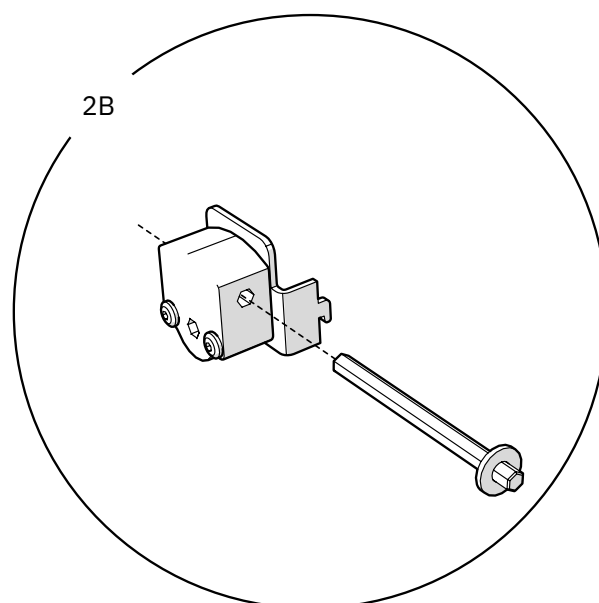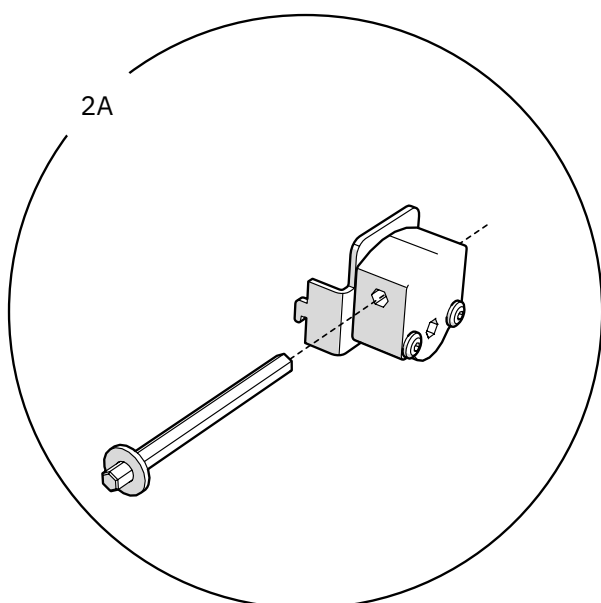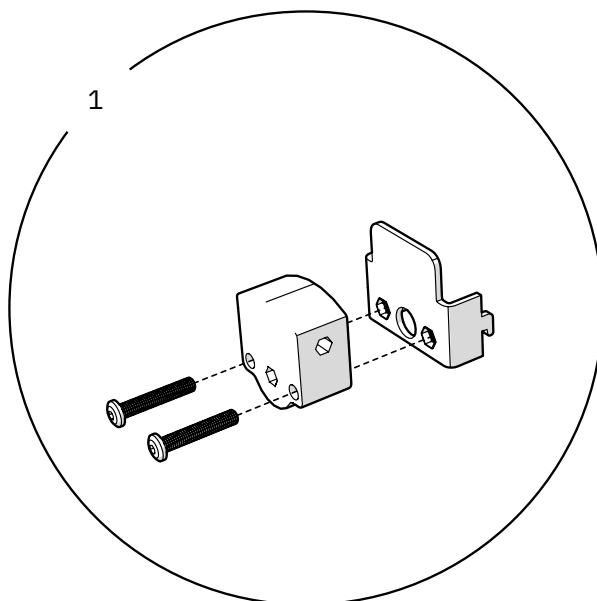


2.5. Mounting the drive

- ! The drive can be placed on the right or left end of the trough depending on the desired final location of the height adjustment crank. The pictures below illustrates the drive being mounted on the right side.

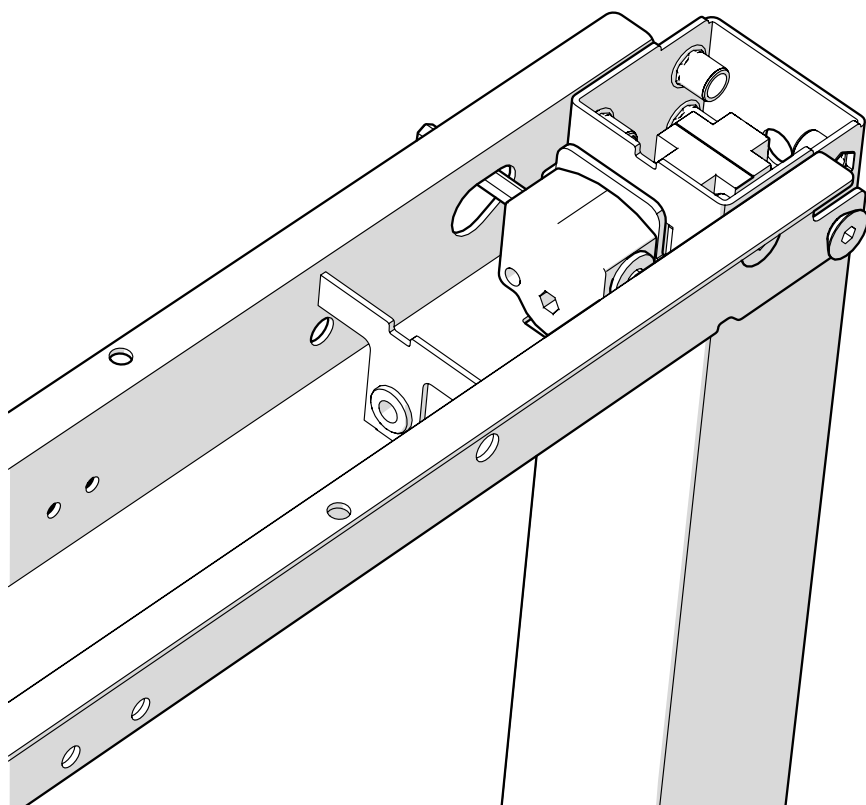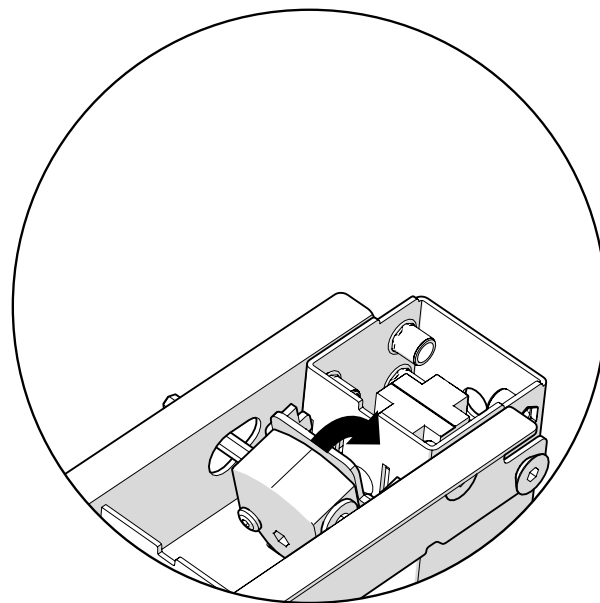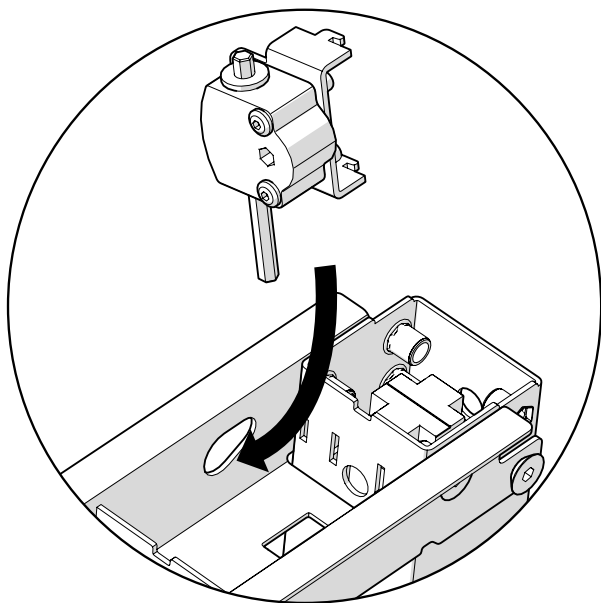


2.6. Mounting the drive shaft



1

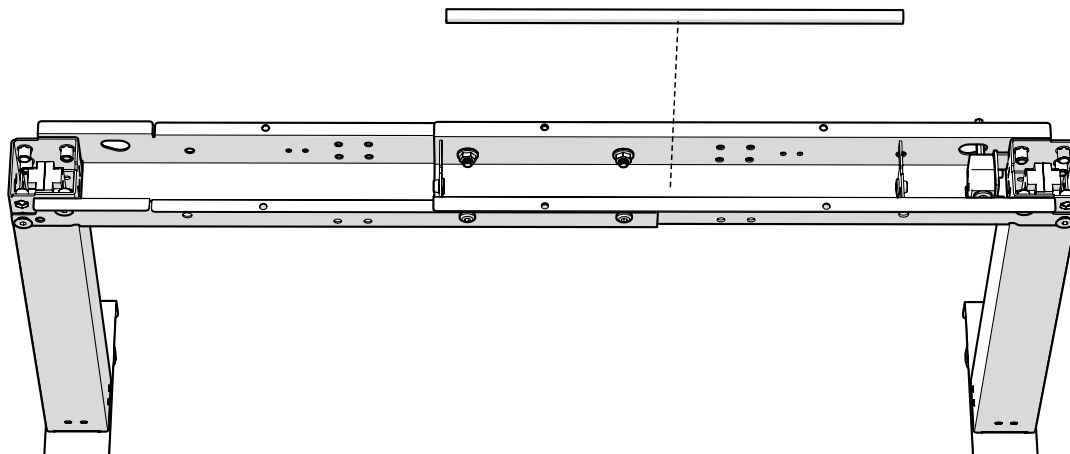

2

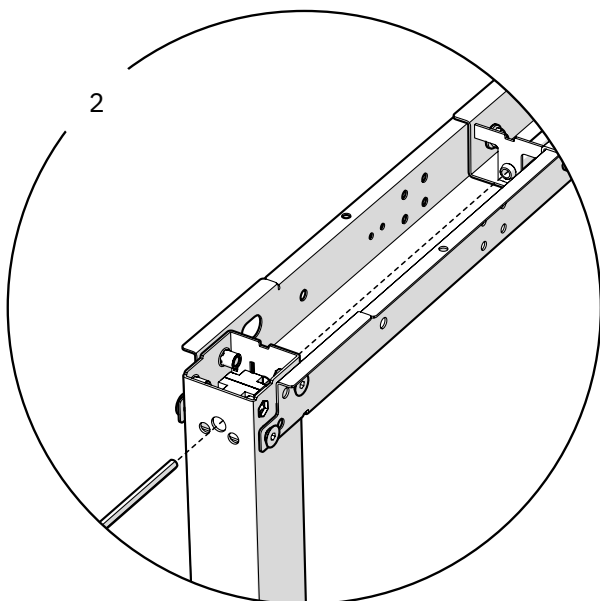


3

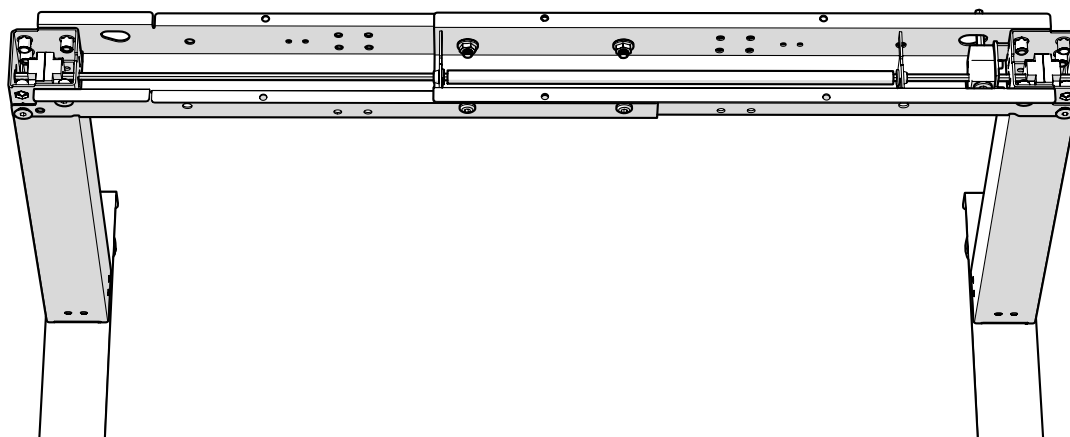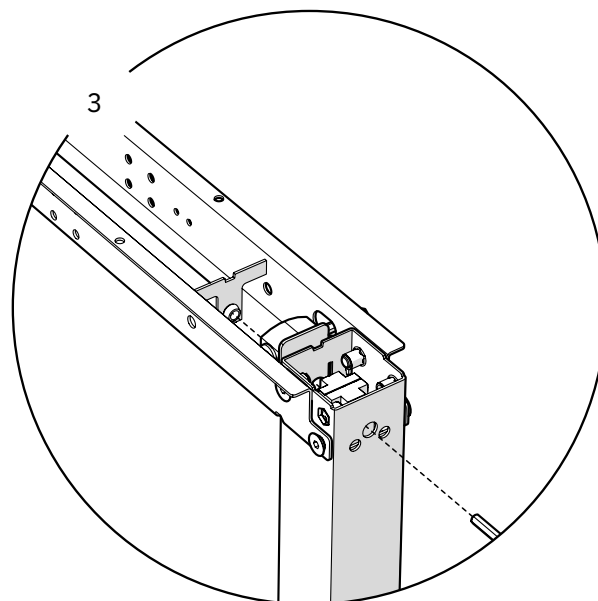


2.7. Mounting the side bars

Fasten both bars securely with screws from the front and back.



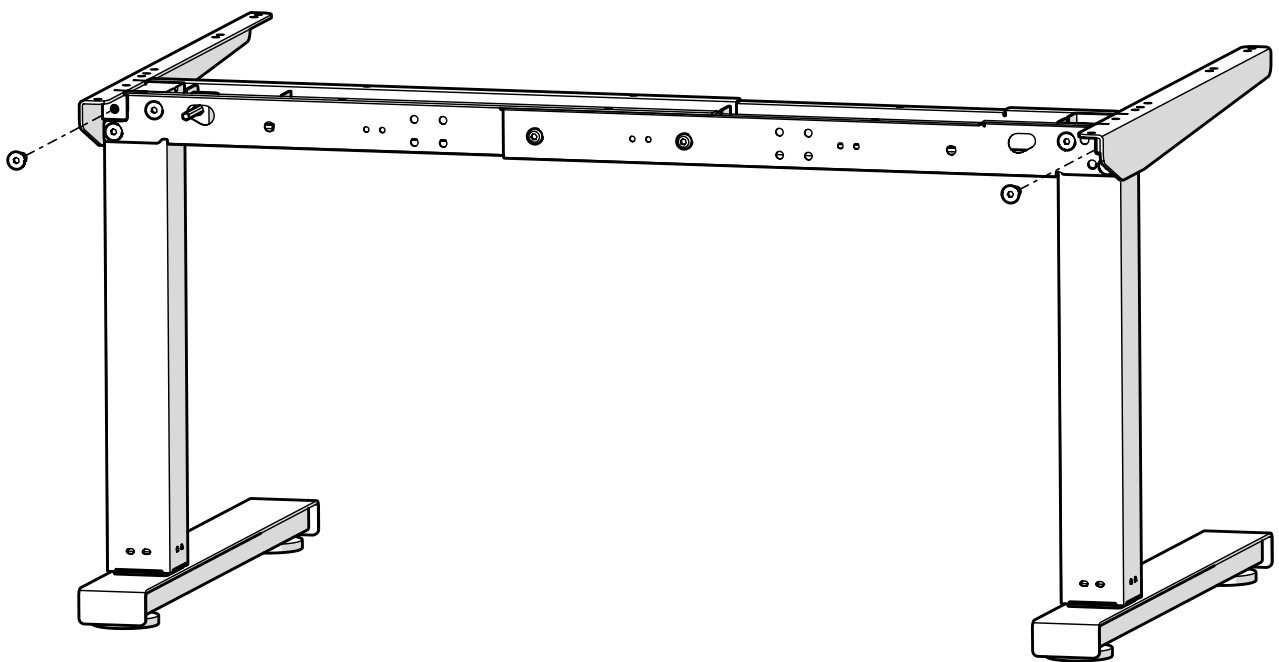


2.8. Mounting the crank rest

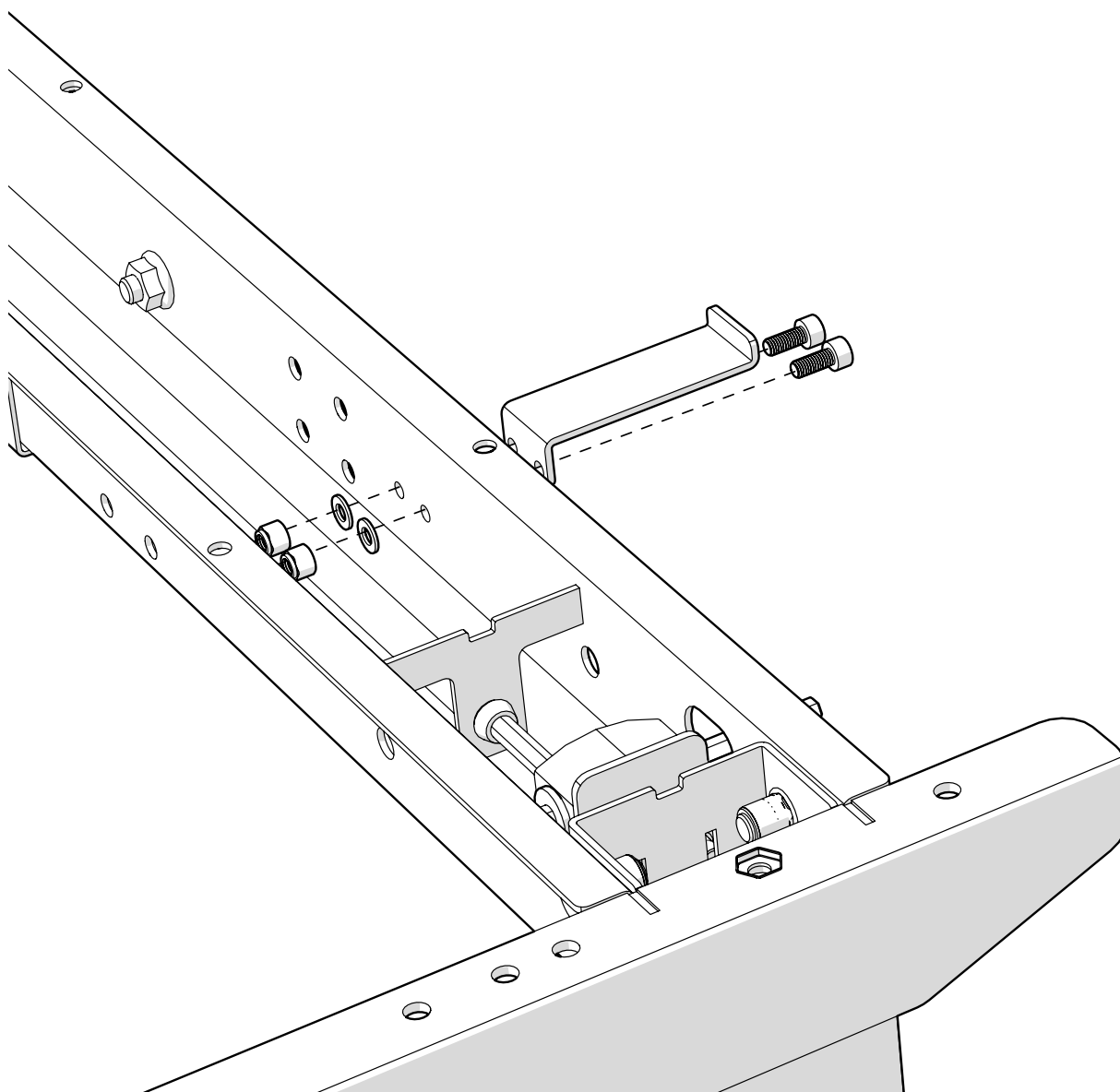

2.9. Mounting the crank


NOTE! To have the crank resting in a downward position, the handle must be oriented correctly when secured to the joint(2). If correct, securing the handle and joint to the drive should look like what is illustrated below.

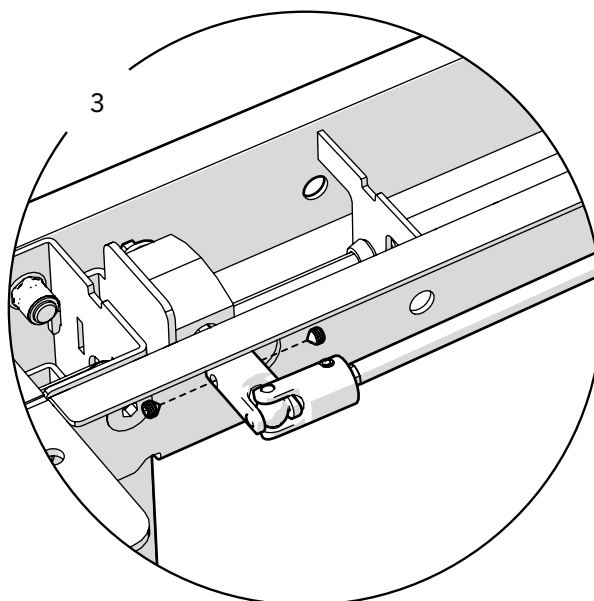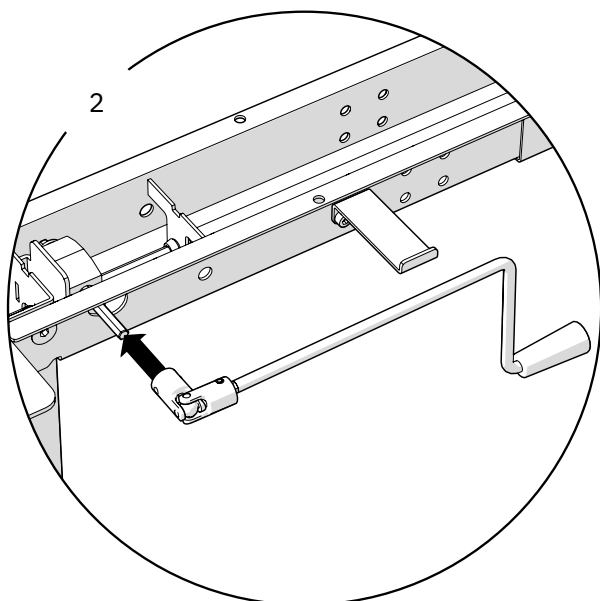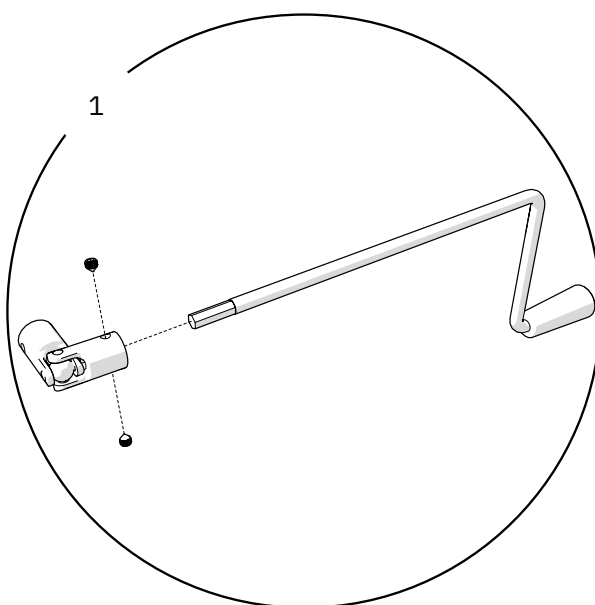


3. Arm & bar

3.1. Placement of support console


! The support consoles have to be oriented accordingly to its position, please see below.

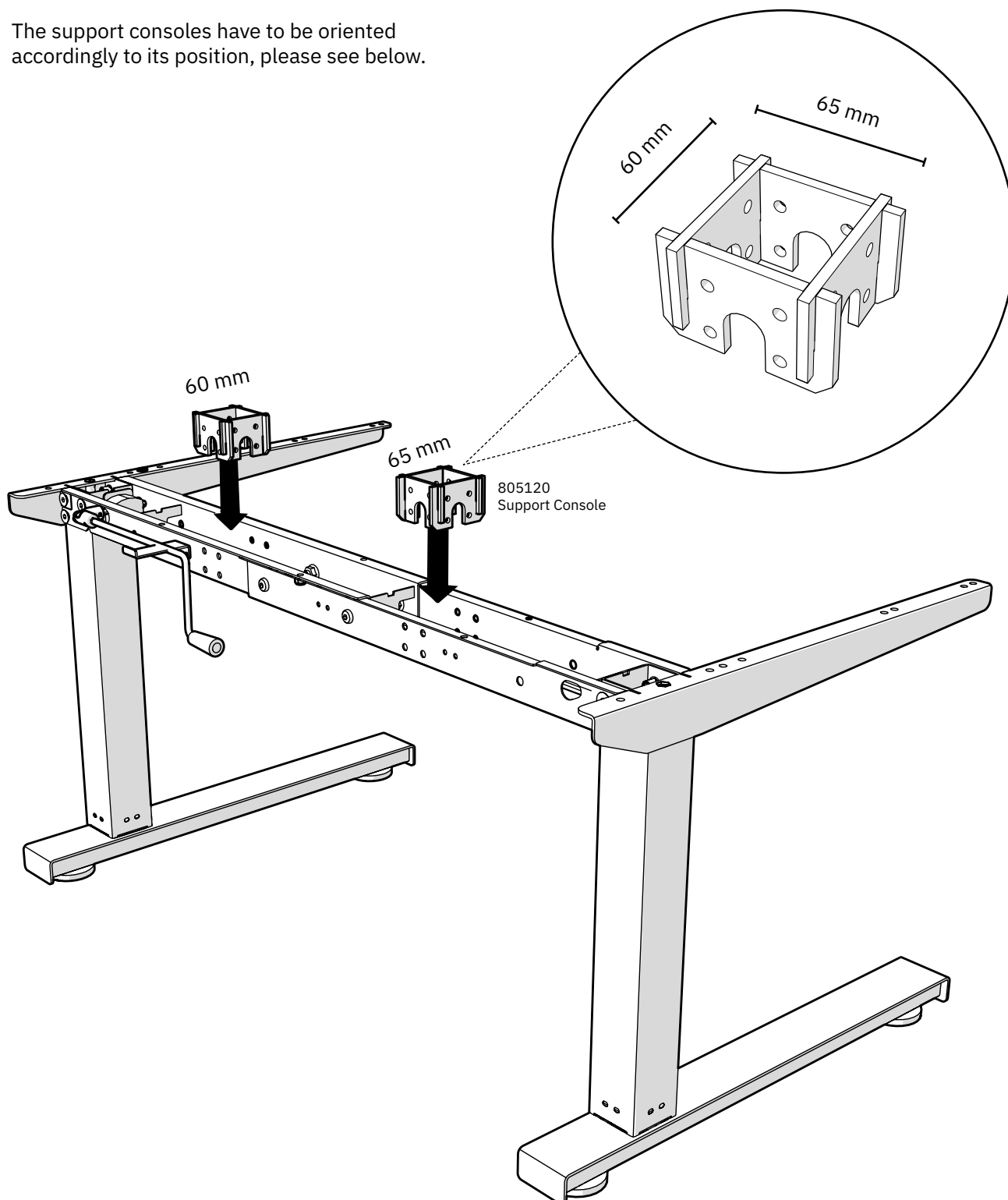

3.2. Mounting the support consoles

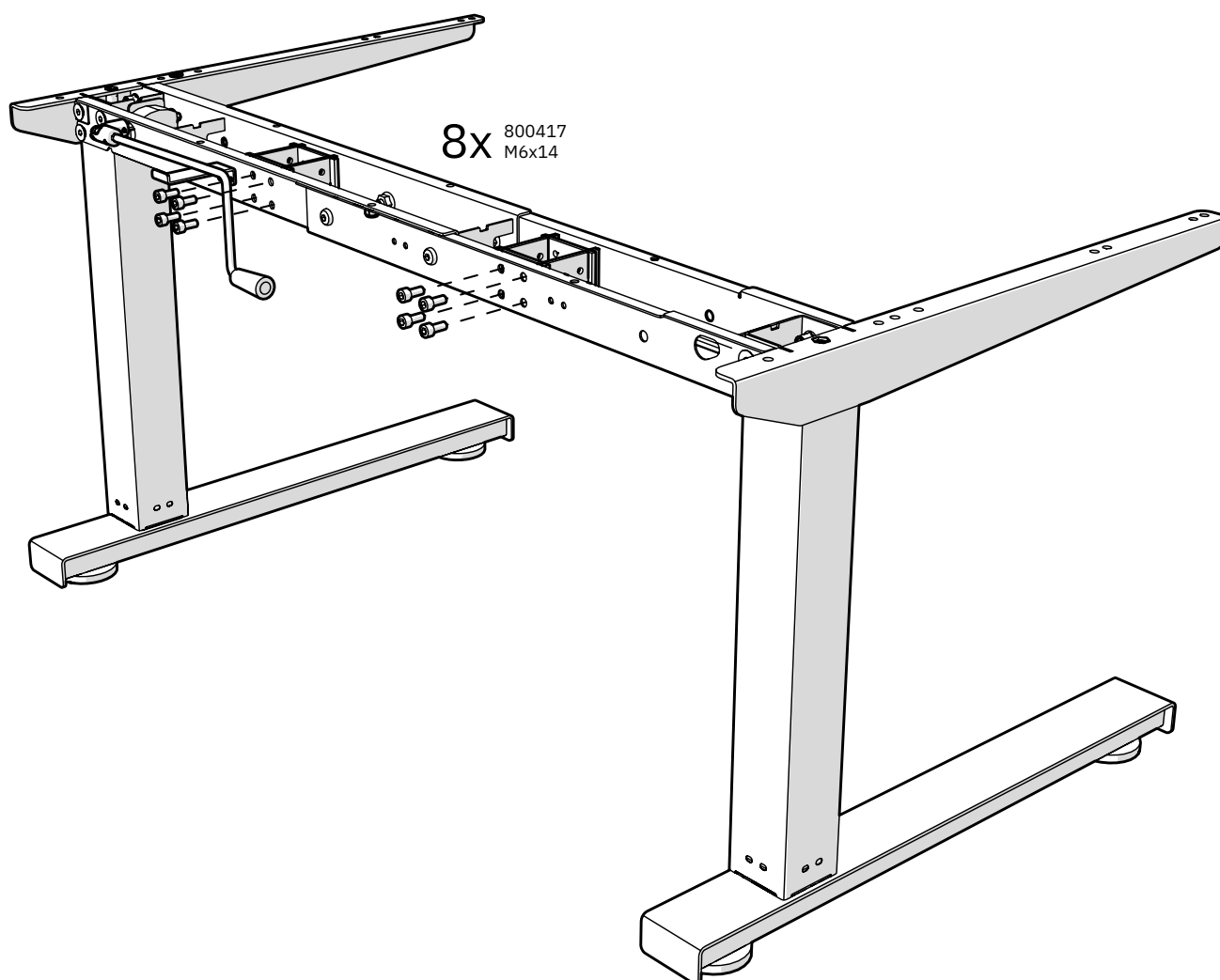


3.3. Mounting the support arm and securing the second support console

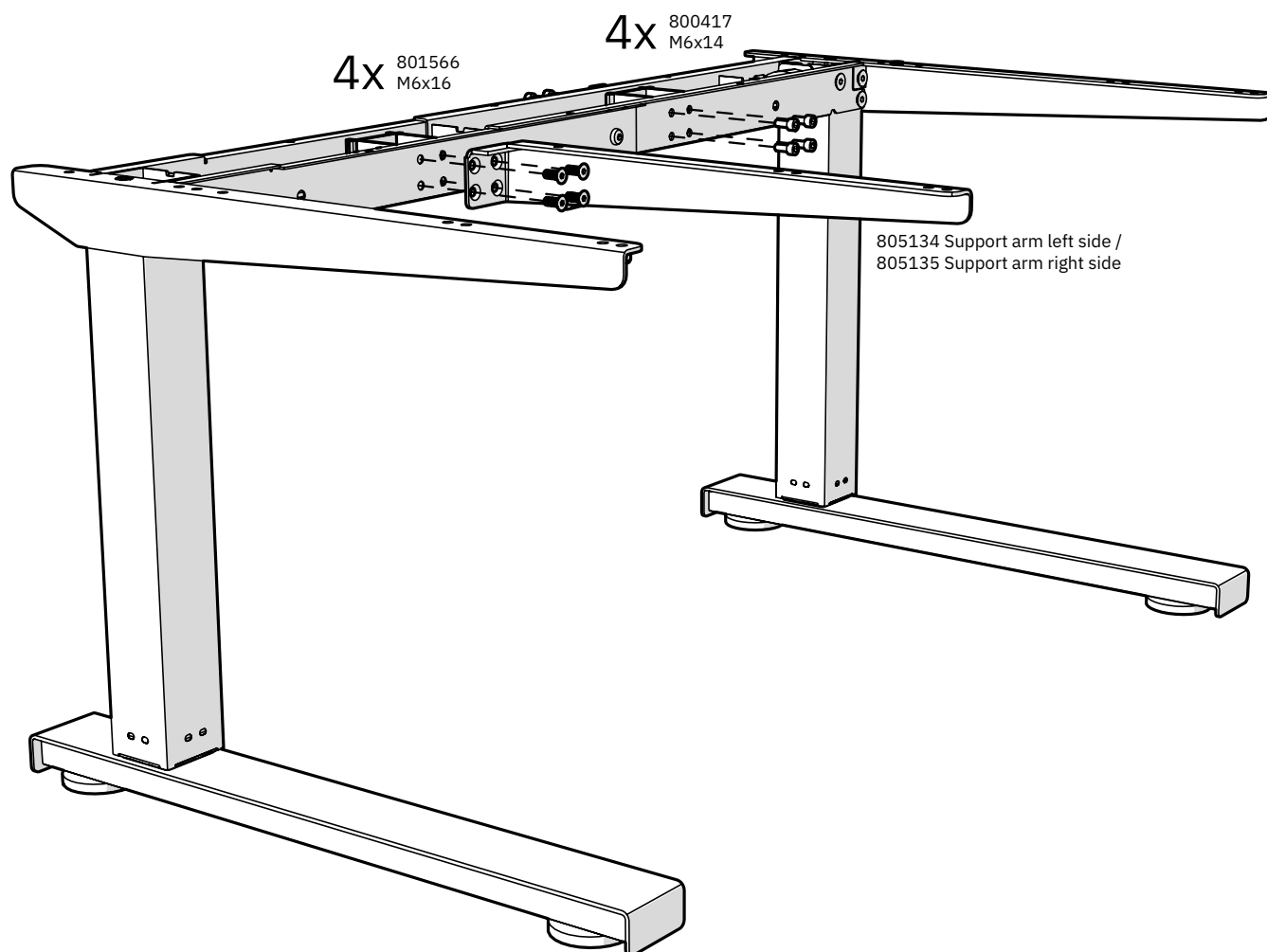


3.4. Mounting drawer box crossbars

Depending on your table configuration, the mounting of one or both of the crossbars may be required. Please check the Parts list of your configuration to see what is applicable for your table.

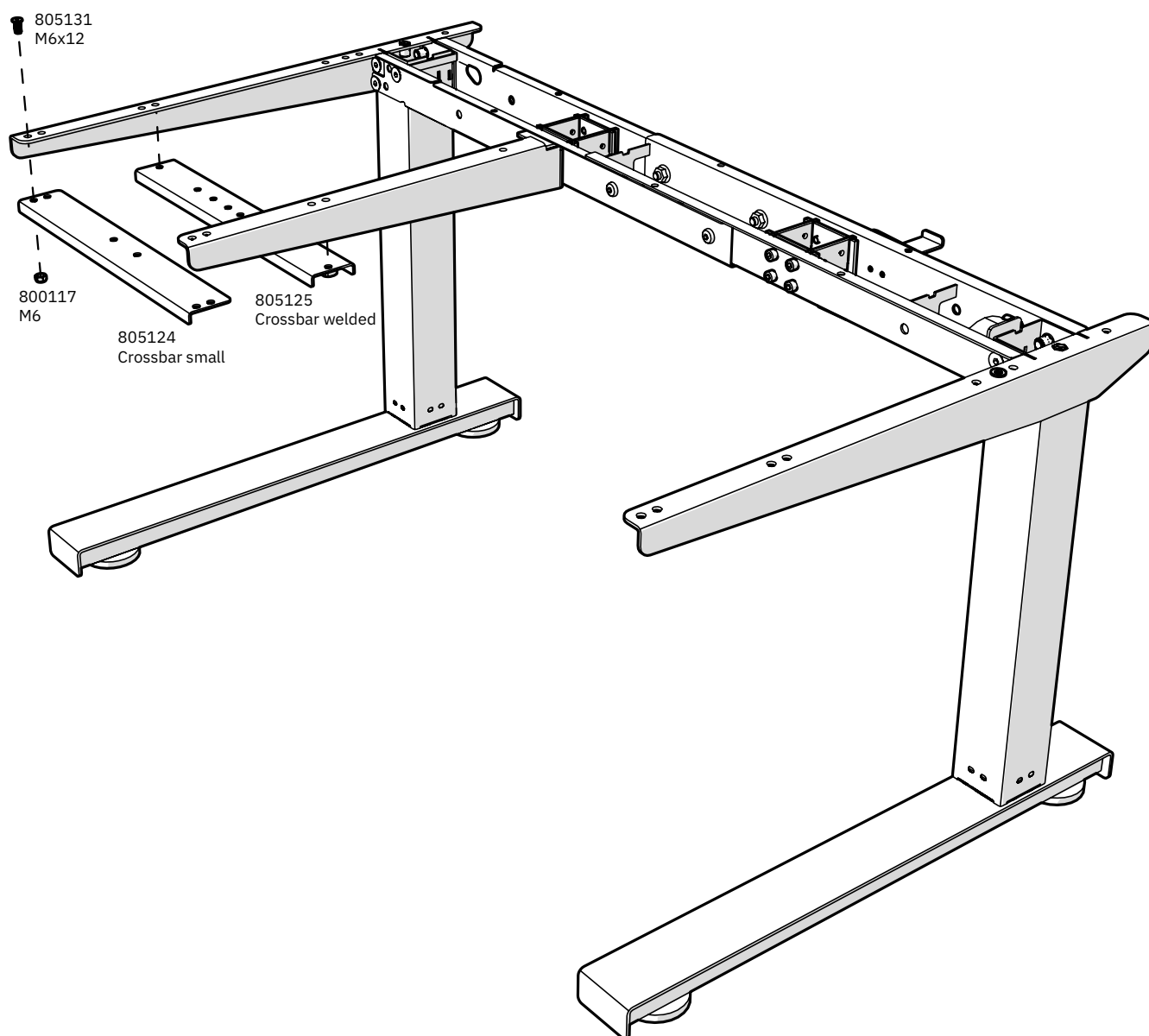


4. Drawer Box

4.1. Mounting crossbar for drawer


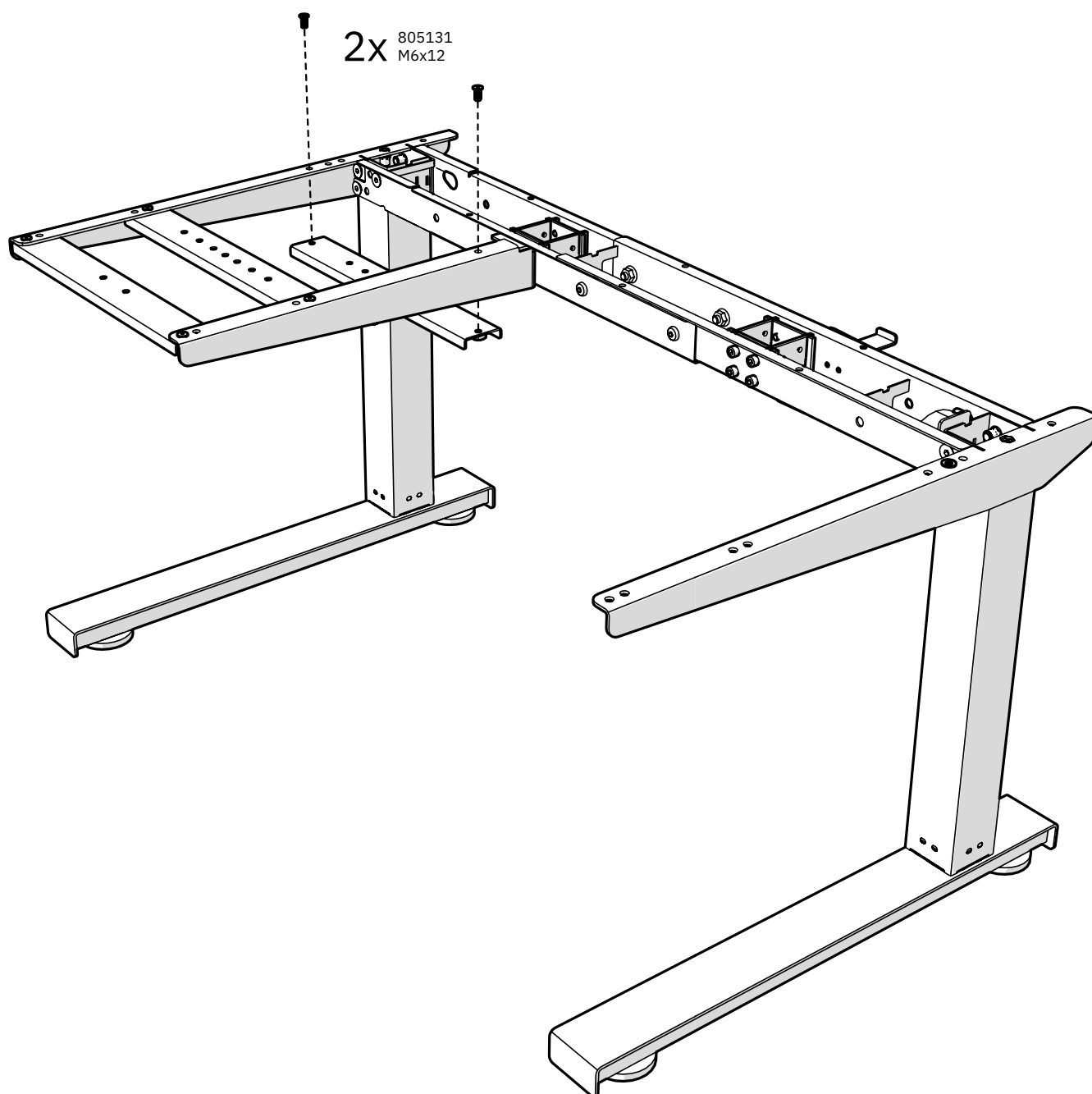


4.2. Mounting drawer box

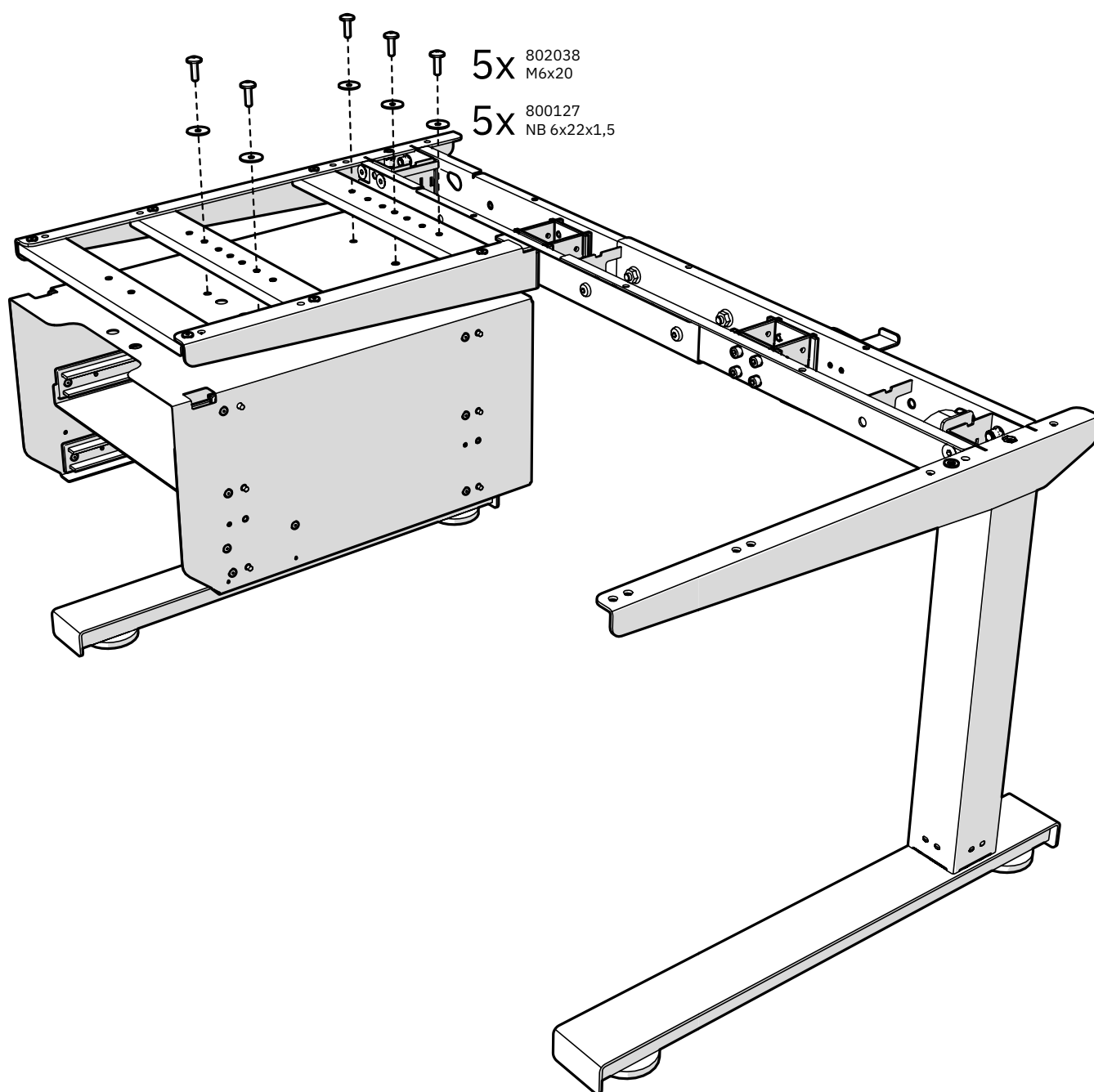


5. Frame Cover

! The cover should be oriented so that the CE label is positioned on the underside of the frame cover, towards the back of the table. This is to ensure visibility when the table is fully assembled.

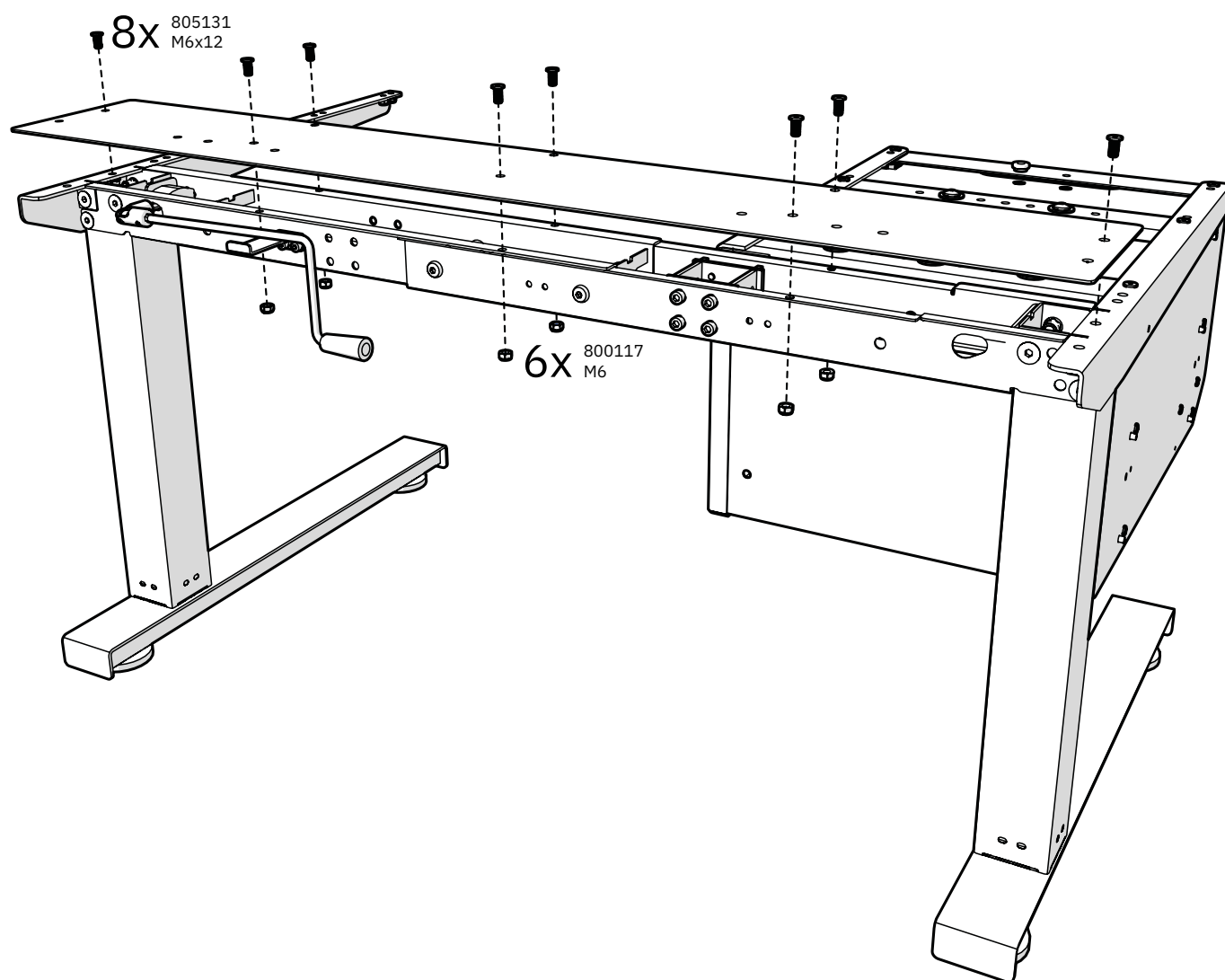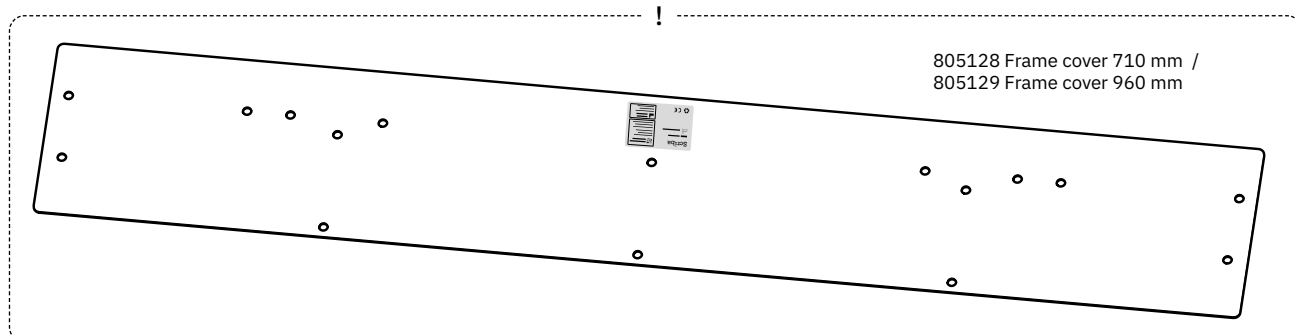

6. Rastomat Bracket


6.1. Mounting the rastomat bracket

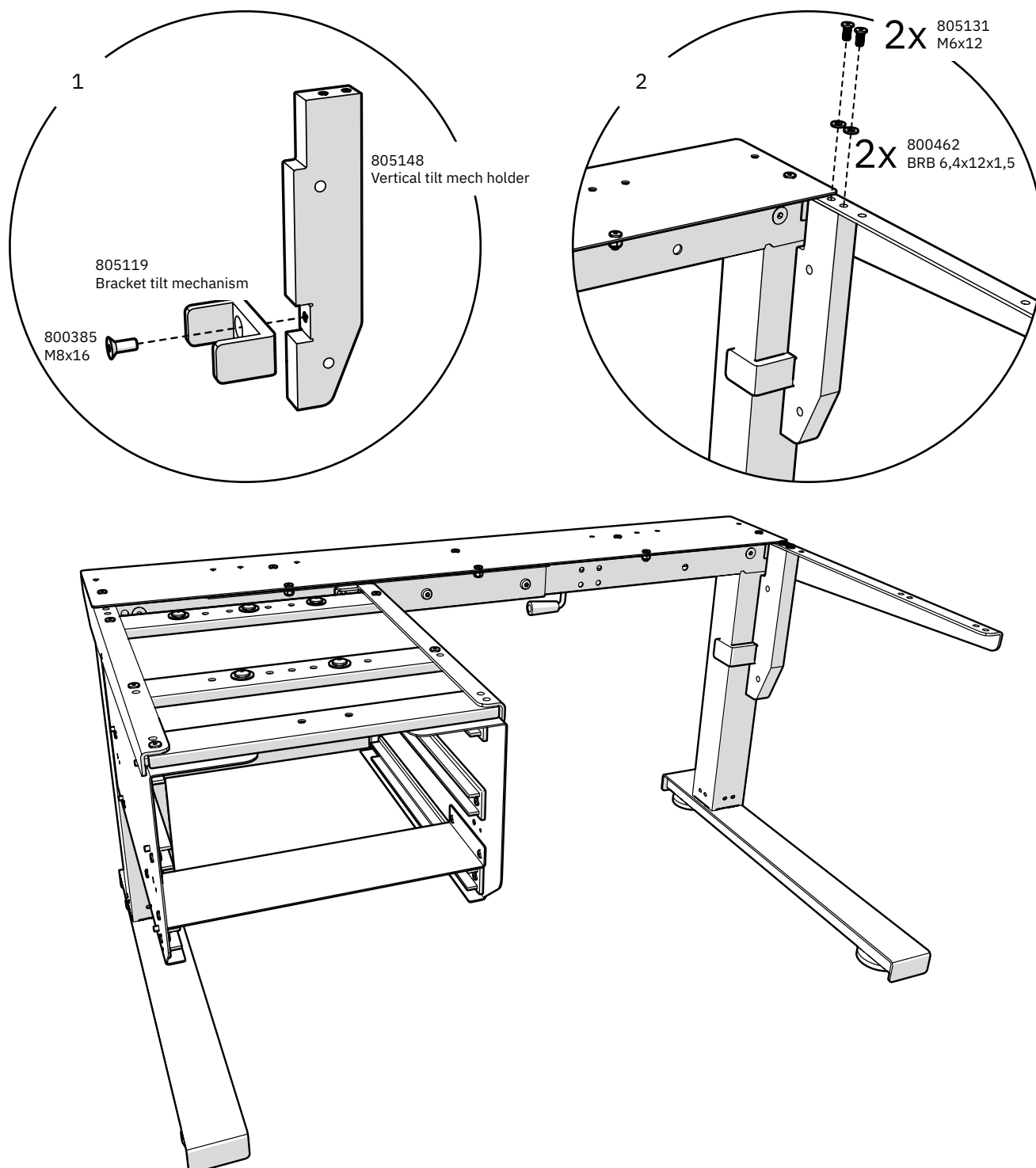


7. Tabletop Attachments

This section covers general assembly procedure for each of the attachments used for the table tops. For placement of the attachments, please refer to section 8. Attachment Points.


7.1. Rotation joint

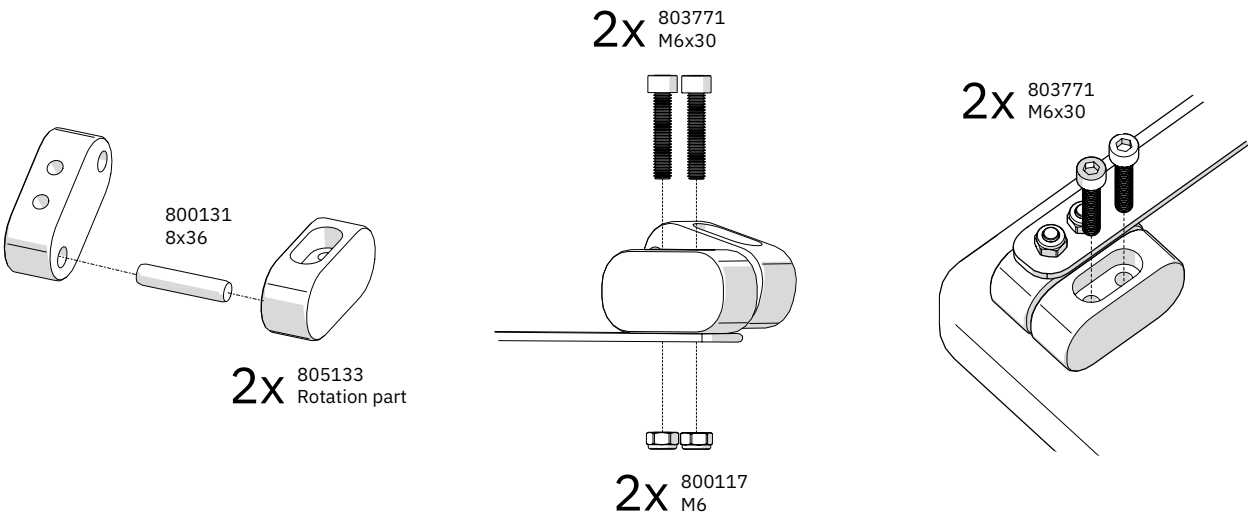

7.2. Tabletop support, fixed


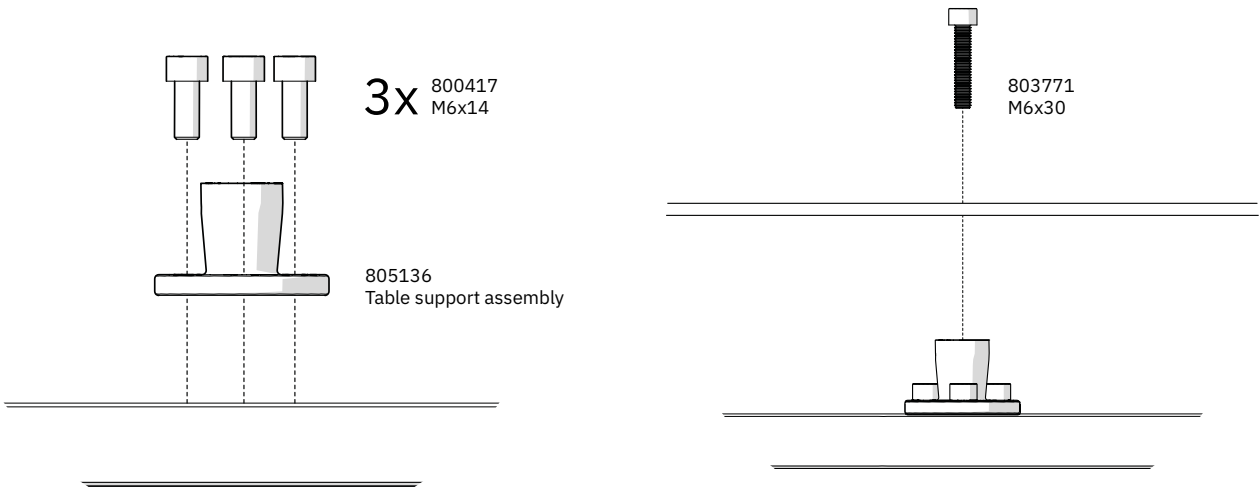


7.3. Tabletop support, furniture pad

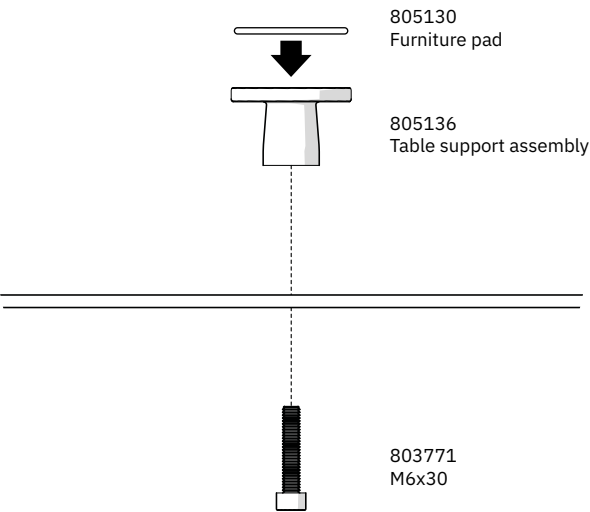


7.4. Spacer (for tabletop support)

! The orientation of the table support distance is depending on its placement, see below.

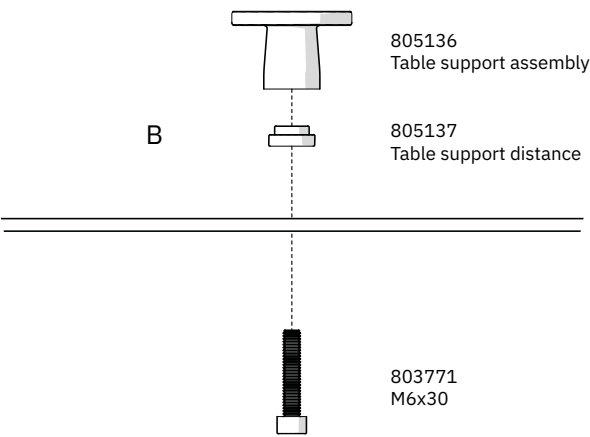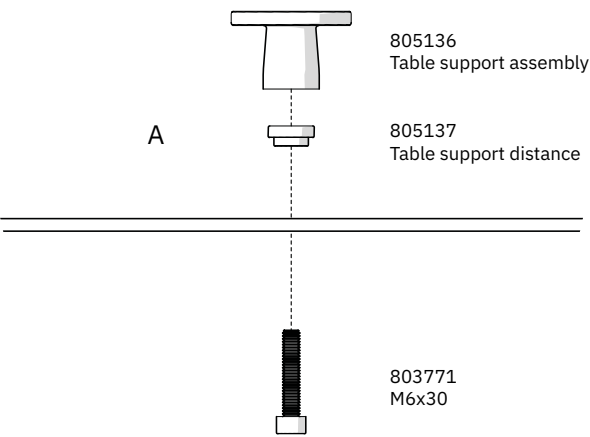


8. Attachments Points

8.1. Possible attachment points on the frame

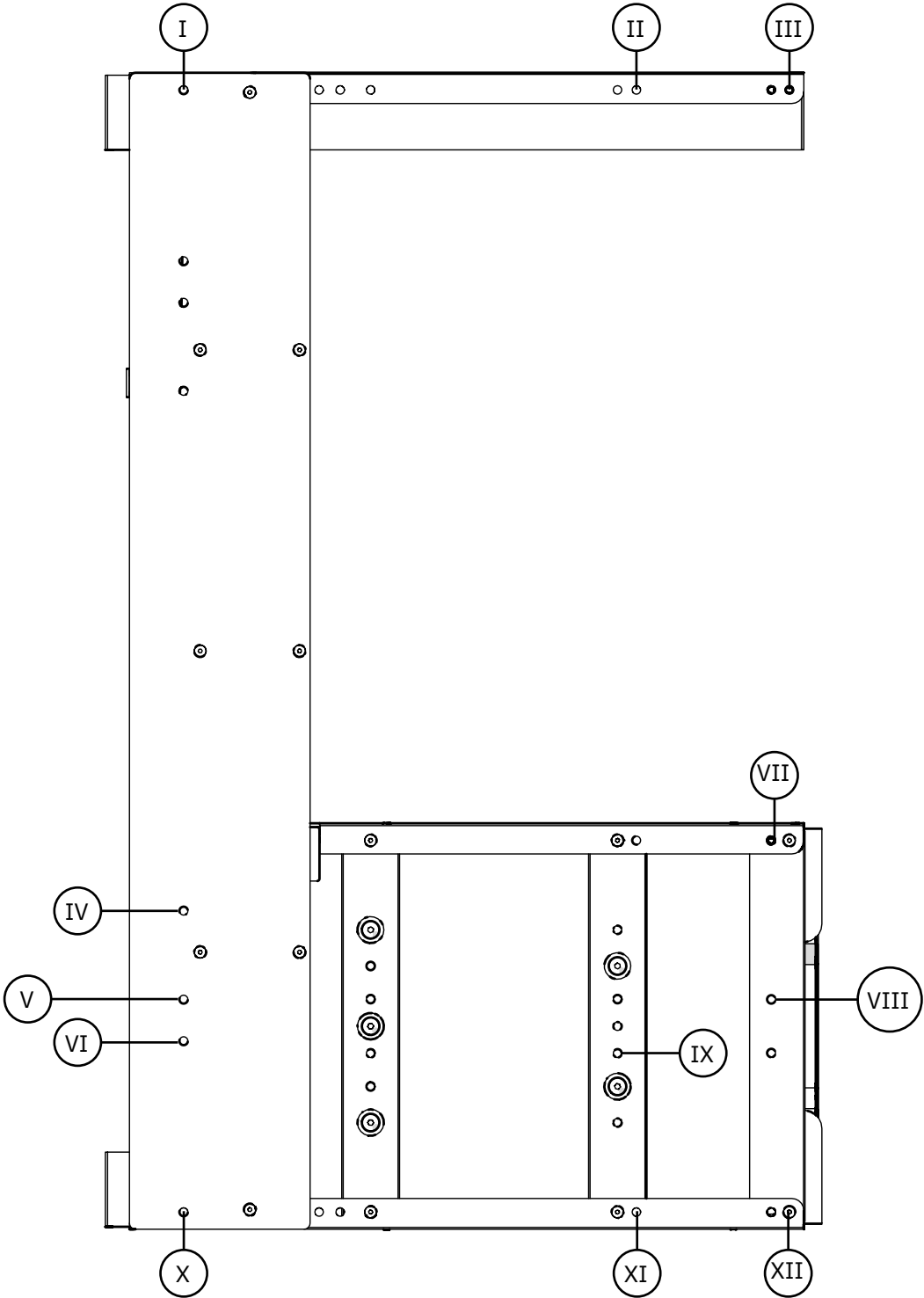

8.2. Tabletop 750x600mm (Flat)

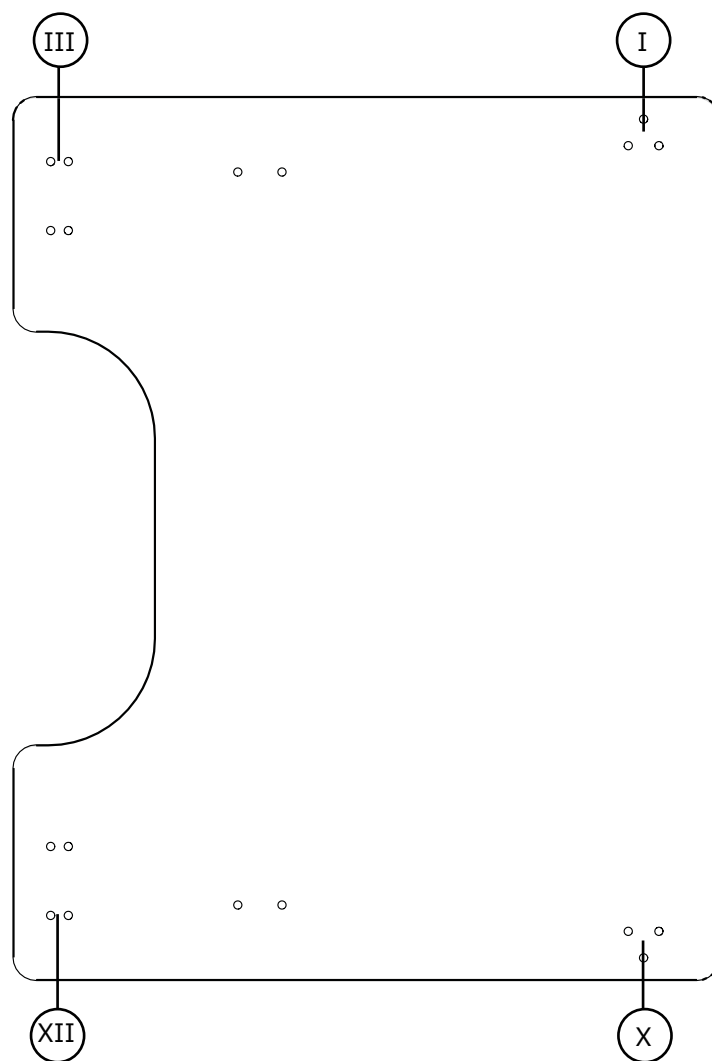

8.3. Tabletop attachments


I - Tabletop support, fixed

III - Rotation joint

X - Tabletop support, fixed

XII - Rotation joint

8.4. Tabletop 750x600mm (Tilt)

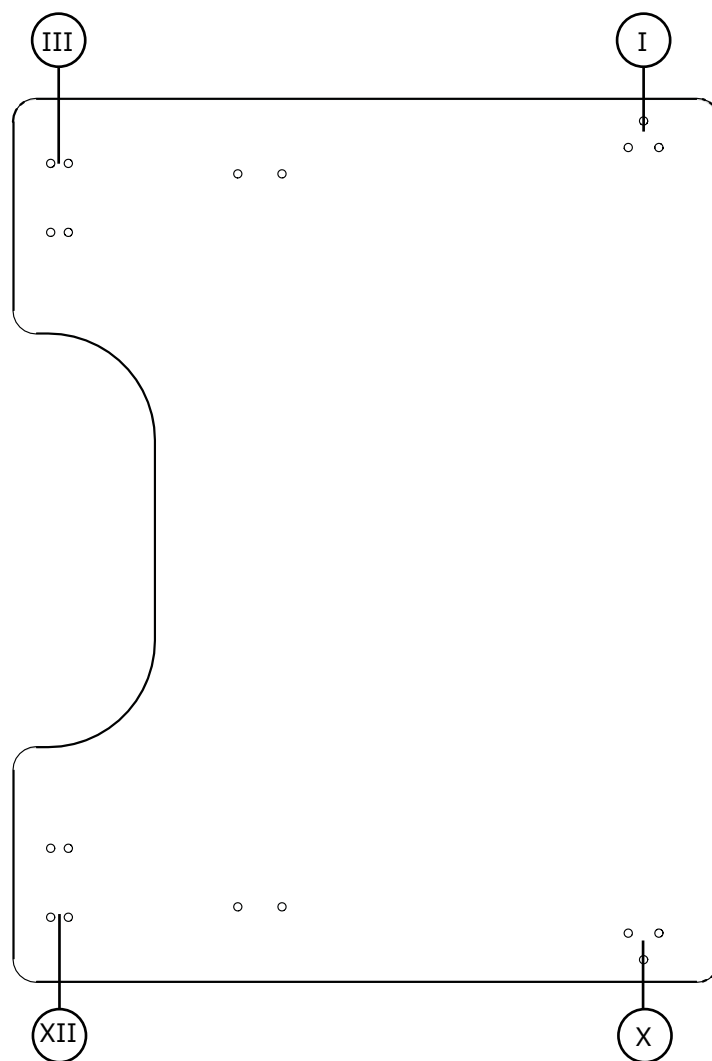

8.5. Tabletop attachments


I - Tabletop support, furniture pad

III - Rotation joint

X - Tabletop support, furniture pad

XII - Rotation joint

8.6. Tabletop 750x600mm (Hinge)

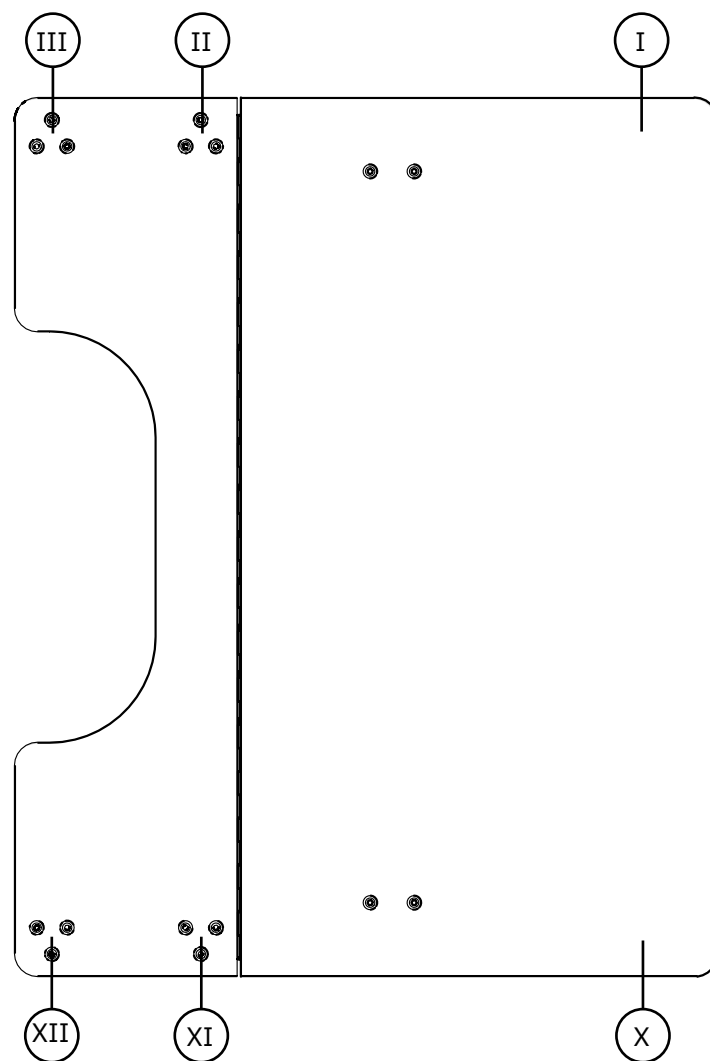

8.7. Tabletop attachments

I - Tabletop support, furniture pad

II - Tabletop support, fixed + Spacer(B)


III - Tabletop support, fixed + Spacer(B)

X - Tabletop support, furniture pad

XI - Tabletop support, fixed + Spacer(B)

XII - Tabletop support, fixed + Spacer(B)

8.8. Tabletop 1000x600mm (Flat)

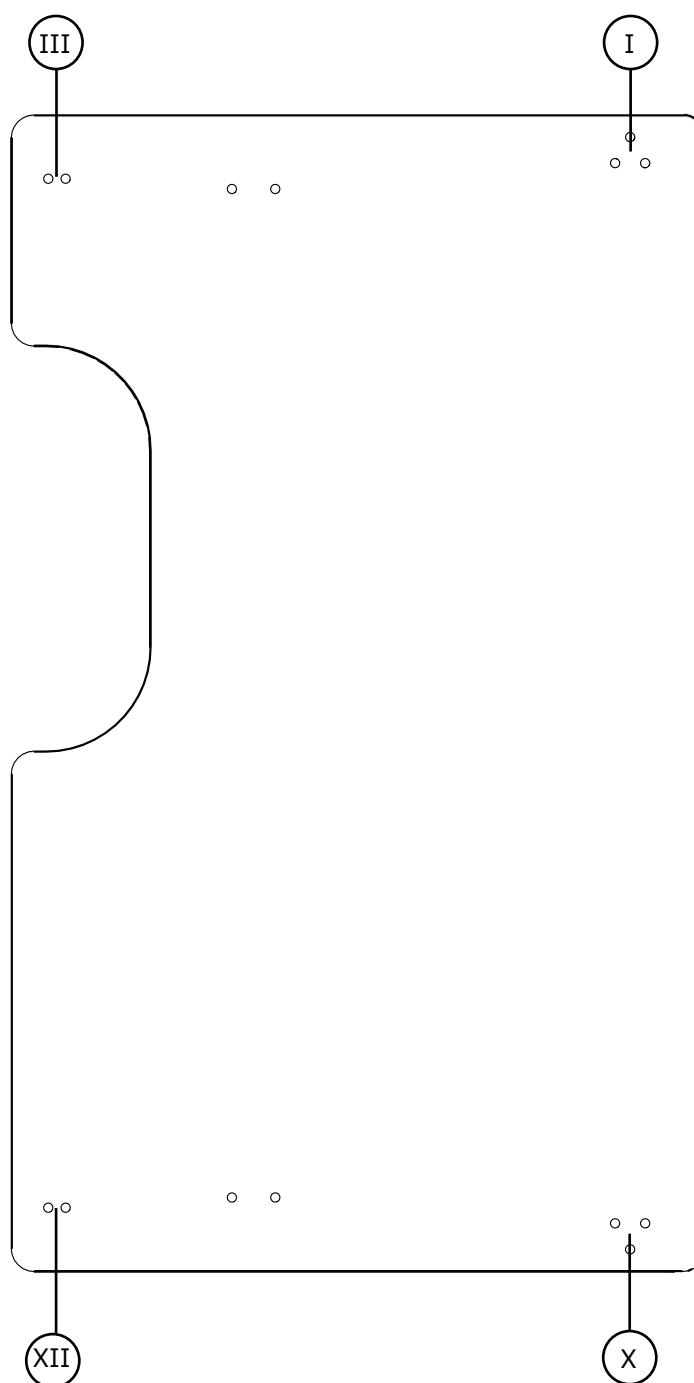

8.9. Tabletop attachments


I - Tabletop support, fixed

III - Rotation joint

X - Tabletop support, fixed

XII - Rotation joint

8.10. Tabletop 1000x600mm (L-shape, hinge)

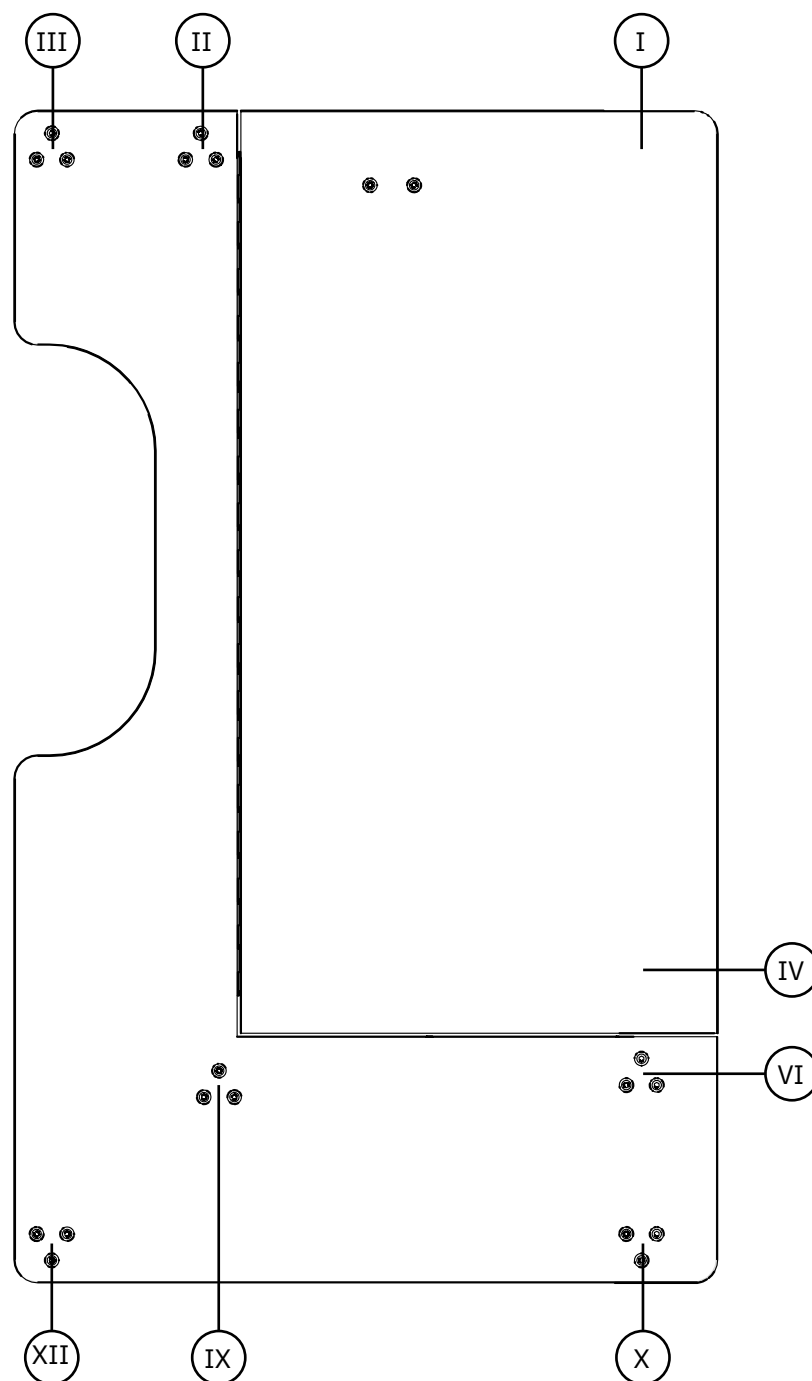

8.11. Tabletop attachments

I - Tabletop support, furniture pad

II - Tabletop support, fixed + Spacer(B)

III - Tabletop support, fixed + Spacer(B)

IV - Tabletop support, furniture pad


VI - Tabletop support, fixed

IX - Tabletop support, fixed + Spacer(A)

X - Tabletop support, fixed

XII - Tabletop support, fixed + Spacer(B)

8.12. Tabletop 1000x600mm (Divided, tilt)

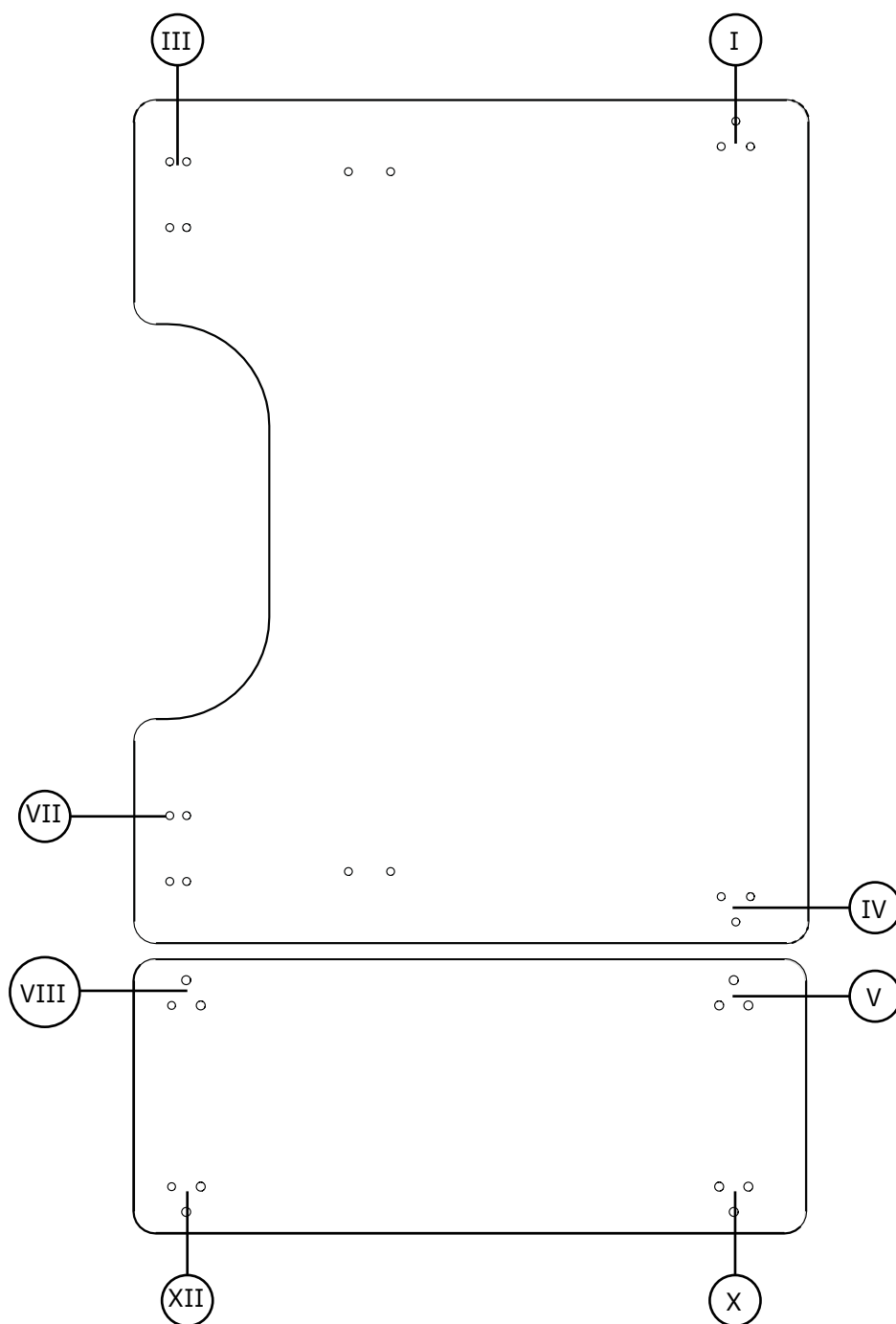

8.13. Tabletop attachments

I - Tabletop support, furniture pad

III - Rotation joint

IV - Tabletop support, furniture pad

V - Tabletop support, fixed

VII - Rotation joint


VIII - Tabletop support, fixed + Spacer(A)

X - Tabletop support, fixed

XII - Tabletop support, fixed + Spacer(B)

8.14. Tabletop 1000x600mm (Centered, hinge)

NOTE! This tabletop must be used with tilt mechanism on both sides.

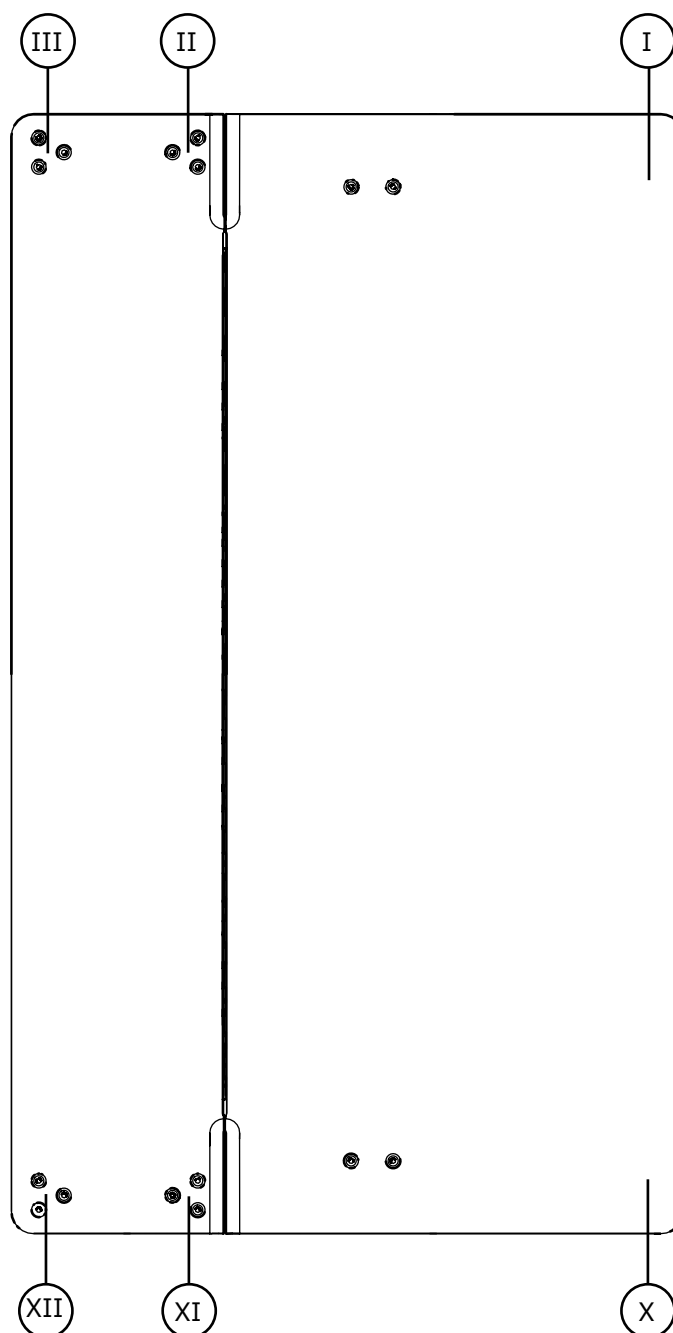

8.15. Tabletop attachments

I - Tabletop support, furniture pad

II - Tabletop support, fixed + Spacer(B)


III - Tabletop support, fixed + Spacer(B)

X - Tabletop support, furniture pad

XI - Tabletop support, fixed + Spacer(B)

XII - Tabletop support, fixed + Spacer(B)

8.16. Tabletop 1000x600mm (Centered, flat)

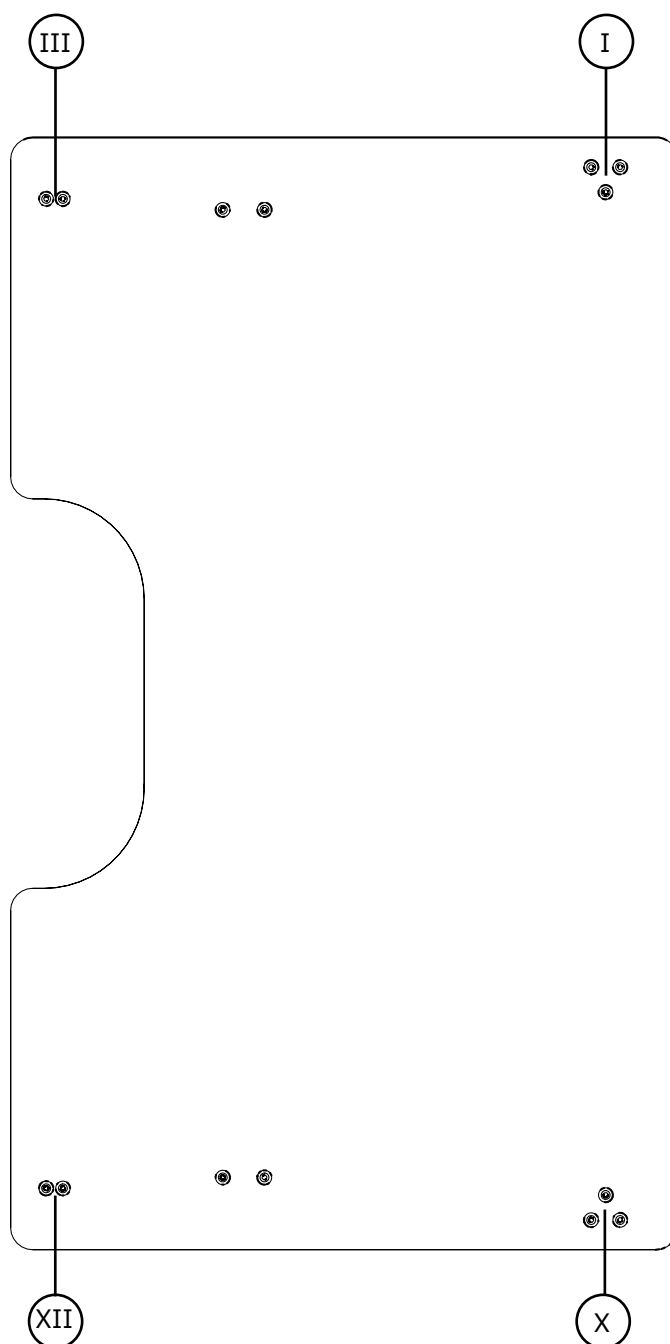

8.17. Tabletop attachments

I - Tabletop support, fixed


III - Rotation joint

X - Tabletop support, fixed

XII - Rotation joint

8.18. Tabletop 1000x600mm (Centered, tilt)

NOTE! This tabletop must be used with tilt mechanism on both sides.

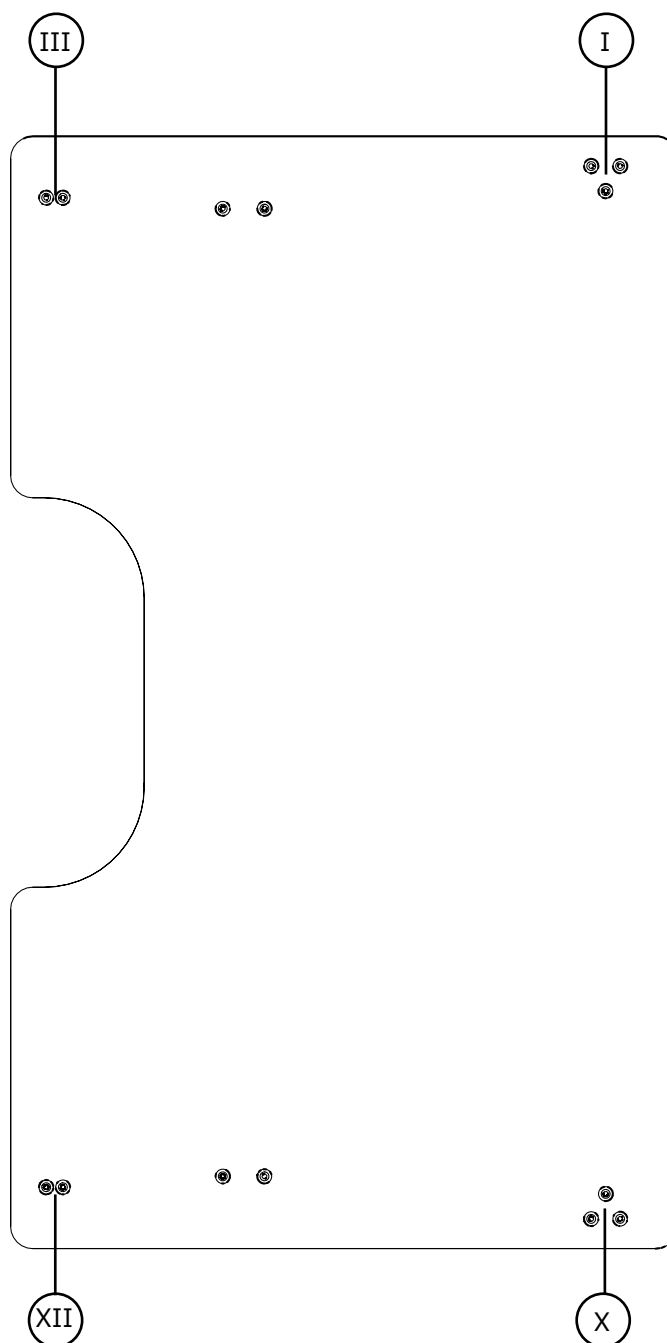

8.19. Tabletop attachments

I - Tabletop support, furniture pad





X - Tabletop support, furniture pad

III - Rotation joint



XII - Rotation joint

9. Tilt mechanism

9.1. Mounting the tilt mechanism

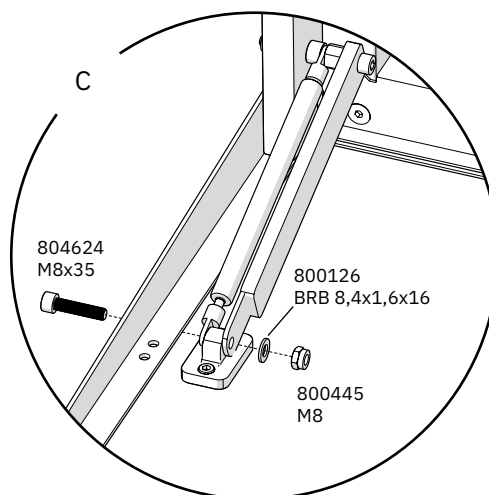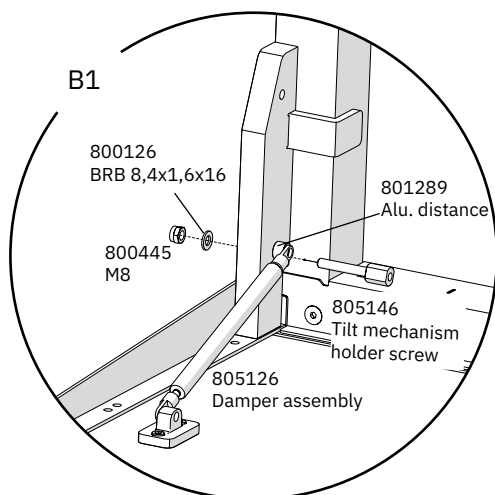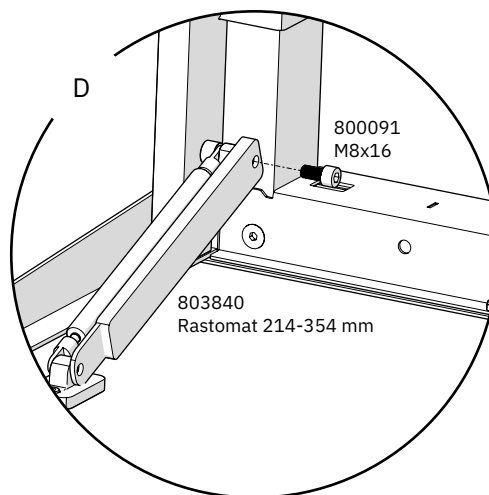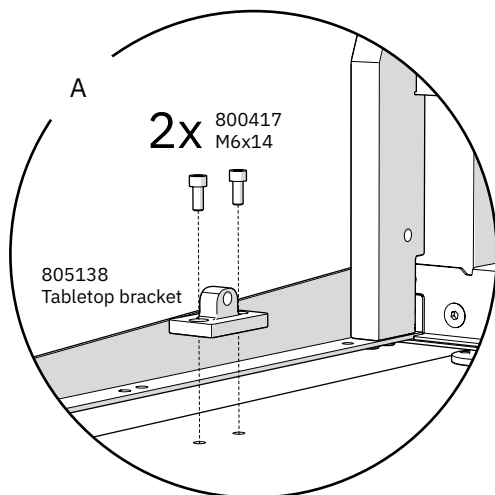

NOTE! The damper is to be mounted as per shown in B2 when using a hinge folded tabletop.

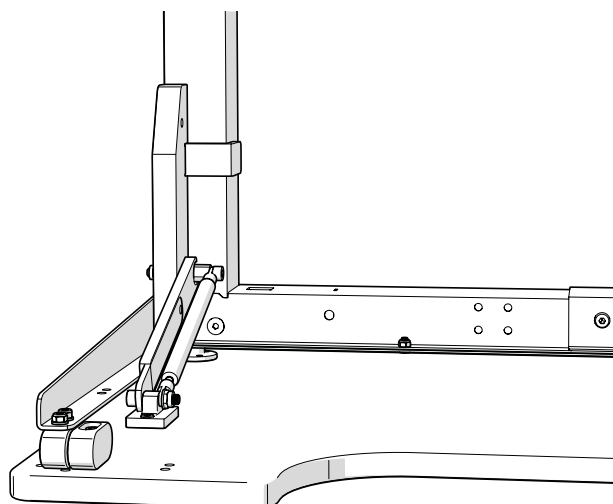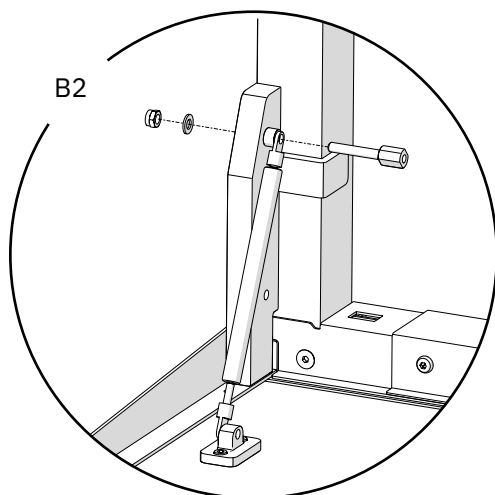

Contact information

Mercado Medic Nederland BV



Mercado Medic Nederland BV

Nobelweg 22

3899 BN Zeewolde



Phone: +31 36 521 99 95

Fax: +31 36 521 99 90



www.mercado-medic.nl

admin@mercado-medic.nl

Room for product label
with serial number.



Mercado Medic AB
Tryffelslingan 14
SE-181 57 Lidingö
SWEDEN

Phone: +46 8 555 143 00
E-mail: info@mercado.se