

Limitations on CE Marking Regarding Third-Party neckrests

To use third-party neckrests while maintaining CE marking on the product series **REAL 9000 PLUS**, **REAL 6100 PLUS**, and **REAL 6100 PLUS LiNX** from Mercado Medic AB, the following restrictions and limitations apply during assembly:

Mounting Requirements

The headrest must be mounted on Mercado Medic's high backrest or on the high Slingback backrest mechanism.

When ordering a high backrest with hole pattern compatibility for "Anatomic Sitt" neckrests, you need to specify the desired size and configuration of the backrest.

Additionally, complete the order with Article 982783 to ensure the backrest plate is delivered with a complete hole pattern.

Custom Hole Pattern Placement

If a different placement of the hole pattern is required (than the default described below), this must be specified in the order.

982783: Hole Pattern: M5 55x65, 40 mm from the upper edge of the backrest unless otherwise specified.

Slingback 40 cm Backrests

For Slingback backrests measuring 40 cm in height, the neckrest mount is attached to the upper crossbar using a clamp connection.

805665: Headrest Mount for Anatomic Sitt neckrest on Slingback Backrests (complete).

If the above criteria are met, the CE marking of the product is maintained. This means that the product can be restored to its original condition without requiring approval from Mercado Medic. However, the product still has a maximum expected lifespan of 10 years and will require an inspection thereafter, as outlined in Chapter 6.3 of the user manual for the aforementioned products.