



Restrictions within CE marking regarding third party Trunk support.

To use a third-part trunk supports with maintaining CE marking on the REAL 9000 PLUS, REAL 6100 PLUS and REAL 6100 PLUS LiNX product series from Mercado Medic AB, the following restrictions needs to be applied during installation.

Assembly must be done with Mercado Medic's rail system for Trunk support mounted on Mercado Medic's back rests intended for this purpose, with M6 thread for the rail system.

805575 Rail 300mm complete for accessories on back rest (Pcs)

805598 Rail 200mm complete for accessories on back rest (Pcs)

Example of Back Plate for Rail System

S-BR1010XX: Backrest ErgoMedic 46x43 for rail system.

- (S- = Threaded hole pattern for rail system)
- (XX = Color code): Currently available in the following:
 - 45 = Plush Real Blue
 - 46 = Plush Gray
 - 48 = Plush Black

If the above criteria are met, the CE marking can be maintained on the product. This means that the product can be restored to its original condition without requiring Mercado Medic's approval. However, the product still has a maximum expected lifespan of 10 years, and then requires an inspection according to chapter 6.3 in the instructions for use for the above-mentioned products.