

Article number

801213	Castor 100 XL M12
801144	Castor 100 SL M12
803162	Castor 100 BXL M12
800904	Castor 125 mm
800905	Castor 150 mm

Necessary Tools

Ring spanner 19 mm

Loctite 243

Instructions

Remove the old castor using a 19 mm ring spanner too loosen the bolt. An optional method is to remove just the wheel using two 5 mm allen keys and then remove the bracket with a socket wrench.

The bolts should be secured using Loctite 243 (or a similar product) and a torque of 80-90 Nm. Mount the new Castor.

