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ECN-S08735

Datum: 2025-04-23

Utfärdare: Fredrik Nyblom

Sida 1 av 3

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
Issued by: Fredrik Nyblom

Page 1 of 4


UPDATE REGARDING CHARGING AND BATTERIES FOR THE ELECTRIC WHEELCHAIR REAL 6100 PLUS

Information about changes in Mercado Medic's products or services.


ECR(s): S08735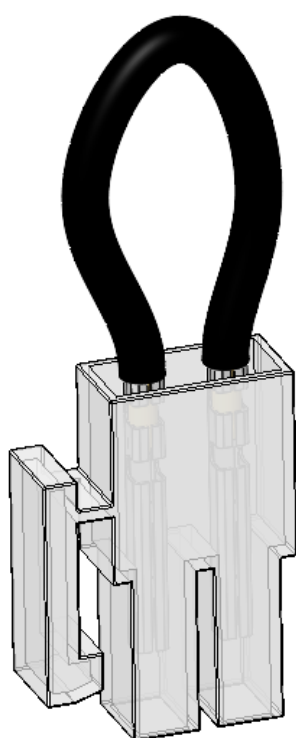

Picture 1 old jumper

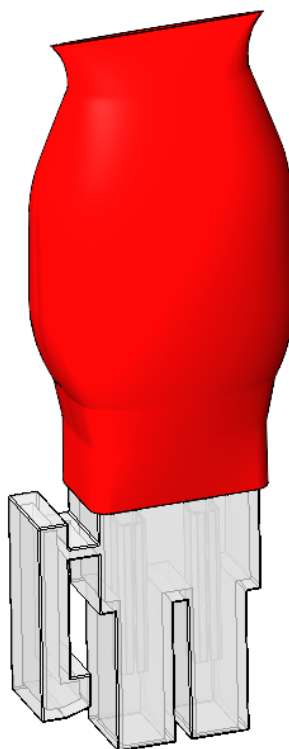Picture 2 New jumper

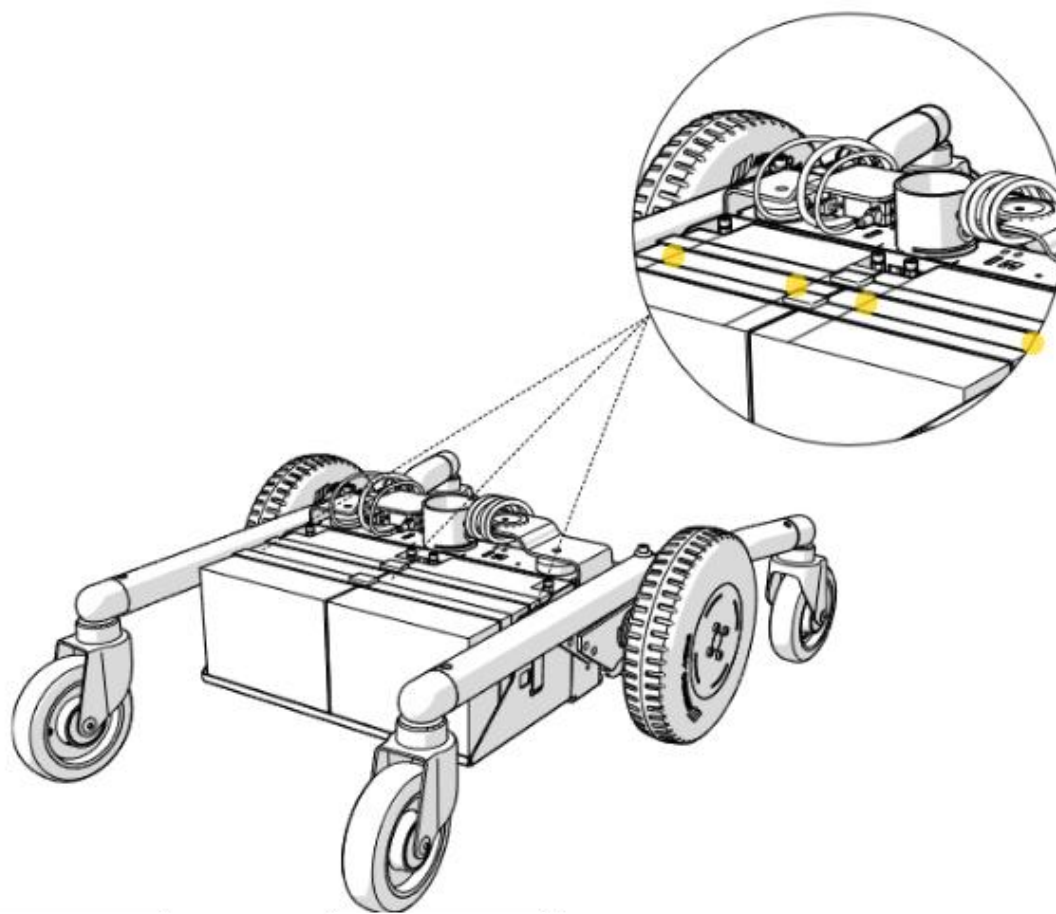Picture 3 Ventilation holes

Summary

Mercado Medic has identified corrosion and swelling on the batteries of the power wheelchair REAL 6100 PLUS. To improve the lifespan of our batteries, we are implementing a number of updates.

Affected article numbers

600101	Contact Mobil 6100 cpl Molex
804023	Cord for battery L 1200mm W 18mm f Real 6100 Plus
BAC1011	Battery Charger 6100 PLUS UK
BAC1010	Battery Charger 6100 PLUS

These articles have been revised according to the explanation below.

Background

Together with our suppliers, we have analysed data related to the affected batteries and have identified several causes, such as:

-Use of battery chargers with higher current/voltage or different charging programs than what Mercado Medic's original charger, item no. BAC1010/BAC1011, is approved/specified for. Please use only our tested original charger to maintain CE certification.

-The tension straps used to secure the batteries in place may cover the ventilation holes.

As part of resolving this issue, a number of updates will be implemented.

Description of the change

Implemented solution 1

Under the battery cover on a REAL 6100 PLUS, there is a bracket (see image 1) on the brake release cable of the right drive motor. This bracket has shown signs of discoloration. To resolve this, we have applied red heat shrink tubing (see image 2). The red colour is used to avoid confusion with the bracket located under the seat frame, which has black heat shrink tubing. We have also changed the colour of the cable under the heat shrink tubing to red.

Implemented solution 2

We have also replaced the material of the buckle on the straps with stainless steel to further reduce the risk of corrosion.

Implemented solution 3

Our batteries are of the gel type and should not emit any gases during normal charging. However, if charging is performed with excessive current/voltage gas may be released. This occurs through small vents located on the top of the batteries. The yellow markings (see image 3) indicate their approximate location. We have updated our instructions to emphasize that these vents must not be covered by straps during installation or battery replacement. Please refer to the updated instruction manual 'Battery Replacement REAL 6100 PLUS' under Assembly Instructions on our website: [Documents – Mercado Medic](#).

Implemented solution 4

Our chargers have received a software update that makes charging gentler and more efficient.

Estimated time for implementation

BAC1010/BAC1011 Q2 2025, subject to final use of items in stock. All other solutions have been implemented.

Compatibility

None of these changes affect compatibility.

Safety information

No known risks are associated with this change.

Questions

If you have any questions, please contact our Product Manager Ronny Fogelqvist.

Contact Information:

+46 708 27 96 14

ronny.fogelqvist@mercado.se