


Limitations within CE Marking Regarding Third-Party Joystick Grips


To use third-party joystick grips with maintained CE marking on the indoor power wheelchair, Class A product series REAL 6100 PLUS LiNX from Mercado Medic AB, the following restrictions and limitations apply:

- **The joystick grip must be firmly press-fitted** onto the joystick shaft and must not be removable by pulling.
- **The two wings for rotation locking** on REM 211, ACU200, and CR400/CR400LF may be cut off to apply a third-party joystick grip. **After this, the control box cannot be restored to its original state since the original grip rotates without the wings.**
- **Applies to joysticks listed in the tables below.** The weight and height of the third-party joystick grip must not exceed the maximum limit, as the joystick must always be able to return to neutral position.
- **Mounting:** Press the joystick **all the way down** onto the control box shaft until it **bottoms out below the wings (for rotation lock)**.
- **Rotation Locking:** If no suitable slot exists in the joystick grip, it is permitted to mill slots for the wings to achieve rotational locking, provided the material and the supplier's instructions allow for this.
- **Inspection:** Ensure zero play, full range of motion without friction, and that any locking mechanism works according to specification. Any machining is carried out at the user's own risk.

Maximum weight in relation to height: REM211, REM400, CR400, ACU200.

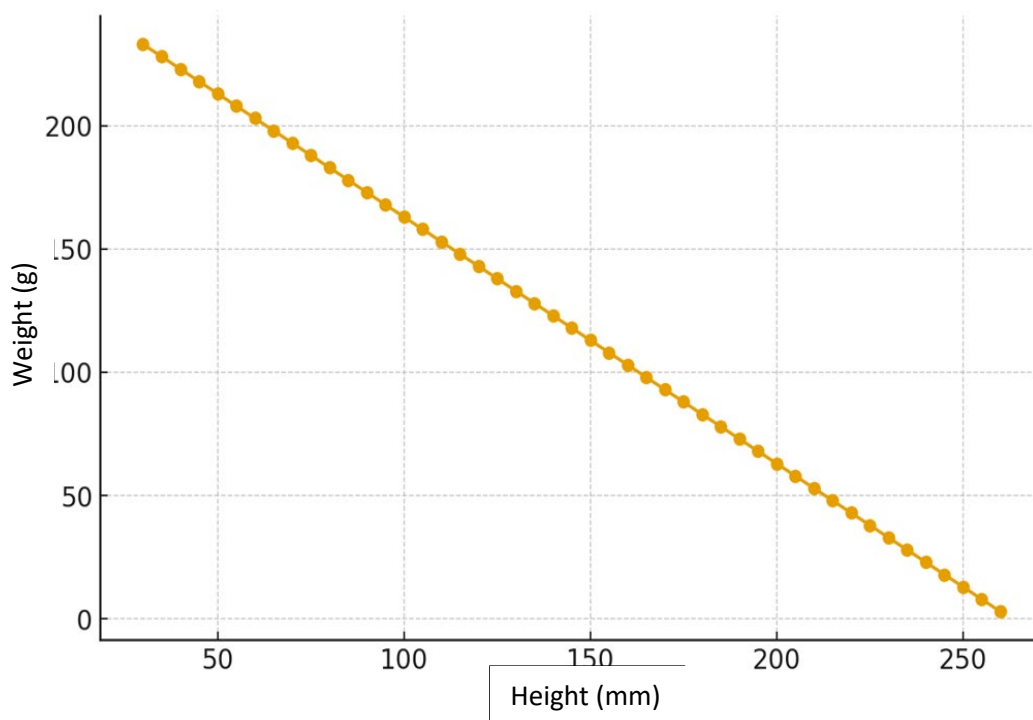

Maximum weight in relation to height: CR400LF (LF = Low Force)

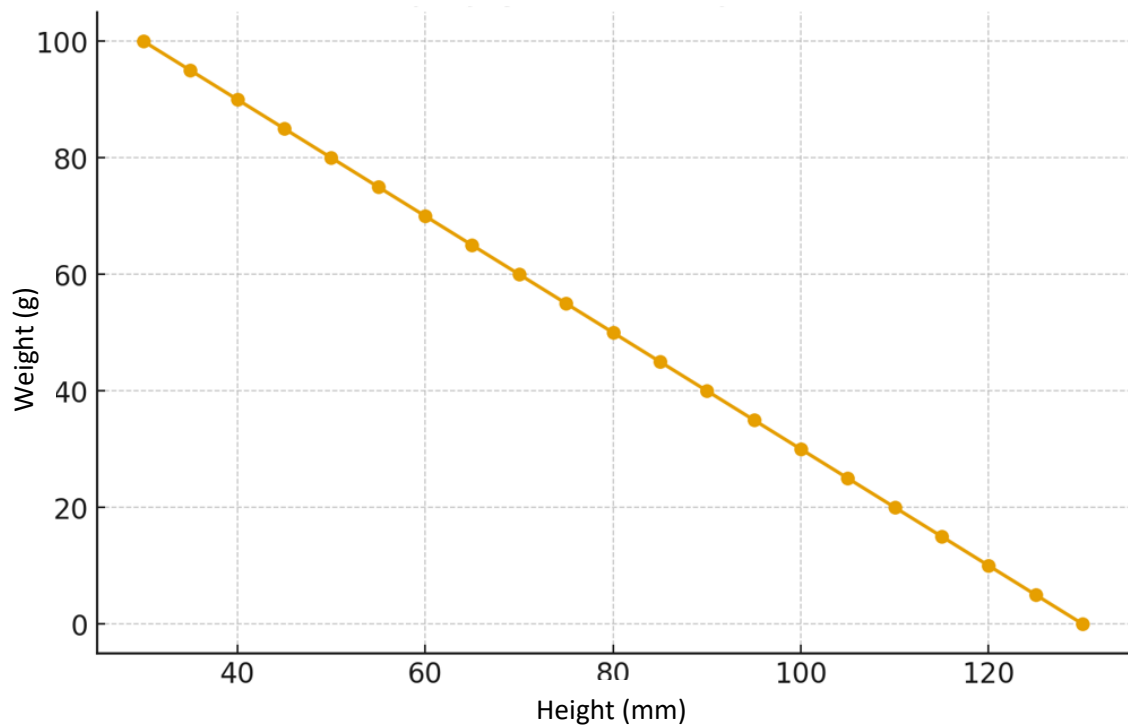

Compliance Statement

If these criteria are met, the **CE marking can be maintained** on the product. This means the product can be restored to its original condition **without requiring approval from Mercado Medic**. Please note that the product still has an **expected service life of up to 10 years** and must undergo an inspection thereafter **in accordance with Chapter 6.3 of the user manual** for the products.

Questions

For any questions regarding this document, please contact our Product Manager:

Ronny Fogelqvist

Phone: **+46 708 27 96 14**

Email: **ronny.fogelqvist@mercado.se**